




PCEye 5

Getting Started Guide

What's in the box?

- | | |
|-------------------------|----------------------------------|
| ❶ PCEye 5 | ❺ Safety and Compliance Document |
| ❷ Getting Started Guide | ❻ Carry Case |
| ❸ Mount Plate | ❼ USB-C to USB-A Adapter |
| ❹ Mount Plate Prep Kit | |

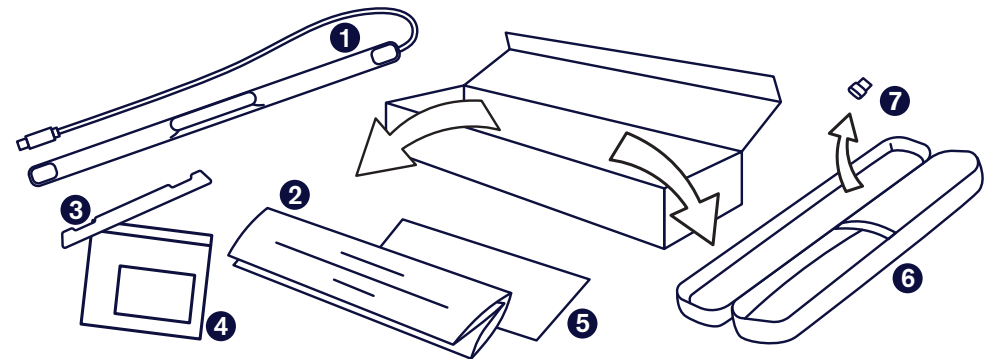


Software

PCEye 5 includes the TD Control Software. If you purchased the Magic EyeFx bundle, please see the license card for more information.

Getting to Know Your Device

- | | |
|-----------------|-------------------------|
| ❶ Eye Tracker | ❸ Installation Markings |
| ❷ USB Connector | ❹ Mounting Magnets |

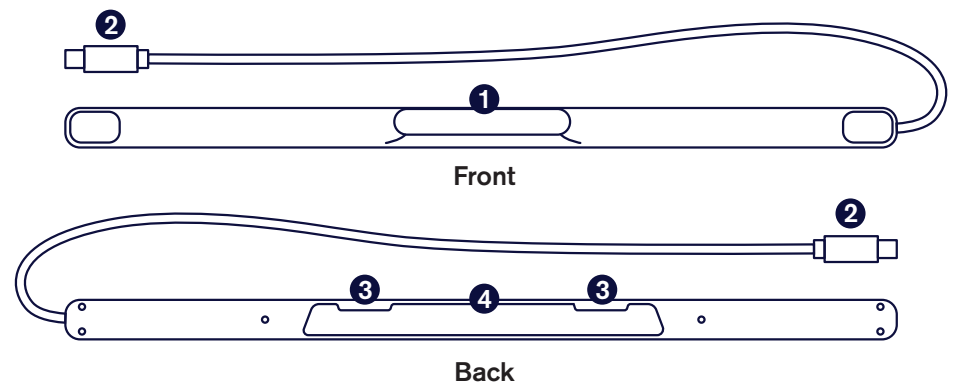

We Are Here to Help

Learning Hub

Tobii Dynavox Learning Hub:
learn.tobiidynavox.com

Additional Support

Community: <http://qrco.de/TDFB>

North America Technical Support:
1-800-344-1778 ext. 1

Training Materials

User Manual: <http://qrco.de/PCEyeManual>

TD Control Manual: <https://qrco.de/TDCmanual>

Communicator 5 Training Cards: <https://qrco.de/C5Cards>

TD Snap Core First Training Cards: <https://qrco.de/TDSnapCards>

TD Snap Text Training Cards: <https://qrco.de/TextCards>

Additional PCEye 5 resources are available here:


qrco.de/PCEyeHelp



Getting Setup

Complete the three steps in this guide to download the required software and mount the PCEye 5 device to your computer. Downloading the software FIRST will provide onscreen instructions and walk you through the steps for set up and use.

You can use the PCEye 5 on a laptop or desktop computer.

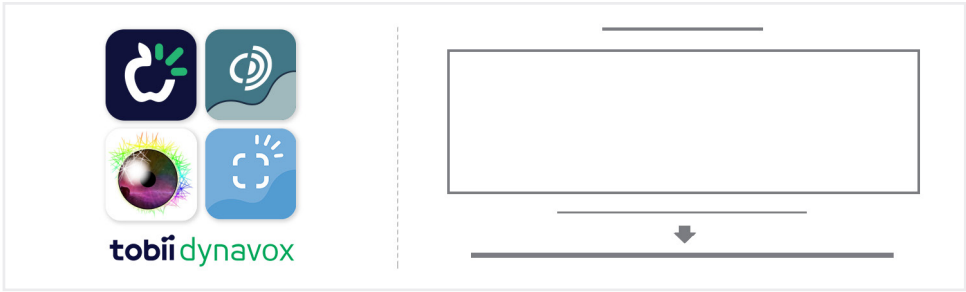

Step 1: Download PCEye 5 Software

Go to <https://qrco.de/PCEyeCC> and download PCEye 5 Software.

Step 2: Mount PCEye 5 & Calibrate

If using the PCEye 5 with the optional PCEye 5 Bracket, skip this step and see included instructions for detailed information.

Follow the onscreen prompts. Return to guide for more information about the software.

Step 3: Launch Software

If you purchased a software bundle or received additional license cards, follow the download and installation instructions on each card (see left).

Getting Started with TD Control Software

Paired with the PCEye 5, TD Control will give you access to mouse and keyboard functions. When TD Control is opened for the first time, the Quick Start Guide will provide an overview of the different actions.

You can return to the Quick Start Guide by selecting the Information button.

Select Hide Guide to exit.

Explore the Trace

The Trace will give you feedback about where you are looking on the screen. When you fixate on an item, the Activator button will appear. Fixating on the Activator button will open the Interactions Menu.



Explore Interactions Menu

The Interactions Menu provides access to the keyboard and primary mouse functions.



For detailed description of each function, please see the User Manual.

Explore the Off-Screen Menu

The Off-Screen Menu gives you access to additional features, such as Continuous Click, Settings, Pause, etc.

To recalibrate, access the Off-Screen Menu, select More, then select Quick Calibrate.

As you explore TD Control, it is OK to recalibrate if your accuracy is not optimal.

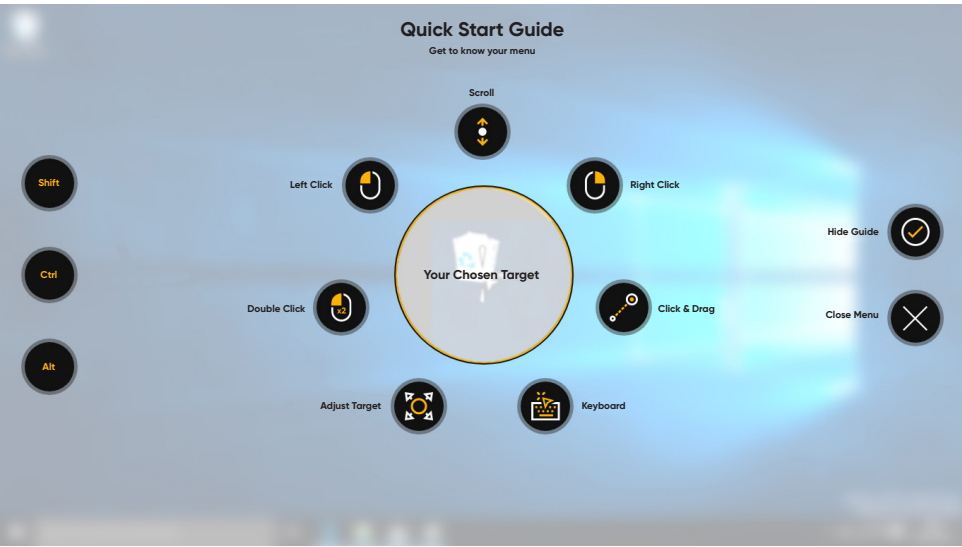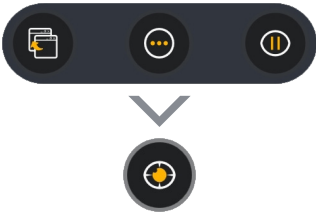

Helpful Tips for Using TD Control

- Use Pause when you need to rest your eyes.
- The desired item doesn't need to be perfectly placed in the center of the Trace.
- To increase accuracy, use Adjust Target.
- When using Scrolling, the farther away from the scroll icon your eyes are looking, the faster the scroll will go. Scroll works up and down and side to side.
- For typing into text fields, place the Trace in the text field, then select Keyboard. No left click is required.



Pause



Adjust Target