
tobiidynavox

Speech Case Keyguard Holder

Installation Instructions

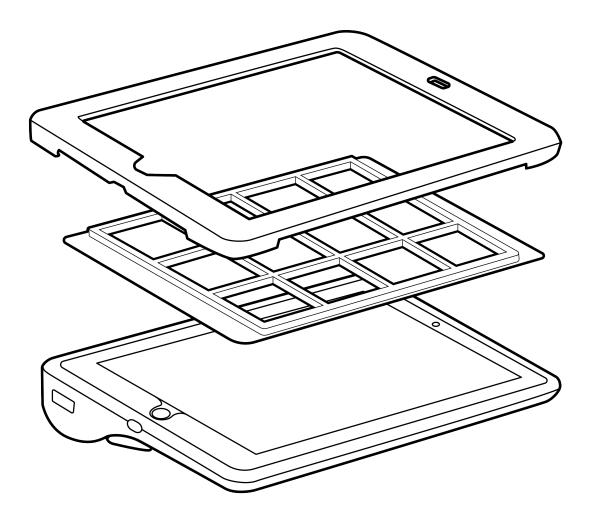



Before installing the keyguard holder, the iPad must be installed in the Speech Case as specified in the Tobii Dynavox Speech Case and SC Tablet Installation Instructions document.



Your Tobii Dynavox Speech Case comes with a keyguard holder that should be installed even if you are not using a keyguard. The keyguard holder stabilizes the iPad inside the Speech Case.


Support for Your Tobii Dynavox Device


Get Help Online
See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller


tobiidynavox


- 1. Place the optional keyguard on the iPad with the raised edges of the keyguard facing up.
- 2. Press the keyguard holder onto the Speech Case/iPad/ keyguard assembly.

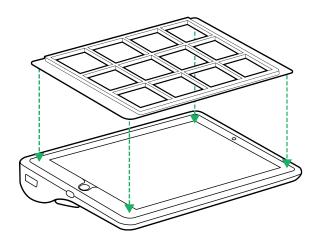

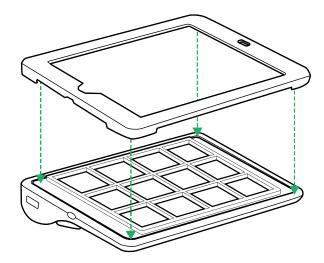

- 3. When properly positioned, the keyguard holder will snap in place.
- 4. Wiggle the optional keyguard in a gentle circular motion with your fingers until it locks in place. Your key guard is now ready to use.

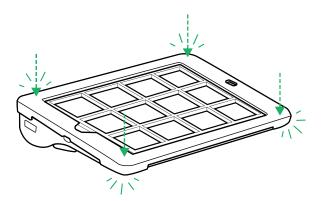

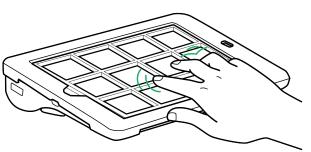

Support for Your Tobii Dynavox Device

Get Help Online
See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller