

TD Browse

User's Manual



User's Manual TD Browse

Version 1.0

2023-04-11

All rights reserved.

Copyright © Tobii Dynavox AB (publ)

No part of this document may be reproduced, stored in a retrieval system, or transmitted in any form, by any means (electronic, photocopying, recording, or otherwise) without the prior written permission of the publisher.

Copyright protection claimed includes all forms and matters of copyrightable material and information allowed by statutory or judicial law or hereafter granted, including without limitation, material generated from the software programs which are displayed on the screen such as screen displays, menus, etc.

The information contained in this document is proprietary to Tobii Dynavox. Any reproduction in part or whole without prior written authorization by Tobii Dynavox is prohibited.

Products that are referred to in this document may be either trademarks and/or registered trademarks of the respective owners. The publisher and the author make no claim to these trademarks.

While every precaution has been taken in the preparation of this document, the publisher and the author assume no responsibility for errors or omissions, or for damages resulting from the use of information contained in this document or from the use of programs and source code that may accompany it. In no event shall the publisher and the author be liable for any loss of profit or any other commercial damage caused or alleged to have been caused directly or indirectly by this document.

Content subject to change without notice.

Please check Tobii Dynavox website.

www.TobiiDynavox.com for updated versions of this document.

Contact Information:

Tobii Dynavox AB
Karlsrovägen 2D
182 53 Danderyd
Sweden

+46 8 663 69 90

Tobii Dynavox LLC
2100 Wharton Street, Suite 400
Pittsburgh, PA 15203
USA

+1-800-344-1778

Tobii Dynavox Ltd.
Sheffield Technology Parks
Cooper Buildings
Arundel Street
Sheffield S1 2NS
United Kingdom

+46 8 663 69 90

Tobii Dynavox (Suzhou) Co. Ltd
Unit 11/12, Floor 3, Building B, No.5 Xinghan Street, SIP,
Suzhou
P.R.China 215021

+86 512 69362880



Table of Contents

1	About TD Browse	5
1.1	Intended Use.....	5
2	First Start Up.....	6
3	Overview of TD Browse	7
3.1	The Navigation Bar	7
3.2	Off-Screen Menus in TD Browse	7
3.2.1	The Off-Screen Menu.....	7
3.2.1.1	The Multimedia Off-Screen Menu	8
3.3	The Dashboard	8
3.4	Home Page	9
3.5	Track Status Box.....	9
4	Using TD Browse.....	10
4.1	The Actions in the Main Screen	10
4.1.1	Scroll Mode in TD Browse.....	10
4.1.1.1	How do I Scroll a whole Web Page in TD Browse?	10
4.1.1.2	How do I Scroll a portion of a Web Page in TD Browse?.....	11
4.1.2	Click Mode in TD Browse	11
4.1.2.1	How do I Select a link in Click Mode in TD Browse?.....	11
4.1.2.2	How do I make a selection in Drop-down menus in TD Browse? ...	11
4.1.3	Navigation Bar in TD Browse	12
4.1.3.1	How do I Type in the URL in TD Browse?.....	13
4.1.3.1.1	Keyboard	14
4.1.3.2	Home Page in TD Browse.....	15
4.1.3.2.1	How do I Access the Home Page in TD Browse?	15
4.1.3.3	Zoom in TD Browse.....	16
4.1.3.3.1	How do I Zoom in or out on a Web Page in TD Browse?	16
4.2	Actions in the Off-Screen Menu	17
4.2.1	How do I Activate Switcher in TD Browse?	17
4.2.2	How do I Activate the Dashboard in TD Browse?	17
4.2.3	Pause in TD Browse	18
4.2.3.1	How do I Pause the Eye Tracking in TD Browse?	18
4.2.3.2	How can I See if the Eye Tracking is Paused in TD Browse?	19
4.2.3.3	How do I Resume the Eye Tracking in TD Browse	19
4.3	The Actions on the Dashboard	20
4.3.1	How do I Activate the Dashboard in TD Browse?	20
4.3.2	How do I Activate an Action from the Dashboard in TD Browse?	20
4.3.3	How do I Access the Settings in TD Browse?	21
5	TD Browse Settings.....	22
5.1	Activation	23
5.1.1	Speed	23
5.1.1.1	How do I Select the Fixation Time for the Color Links in TD Browse?	23
5.1.1.2	How do I Change/Set the Dwell Time for the Buttons in TD Browse?	24

5.2	Calibration	24
5.2.1	How does the Calibration Process Work in TD Browse?	24
5.2.2	How do I Check the Positioning of the User using Track Status in TD Browse?	26
5.2.3	How do I Calibrate a User in TD Browse?	26
5.2.4	How do I Change the Display Setup for my Eye Tracker in TD Browse?	27
5.3	Keyboard	28
5.3.1	Buttons	28
5.3.1.1	How do I Change/Set the Dwell Time for the Keyboard in TD Browse?	28
5.3.2	General	29
5.3.2.1	How do I Select my Keyboard Language in TD Browse?	29
5.3.2.2	How do I Download a new Keyboard Language in TD Browse?.....	29
5.4	General	30
5.4.1	How do I Select the Application Language for TD Browse?	30
6	TD Switcher	31
6.1	How do I Access TD Switcher?	31
6.2	How do I Switch to an Application in TD Switcher	31
6.3	How do I Add an Application to TD Switcher?	32
6.4	How do I Remove an Application from TD Switcher?	32

1 About TD Browse

1.1 Intended Use

TD Browse is a custom-built app for browsing the web with just your eyes in a familiar way, even without previous eye tracking experience. It is custom designed for people who cannot use their hands to browse the internet, with conditions such as ALS, CP, spinal cord injury or traumatic brain injury. Scroll, surf and click in a familiar intuitive way and switch easily and naturally between these functions. Hit large and small target with ease and precision plus quickly jump to preferred websites from the browsing history.

TD Browse has been built to encourage quick success for those new to eye gaze. TD Browse design prioritizes the content of the website so that a person feels immersed in their browsing experience. On average, in user testing, it has taken them less than 5 minutes to get up and running with TD Browse.

TD Browse gives people who use eye gaze a simple way to enjoy the web.

2 First Start Up

When TD Browse is launched for the first time, the user will be going through the Configuration and Orientation Experience.. The Configuration and Orientation Experience consists of these things:







1. Term of Services View.
2. The TD Browse Getting Started Tutorial.

3 Overview of TD Browse

3.1 The Navigation Bar



Table 1: The Navigation Bar

Icon	Button Name	Description
	Go Back	Takes you back the previous page.
	Go Forward	Moves you forward to the page that you used the Go Back button from.
	Refresh	Refreshes the active page.
	Keyboard	Opens the Keyboard for typing in a new URL. For more information, see 4.1.3.1 How do I Type in the URL in TD Browse?, page 13
	Home Page	Opens the Home Page. For more information, see 3.4 Home Page, page 9
	Zoom Settings	Opens the Zoom settings for zooming in to the page.

3.2 Off-Screen Menus in TD Browse





There are a several different off-screen menus throughout TD Browse. One thing they have in common is that all of the off-screen menus are contextually based.


3.2.1 The Off-Screen Menu

There is on main off-screen menu called the Off-Screen Menu while others have different names though they are accessed in the same way.



Table 2: The Off-Screen Menu






Icon	Button Name	Description
	Multimedia Mode	Opens the Multimedia Off-Screen Menu. For more information, see 3.2.1.1 The Multimedia Off-Screen Menu, page 8  This button is only visible if a video or music if playing.
	Switcher	Opens Switcher. For more information, see 6 TD Switcher, page 31
	Dashboard	Opens the Dashboard. For more information, see 3.3 The Dashboard, page 8 .

Icon	Button Name	Description
	Pause	Pauses the interaction until the Resume Gaze button is selected.

3.2.1.1 The Multimedia Off-Screen Menu



Table 3: The Multimedia Off-Screen Menu

Icon	Button Name	Description
	Close Multimedia Mode	Closes the Multimedia Off-Screen Menu.
	Play Pause	Starts/resumes or pauses the video or music playing.
	Mute Unmute	Mutes or unmutes the sound.
	Volume Down	Lowers the volume.
	Volume Up	Raises the volume.

3.3 The Dashboard

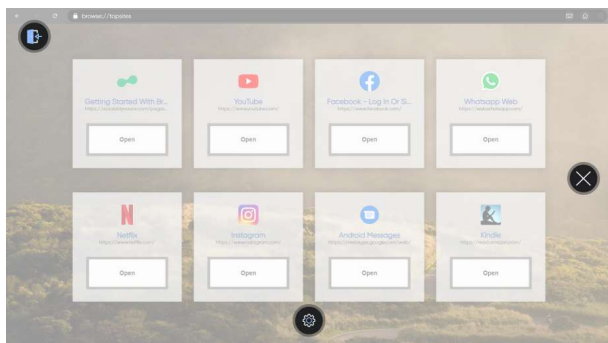





Illustration 1: The Dashboard.

Table 4: The Dashboard

Icon	Button Name	Description
	Settings	Opens TD Browse Settings. For more information, see 5 <i>TD Browse Settings</i> , page 22.

Icon	Button Name	Description
	Exit TD Browse	Closes down, restarts, and exits the TD Browse software.
	Cancel	Closes the Dashboard.

3.4 Home Page

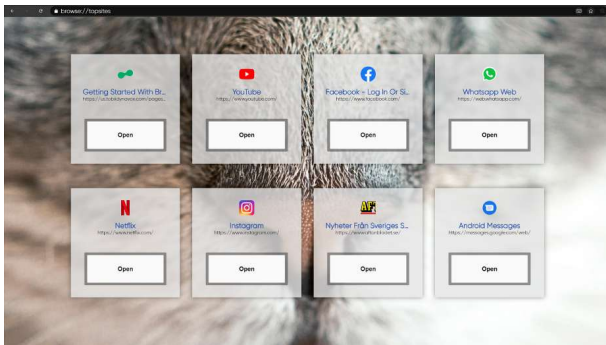


Illustration 2: The Home Page

The Home Page is the first page the user sees when TD Browse is launched, except for first launch. On the Home Page there are eight (8) links to different web pages. The Getting Started Tutorial will disappear automatically after a while, but will remain on the Home Page for easy access in the beginning. The links on the Home Page will automatically change into the most viewed web pages over time. The links on the Home Page are called Top Sites.

3.5 Track Status Box



Illustration 3: Track Status Box

The **Track Status** function checks if the user is correctly positioned in front of the eye tracker.

- The two (2) white dots — The user's eyes and how they are positioned towards the screen. For optimal position, the white dots should be in the middle of the black area.
- The colorful bar on the left with the white arrow — How far or close the user is positioned from the screen.
 - If the white arrow is in the middle of the green area of the bar, the user is at the optimal distance from the screen.
 - If the arrow is in the lower part of the bar — move the user closer to the eye tracker.
 - If the arrow is in the upper part of the bar — move the user further away from the eye tracker.

4 Using TD Browse

TD Browse is purpose-built for eye gaze. It makes browsing the Internet easy while reducing the risks of unwanted clicks.



Most of the functions/actions in TD Browse are also Touch and Mouse enabled, making it easier for a Caregiver or Assistant to help the user if needed. The Caregiver or Assistant can select an Action or change the settings for the user much easier.

4.1 The Actions in the Main Screen

The Interaction Toggle is always available on the right side of the screen. It will turn blue as you switch between Scroll Mode and Click Mode. The icon you see shows what mode you are currently in. The Interaction Toggle toggles between Scroll Mode and Click Mode.

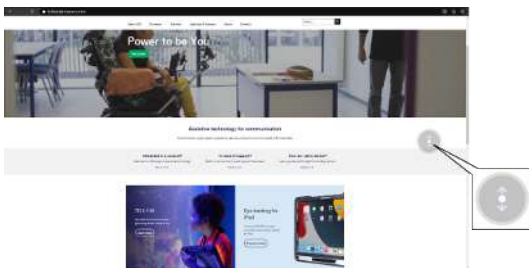


Illustration 4: Scroll Mode is active



Illustration 5: Click Mode is active

4.1.1 Scroll Mode in TD Browse



1. Make sure the Interaction Toggle is in Scroll Mode.
2. Look at the Right, Left, Top or Bottom of the web page to scroll.



4.1.1.1 How do I Scroll a whole Web Page in TD Browse?

1. Make sure the Interaction Toggle is in Scroll Mode.
2. Look at the Right, Left, Top or Bottom of the web page to scroll.



4.1.1.2 How do I Scroll a portion of a Web Page in TD Browse?

For example, the stories section in Facebook.



1. Make sure the Interaction Toggle is in Scroll Mode.
2. Look at the area of the webpage you want to scroll, look for the blue outline around the area.
3. Look at the Right, Left, Top or Bottom of the web page to scroll.

4.1.2 Click Mode in TD Browse




1. Make sure the Interaction Toggle is in Click Mode.



4.1.2.1 How do I Select a link in Click Mode in TD Browse?


1. Make sure the Interaction Toggle is in Click Mode.
2. Fixate your gaze at the desired area of the screen where you want to select a link.

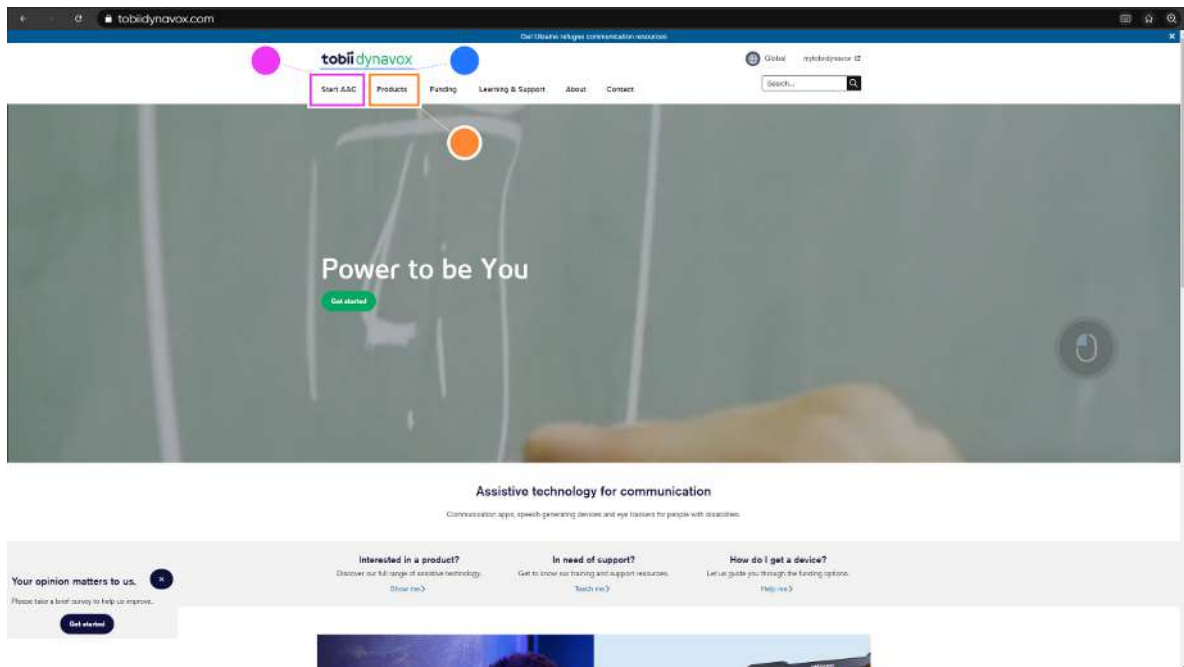


- 1–3 links will be highlighted with a underline or outline with a corresponding dot in different colors.
3. Select the color dot that is associated with the link you want to follow.

4.1.2.2 How do I make a selection in Drop-down menus in TD Browse?

1. Fixate your gaze at the desired area of the screen where you want to select a link.

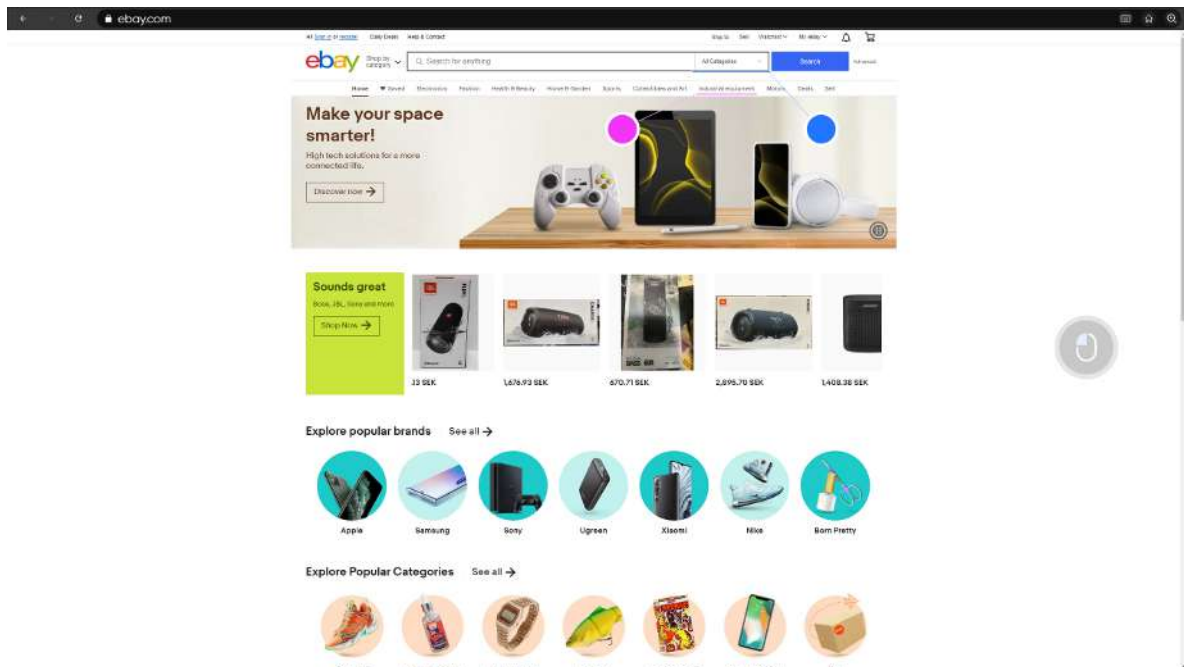


- 1–3 links will be highlighted with a underline or outline with a corresponding a dot in different colors.
2. Select the color dot that is associated with the Drop-down menu you want to open.

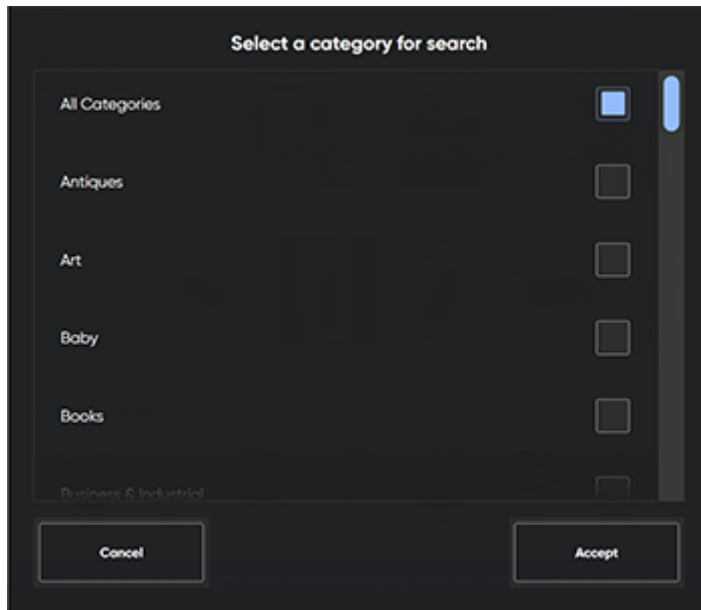


- The Accessible Drop-Down Menu will open.
3. Dwell on the check box associated with your choice(s) to select.
4. Select the **Accept** button to complete you selection or the **Cancel** button to cancel.

4.1.3 Navigation Bar in TD Browse

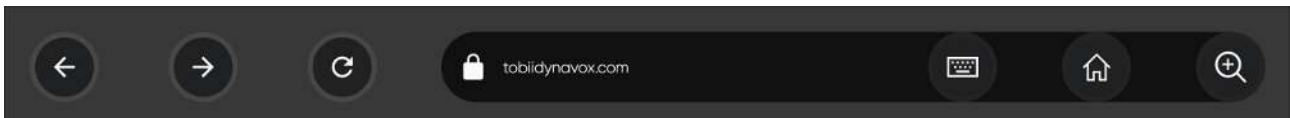



4.1.3.1 How do I Type in the URL in TD Browse?

 For web URLs we use auto completion to speed up the typing.



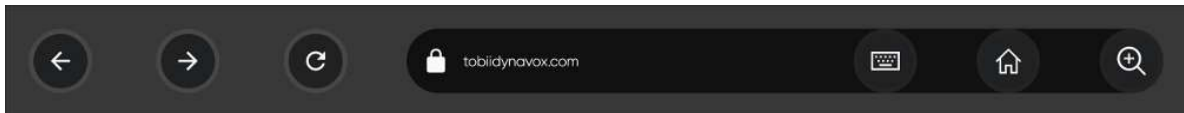

1. Gaze in the upper left or right side of the screen.
The Navigation Bar will expand.






2. Select the  button from the Navigation Bar.
3. Use the keyboard to type in the desired URL.





 For web URLs we specifically do not use Swiftkey for word prediction. TD Browse relies on search suggestion based off geolocation (which ensures that the search suggestions are context relevant) for web URLs.

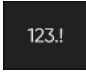4. Select the **Enter** button to go to URL or select the  button to cancel.
5. The new web page will open.

4.1.3.1.1 Keyboard

Depending on the language, the Keyboard consists of three (3) keyboard pages, depending on the language. The user can easily switch between the different keyboard pages by selecting one of these buttons ,  or revert back to the regular keyboard by selecting  buttons located in the top right corner of the Keyboard.

The key on the keyboard where the user is gazing will be indicated by a light frame around the character/function  on the key, and when the key is selected, it will change color briefly to the set activation color .

The keyboard pages are:

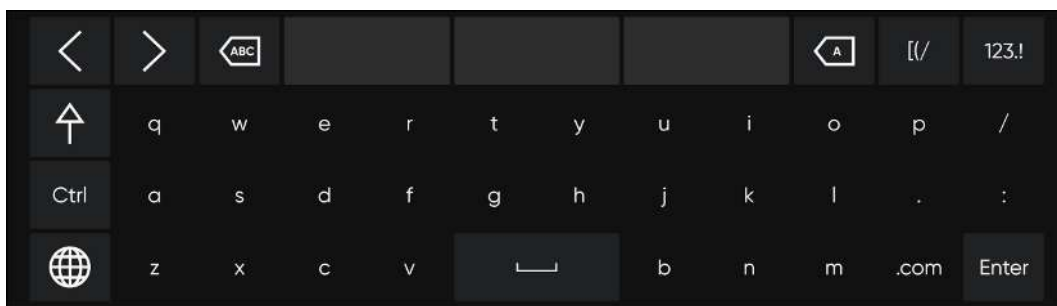


Illustration 6: The Keyboard Page

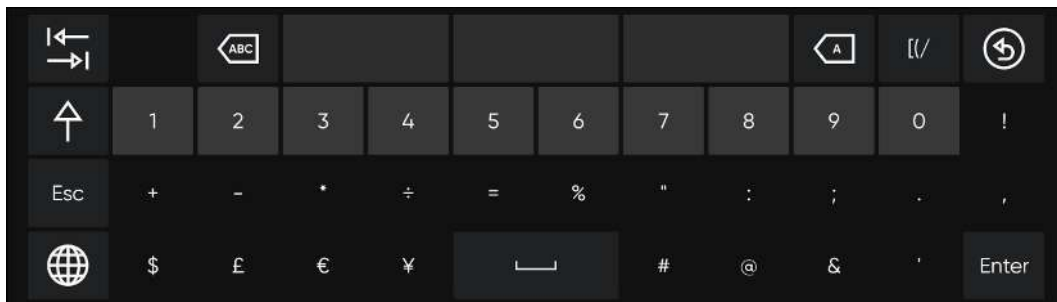


Illustration 7: The Numbers Page

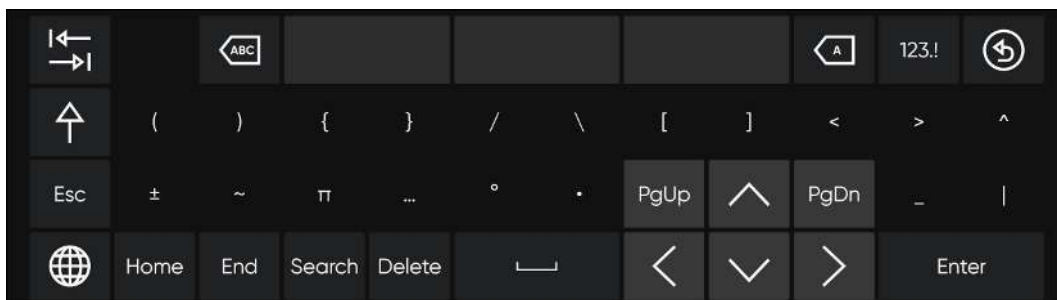


Illustration 8: The Additional Characters Page

To change keyboard language and other settings for the keyboard, see *5 TD Browse Settings, page 22*.

4.1.3.1.1.1 How do I change the Keyboard Language in TD Browse?

1. Make sure the Keyboard is open.



2. Select the
3. Select the preferred language.

To change keyboard language and other settings for the keyboard, see *5 TD Browse Settings, page 22*.

4.1.3.1.1.2 Word Prediction

The word prediction engine used in TD Browse for text fields is from SwiftKey™ and is commonly used in tablets and smartphones. SwiftKey™ uses a blend of artificial intelligence technologies to predict the next word the user intends to type. SwiftKey™ learns from previous texts the user has written and output predictions based on the current input text and what it has learned. The Word Prediction language will change depending on the Keyboard Language setting.

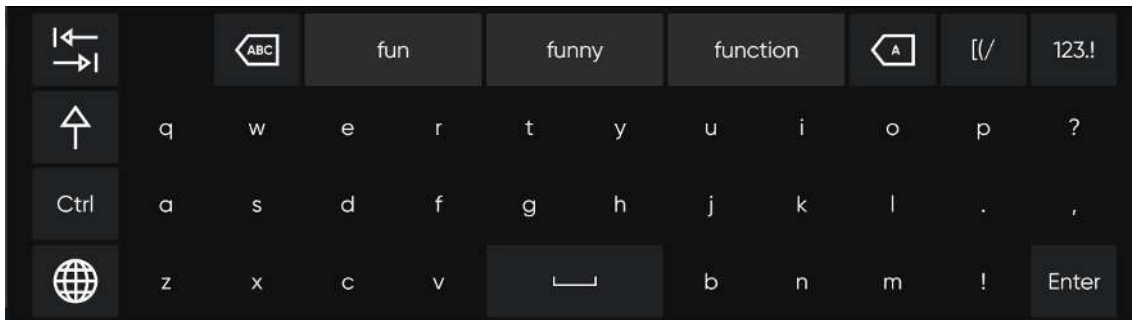


Illustration 9: Keyboard with Swift Key Prediction

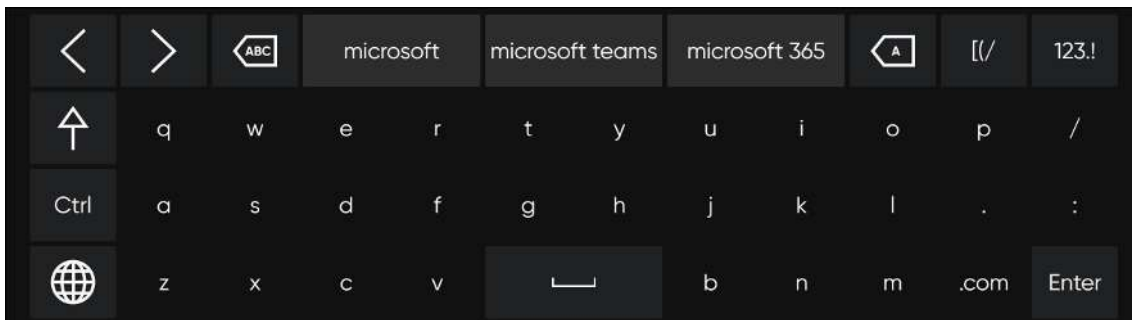


Illustration 10: Keyboard with Geo-search Prediction

When Word Prediction is in use, three (3) boxes will appear above the Keyboard. The prediction buttons are smart and predict geographically-based Search Suggestions when typing a URL but offer word prediction when typing into a text field. The leftmost box will display the most probable predicted word, and the remaining two (2) boxes will contain one predicted word each. To select a predicted word, select the box with the correct word in it. If you can't find the word you are looking for, just keep on typing, and hopefully, the word you are looking for will appear after the next character.





If you are using Word Prediction and want to type a word containing an accent, like café, you will save time and effort by letting Word Prediction handle the accent. Simply type "cafe", and you will get a suggestion for "café".


4.1.3.2 Home Page in TD Browse

4.1.3.2.1 How do I Access the Home Page in TD Browse?

1. Gaze in the upper left or right side of the screen.
The Navigation Bar will expand.

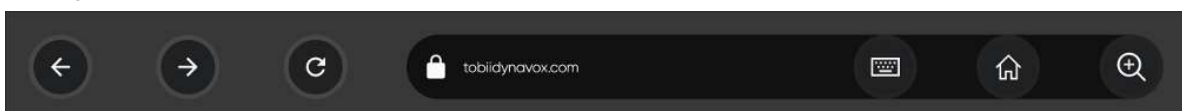


2. Select the  button from the Navigation Bar.
The Home Page will open.

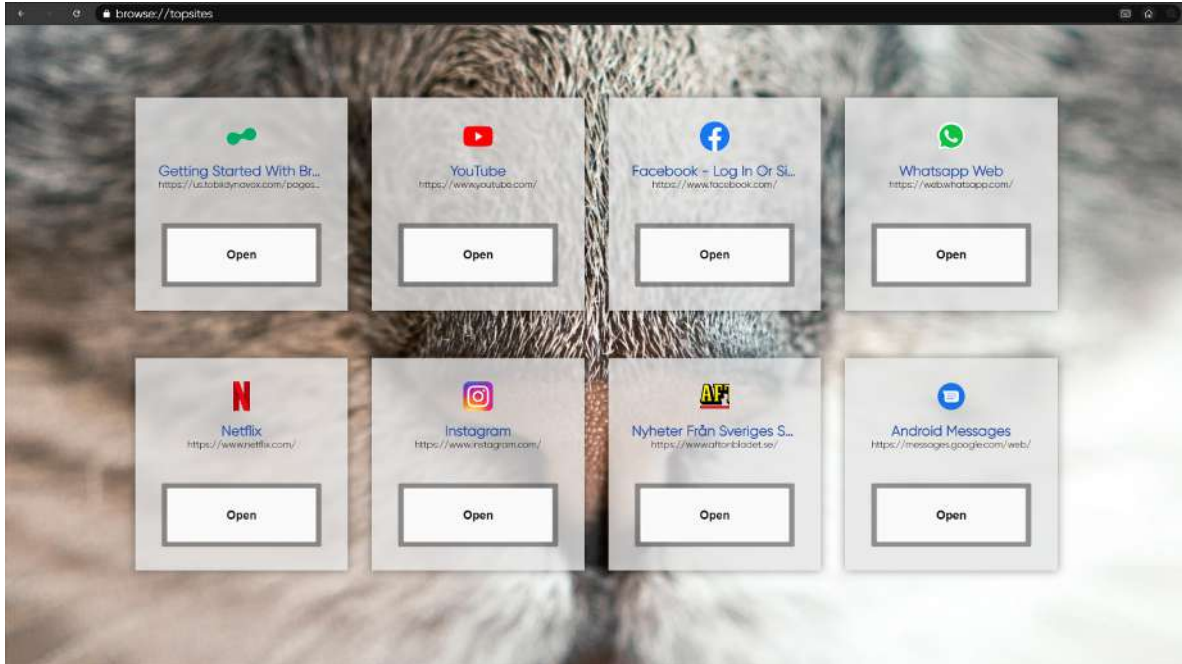


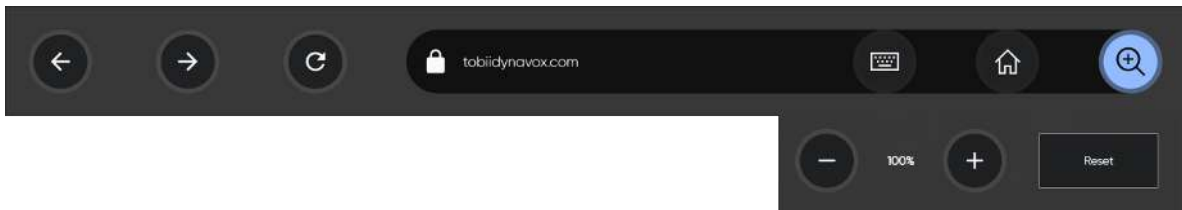
4.1.3.3 Zoom in TD Browse

4.1.3.3.1 How do I Zoom in or out on a Web Page in TD Browse?

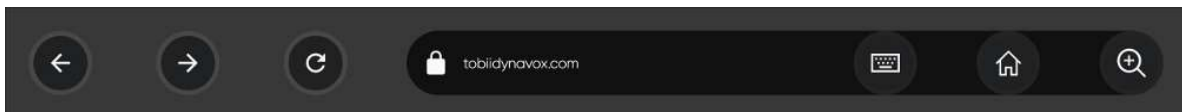

1. Gaze in the upper left or right side of the screen.
The Navigation Bar will expand.





2. Select the  button from the Navigation Bar.
The Zoom menu will open.



3. Select one of the following buttons:

-  — to zoom in.
-  — to zoom out.
- **Reset** — to revert back to 100 % zoom level.

4. Select the  button to close the zoom menu.

 The Zoom setting will be remembered for that webpage until you change it..

4.2 Actions in the Off-Screen Menu




4.2.1 How do I Activate Switcher in TD Browse?

With Switcher, the user can switch between active software and start Tobii Dynavox Applications.

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.


2. Select the  (Switcher) action.
Switcher will open.

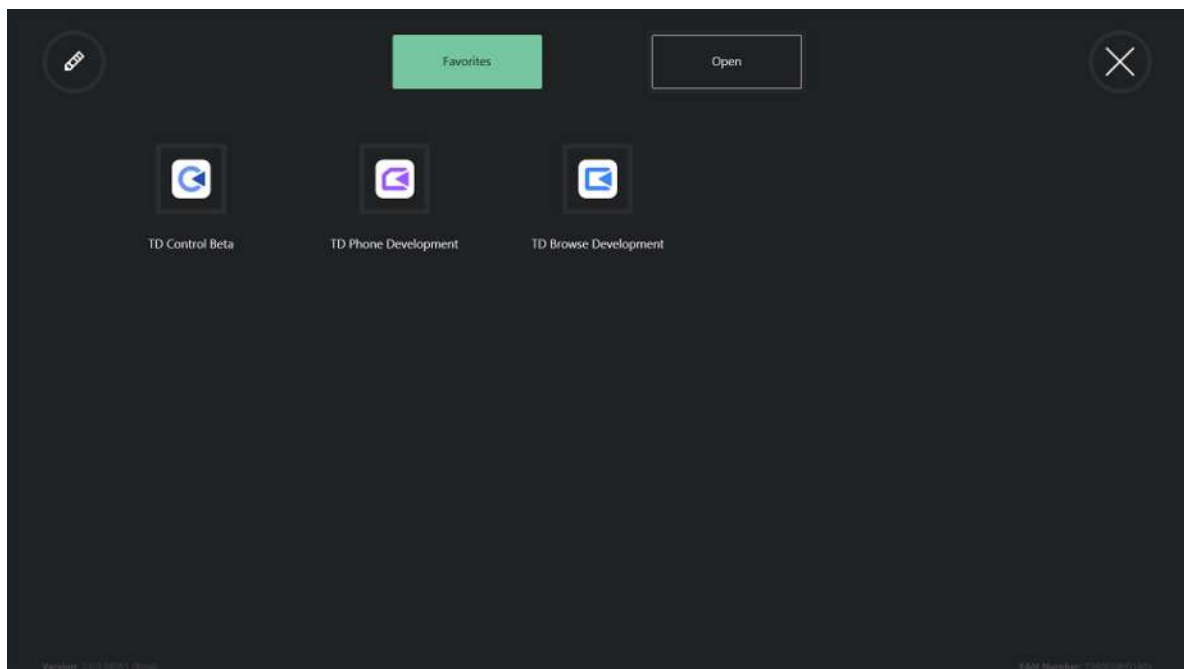


For more information about Switcher, see *6 TD Switcher, page 31*.

4.2.2 How do I Activate the Dashboard in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) icon.
The Dashboard will open over your current webpage.

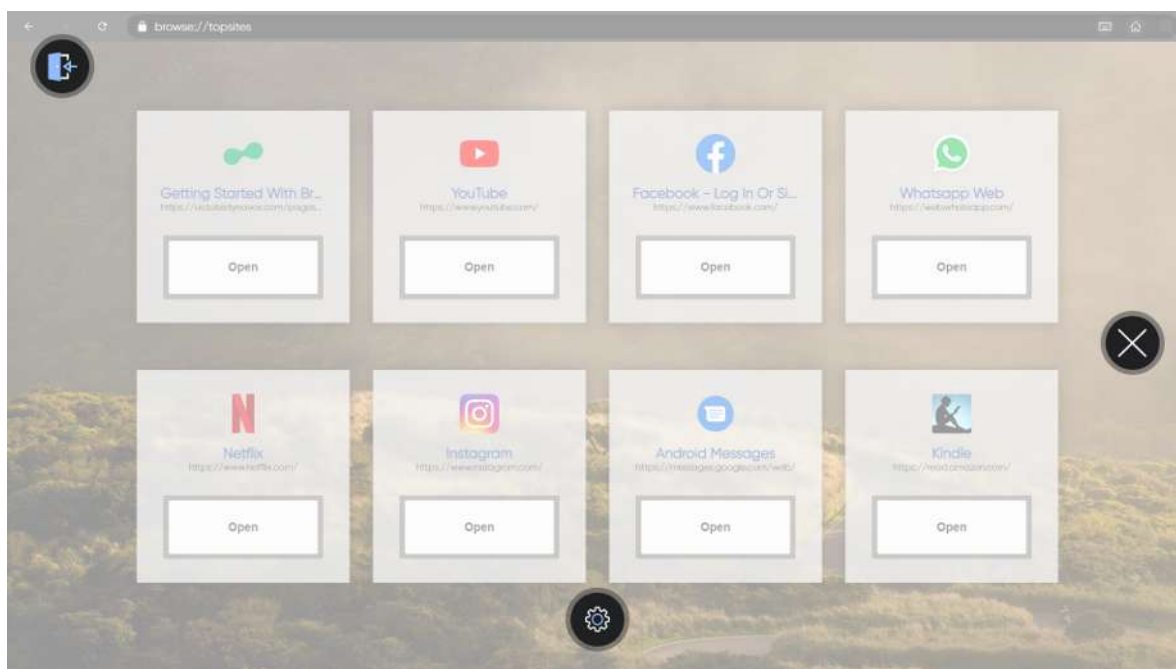

For more information, see 4.3 *The Actions on the Dashboard*, page 20

4.2.3 Pause in TD Browse


Using eye gaze is a repeated physical task, and the muscles in use around the eyes can become fatigued. Pausing gaze not only gives you time to look at the screen without interacting but also provides your eyes with time to rest.


4.2.3.1 How do I Pause the Eye Tracking in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) icon.

3. Select the  (Pause) action.

An  blue pause icon will show briefly on the screen, and there will be an blue border on the outer edges of the screen indicating that TD Browse is paused.

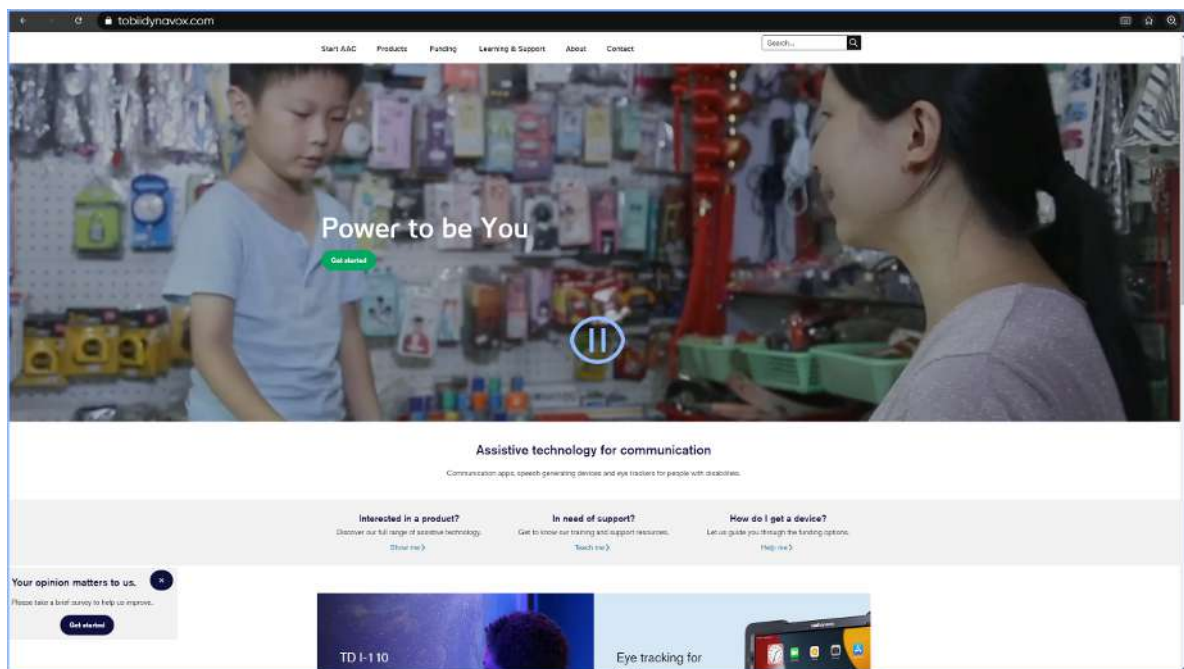

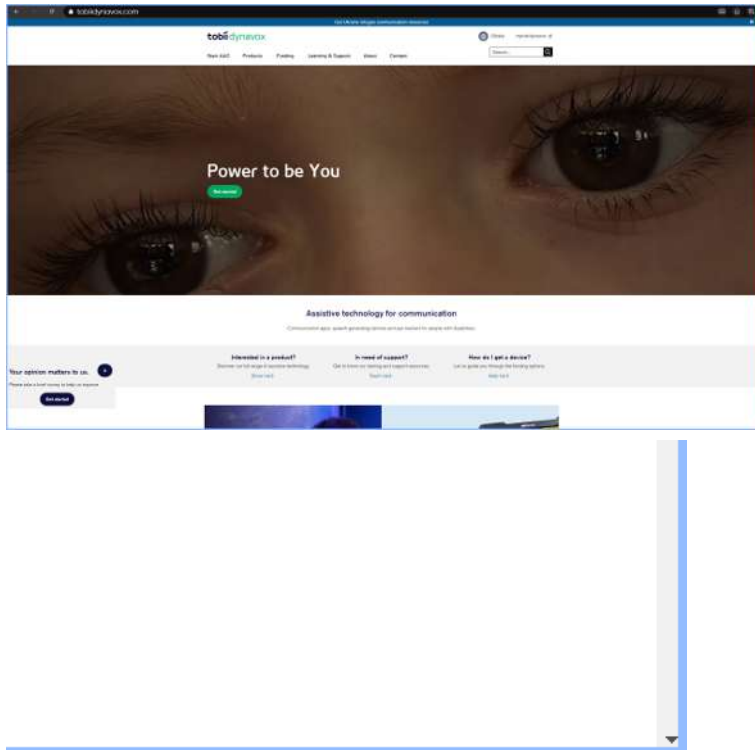

4.2.3.2 How can I See if the Eye Tracking is Paused in TD Browse?
 There will be a blue border on the outer edges of the screen.



4.2.3.3 How do I Resume the Eye Tracking in TD Browse

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.

The  (Resume Gaze) icon will become visible in the lower part of the screen.


2. Select the  (Resume Gaze) icon to resume eye tracking in TD Browse.


4.3 The Actions on the Dashboard

4.3.1 How do I Activate the Dashboard in TD Browse?

The Dashboard is hidden so it will not interfere with and/or distract the user. To call up the Dashboard, follow these steps.


1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) icon.
The Dashboard will open over your current webpage.

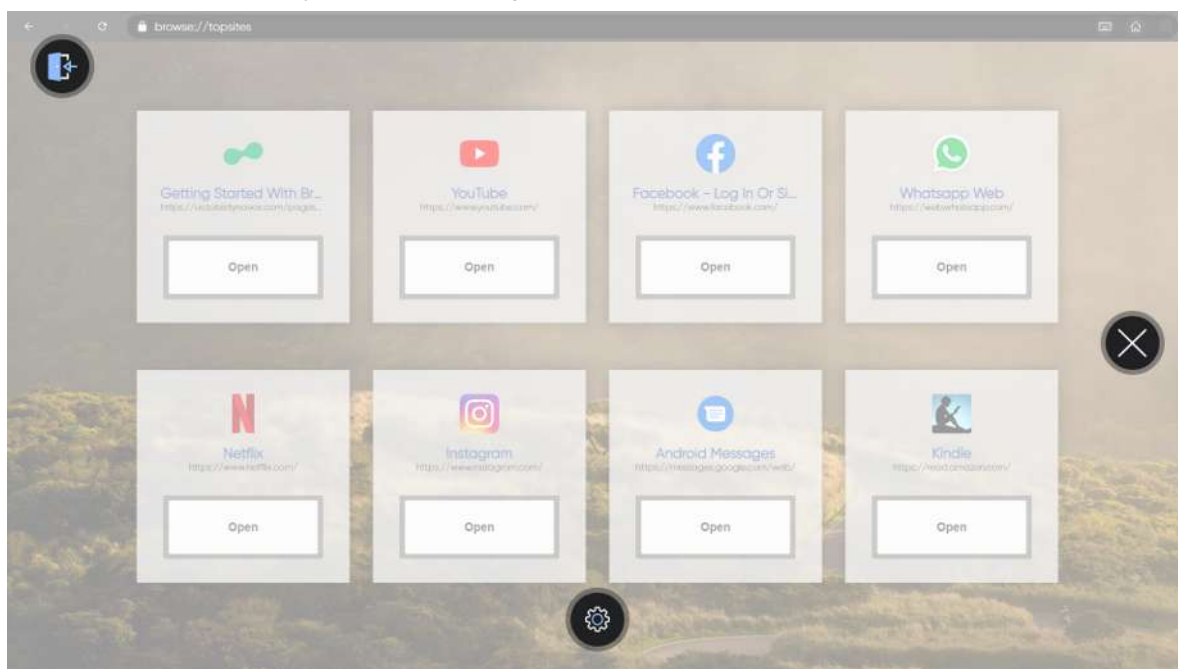

4.3.2 How do I Activate an Action from the Dashboard in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.

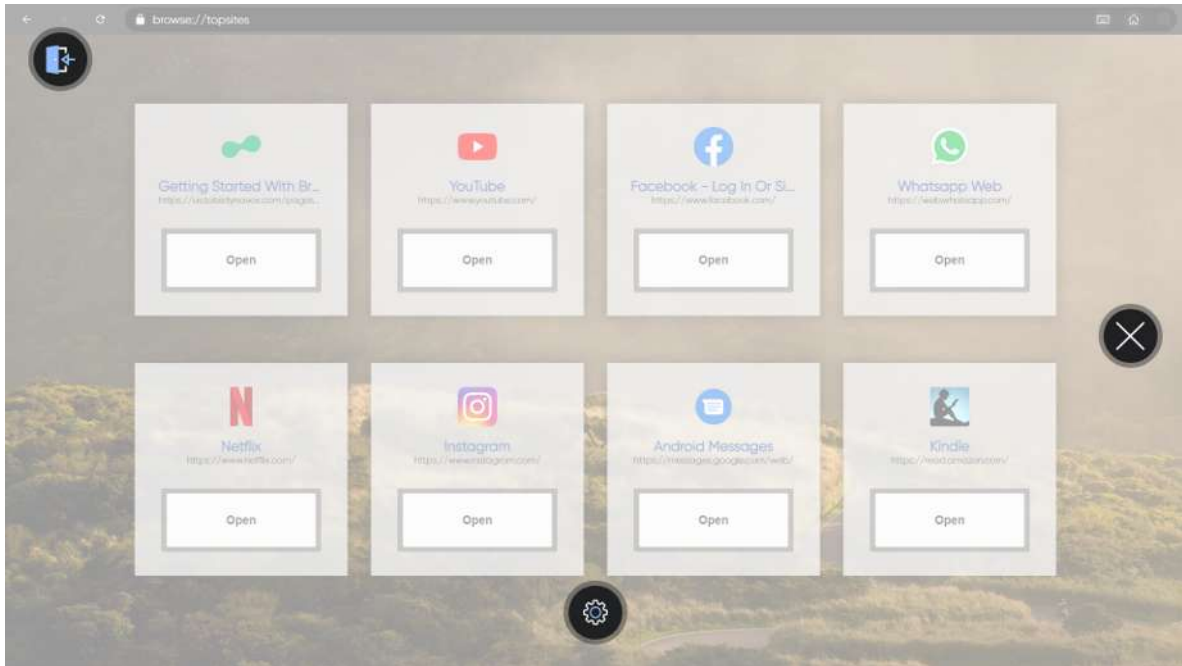

2. Select the  (Dashboard) icon to open the Dashboard.




3. Select the action to be executed from the Dashboard or select the close button to cancel.

4.3.3 How do I Access the Settings in TD Browse?




1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.
2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to access the settings for TD Browse.
For more information about available settings, see *5 TD Browse Settings, page 22*

5 TD Browse Settings

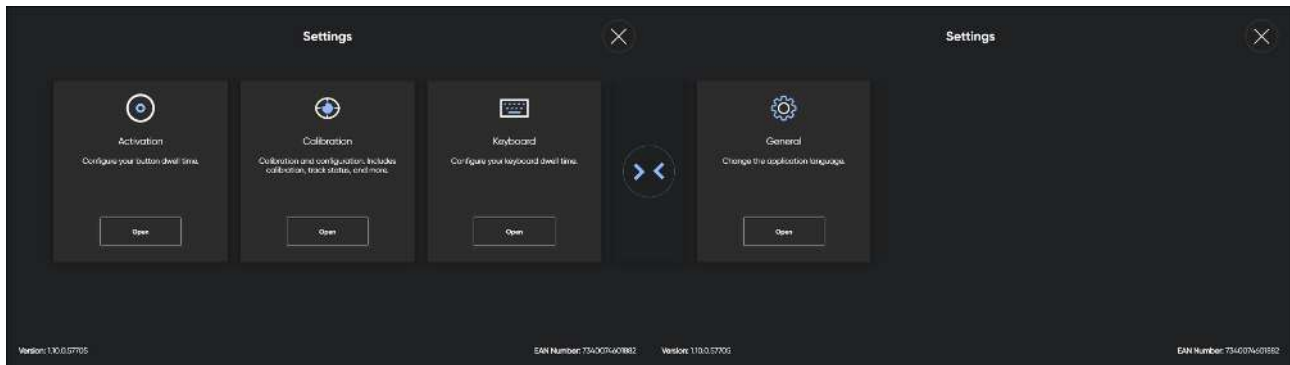

Icons	Action	Description
	Activation	Opens the Activation & Feedback settings. For more information, see <i>5.1 Activation , page 23</i> .
	Calibration	Opens the Calibration settings. For more information, see <i>5.2 Calibration , page 24</i> .
	Keyboard	Opens the Keyboard settings. For more information, see <i>5.3 Keyboard , page 28</i> .
	General	Opens the General settings. For more information, see <i>.5.4 General , page 30</i> .

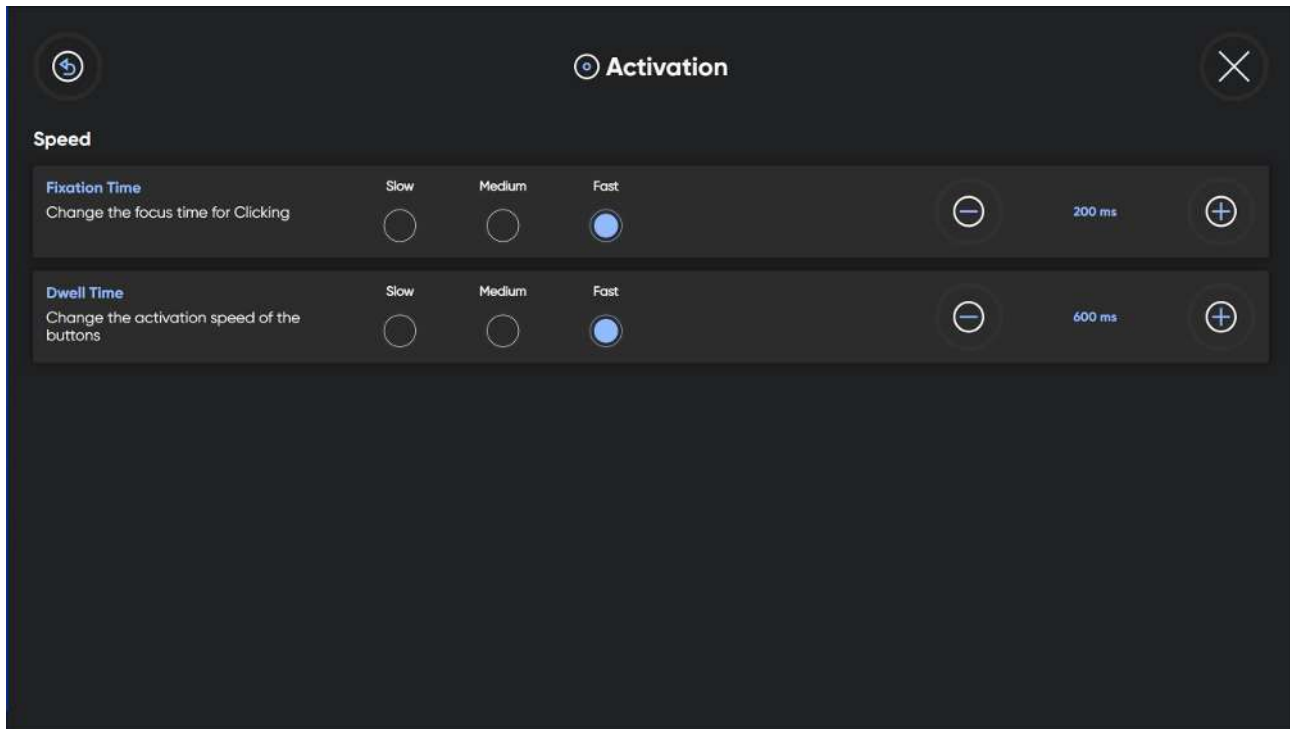


5.1 Activation







5.1.1 Speed

5.1.1.1 How do I Select the Fixation Time for the Color Links in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to open the TD Browse Settings.
4. Look for the  (Activation) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Speed** section.
7. Select the radio button for the **Fixation Time** for one of the following values:
 - **Slow** (default)
 - Medium
 - Fast

8. Fine-tune the time with the  and  buttons, if needed.
9. Select the  (Back) button to go back to TD Browse Settings.
10. Select the  (Close) button to exit TD Browse Settings.

5.1.1.2 How do I Change/Set the Dwell Time for the Buttons in TD Browse?

The dwell time is the time the user have to fixate the gaze upon the action items in the Interaction and Off-screen menus and to activate the selected action.



This setting is just for the dwell time for the buttons. There is also a dwell time setting for the keyboard. For more information, see *5.3.1.1 How do I Change/Set the Dwell Time for the Keyboard in TD Browse?, page 28*.

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.



2. Select the (Dashboard) icon to open the Dashboard.



3. Select the (Settings) action to open the TD Browse Settings.



4. Look for the (Activation) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Speed** section.
7. Select the radio button for the **Dwell Time** for one of the following values:
 - **Slow** (default)
 - Medium
 - Fast



8. Fine-tune the time with the and buttons, if needed.




9. Select the (Back) button to go back to TD Browse Settings.



10. Select the (Close) button to exit TD Browse Settings.

5.2 Calibration

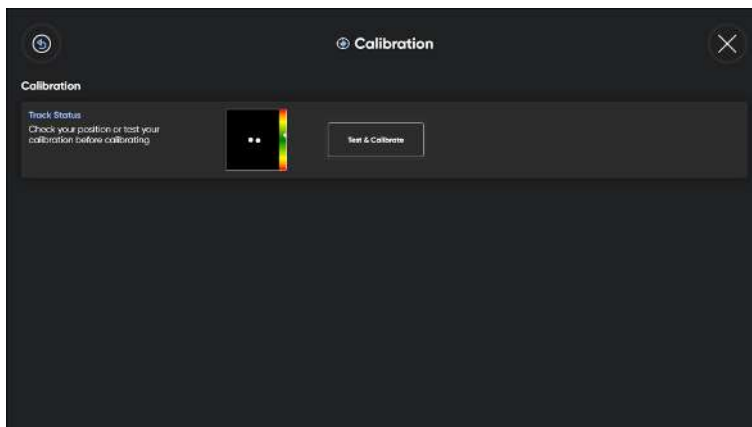


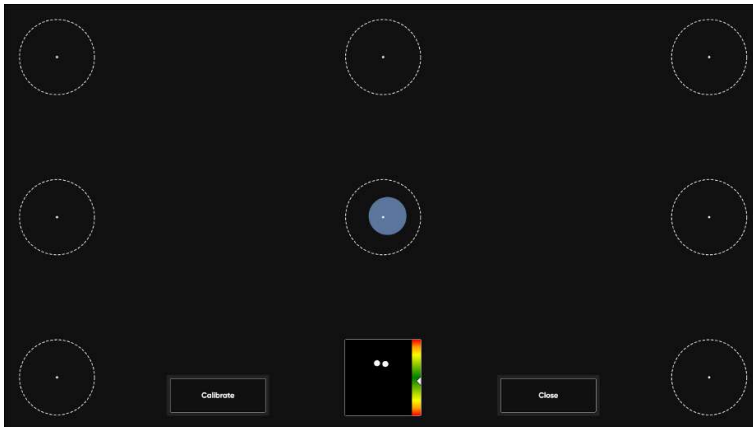
5.2.1 How does the Calibration Process Work in TD Browse?

The calibration process works as follows:


The Test Calibration will be shown when a calibration starts, whether it is a Quick Calibration or a regular Calibration. This page offers you the chance to assess the quality of your calibration.

On the test calibration screen, there are circles with a small dot in the center. The blue dot shows where the user is looking. Fixating the gaze on one of the dots, it shows where the eye tracker registers the fixation in that dot and how small or large offset there is.


 If the blue circle is within the dotted white circle, then the calibration is good enough for using TD Browse.



When the Calibrate button is selected, the first calibration screen will appear. It is a black screen, and in the center of the screen, there is one single blue dot with a black center.

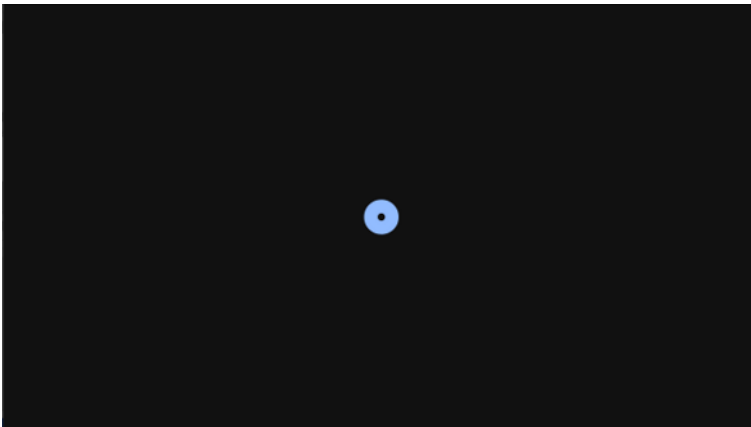

Nothing will happen until the user fixates the gaze on the dot, the eye tracker will register the fixation, and the dot will blink once and disappear.

 As long as the dot remains on the screen, the eye tracker has not registered the gaze data for that position

A new screen will appear. It is a black screen, and in the four corners, an blue dot with the black center will be present.



Nothing will happen until the user fixates the gaze on one of the dots, the eye tracker will register the fixation on that dot, and the dot will blink once and disappear.



As long as the dot remains on the screen, the eye tracker has not registered the gaze data for that position

When the user's have fixated the gaze upon all of the dots on the screen, and all of the dots have disappeared, the calibration process finishes and the test calibration screen will show up again.

Try out the calibration by looking at the circles on the screen. Close the test calibration screen or start a new calibration if needed.

5.2.2 How do I Check the Positioning of the User using Track Status in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.



2. Select the (Dashboard) icon to open the Dashboard.



3. Select the (Settings) action to open the TD Browse Settings.



4. Look for the (Calibration) icon to find the correct settings card.

5. Select the **Open** button.

6. Check the positioning of the user by looking in the **Track Status** box.

For more information, see *3.5 Track Status Box, page 9*



7. Select the (Back) button to go back to TD Browse Settings.



8. Select the (Close) button to exit TD Browse Settings.

5.2.3 How do I Calibrate a User in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.




2. Select the (Dashboard) icon to open the Dashboard.



3. Select the (Settings) action to open the TD Browse Settings.



4. Look for the (Calibration) icon to find the correct settings card.
5. Select the **Open** button.
6. Check the positioning of the user by looking in the **Track Status** box.
For more information, see *3.5 Track Status Box, page 9*
7. Select the **Test & Calibrate** button.
The Check calibration page will open.

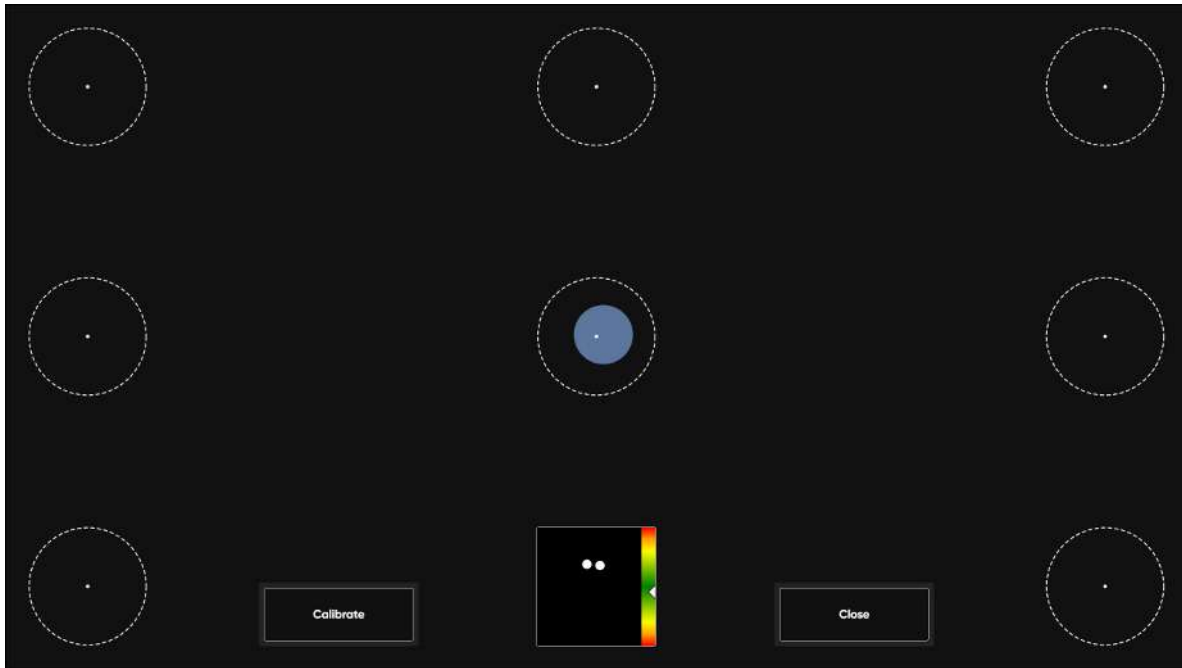

8. Select the **Calibrate** button.
9. Look at the orange dots, one by one that appears on the screen until they disappear.
10. Try out the new calibration.
11. Repeat steps 7–9 until you are satisfied.
12. Select the **Close** button.

5.2.4 How do I Change the Display Setup for my Eye Tracker in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.



2. Select the (Dashboard) icon to open the Dashboard.



3. Select the (Settings) action to open the TD Browse Settings.



4. Look for the (Calibration) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Display Setup** button.




This button is not Gaze enabled and must be selected by touch or by Mouse.


7. Follow the on-screen instructions.
8. Select the **Done** button



9. Select the (Back) button to go back to TD Browse Settings.

10. Select the  (Close) button to exit TD Browse Settings.

5.3 Keyboard

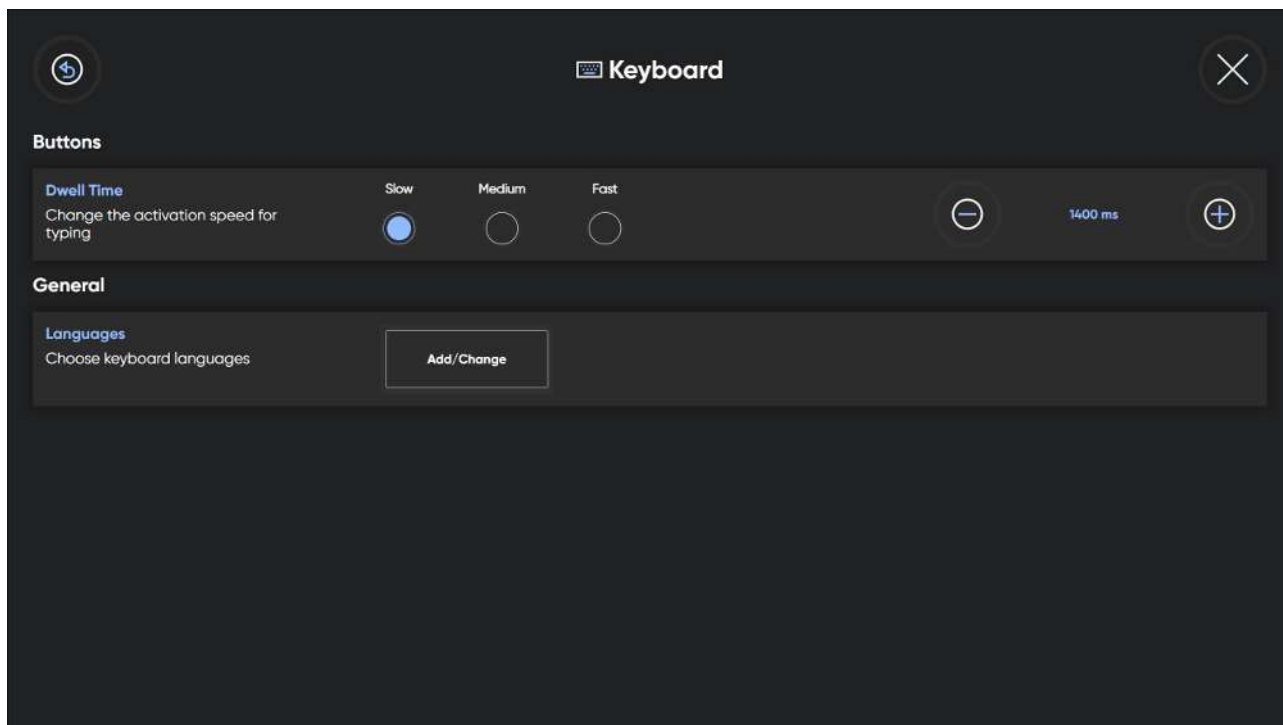


Language



The selected language keyboard layout is shown. Select the **Add/Change** button to change the keyboard layout to another language. There are several different language keyboard layouts available.


5.3.1 Buttons

5.3.1.1 How do I Change/Set the Dwell Time for the Keyboard in TD Browse?





 This setting is just for the dwell time for the keyboard. There is also a dwell time setting for the buttons. For more information, see 5.1.1.2 *How do I Change/Set the Dwell Time for the Buttons in TD Browse?*, page 24.

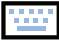1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to open the TD Browse Settings.




4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Look in the **Buttons** section.
7. Select the radio button for one of the following **Dwell Times**:

- **Slow** (default)
- Medium
- Fast

8. Fine-tune the time with the  and  buttons, if needed.
9. Select the  (Back) button to go back to TD Browse Settings.
10. Select the  (Close) button to exit TD Browse Settings.



5.3.2 General

5.3.2.1 How do I Select my Keyboard Language in TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.
2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to open the TD Browse Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the radio button for the installed language you want to have available for the keyboard.






If the wanted language is missing, see 5.3.2.2 *How do I Download a new Keyboard Language in TD Browse?*, page 29.




7. Select the  (Back) button to go back to TD Browse Settings.
8. Select the  (Close) button to exit TD Browse Settings.

5.3.2.2 How do I Download a new Keyboard Language in TD Browse?




The device must be connected to Internet to be able to download a new language.





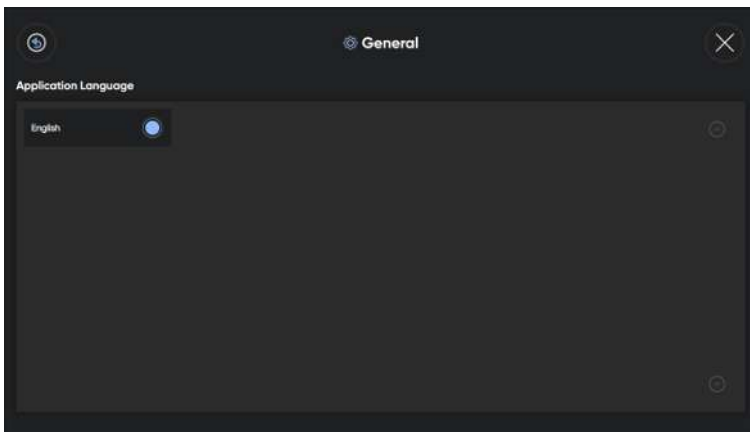

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.
2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to open the TD Browse Settings.
4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Download** tab.
7. Use the “Scroll Up” and “Scroll Down” buttons on the right to scroll through the list of keyboard languages available.

8. Select the  (Download) button to the right of the new language you'd like to download.
9. On the Download a Keyboard Languages dialog, select the **Download** button to start the download or the **Cancel** button to cancel the download.
10. Select the  (Back) button to go back to TD Browse Settings.
11. Select the  (Close) button to exit TD Browse Settings.

5.4 General



5.4.1 How do I Select the Application Language for TD Browse?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker. The Off-Screen Menu will become visible in the lower middle part of the screen.
2. Select the  (Dashboard) icon to open the Dashboard.
3. Select the  (Settings) action to open the TD Browse Settings.
4. Select the  (Next Screen) button on the right-hand side.
5. Look for the  (General) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Application Language** section.
The active language will be shown after the Languages heading.
8. Select the radio button next to the preferred language.
9. Select the  (Back) button to go back to TD Browse Settings.
10. Select the  (Close) button to exit TD Browse Settings.

6 TD Switcher

TD Switcher is a help app, allowing easy switching between installed Tobii Dynavox Apps, apps in general and open windows.



The icons and graphics shown in this section are for TD Control. The only thing that differs is the color of the icons.

6.1 How do I Access TD Switcher?

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.



2. Select the (TD Switcher) button.
TD Switcher will open.

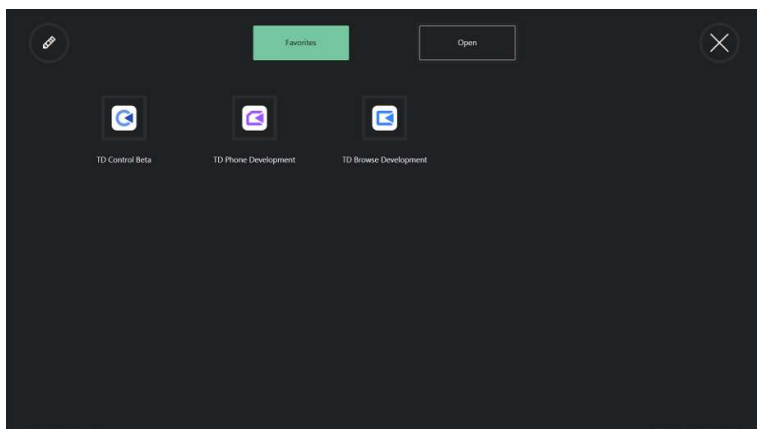


Illustration 11: TD Switcher

6.2 How do I Switch to an Application in TD Switcher




If you want to add a non-Tobii Dynavox application, please make sure to start the wanted application before proceeding.


1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.



2. Select the (TD Switcher) button.
3. Select the application to change to:

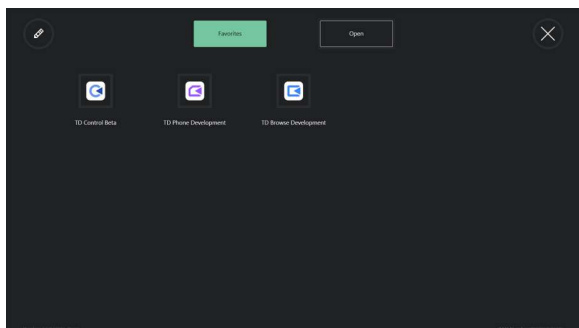



6.3 How do I Add an Application to TD Switcher?

 If you want to add a non-Tobii Dynavox application, please make sure to start the wanted application before proceeding.

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (TD Switcher) button.


3. Select the  (Edit) button in the top left corner.

4. Select the  (Add) button for the location where to add the application.

5. Select the correct tab:

- Tobii Dynavox
- Open


6. Select the application to add.


7. Select the  (Done) button in the top left corner.

6.4 How do I Remove an Application from TD Switcher?


1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.
The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (TD Switcher) button.

3. Select the  (Edit) button in the top left corner.

4. Select the  (Remove) button for the location where to remove the application.

5. Select the **Remove** button to confirm.

6. Select the  (Done) button in the top left corner.

Copyright © Tobii Dynavox AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owners.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact