


Communicator 5 Accessible Apps - Accessible MSN: Quick Guide

This document provides instructions on how to use Accessible MSN within the Communicator 5 Accessible Apps collection to stay connected to international news in the user's preferred language.

Requirements: An unlocked device with access to the Internet is required to use Accessible MSN.

Note: Refer to the **Getting Started with Communicator 5 Accessible Apps Quick Guide** for instructions on how to set up Communicator 5 Home Page with the accessible apps collection.


Launch Accessible MSN

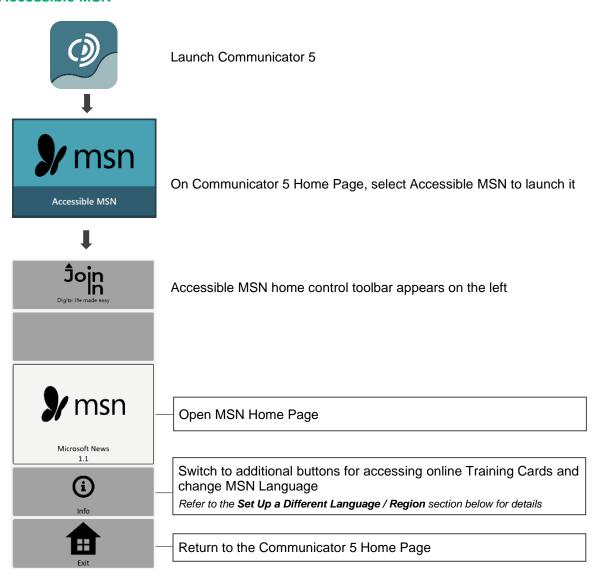


Set Up a Different Language / Region

Accessible MSN selects language and region automatically based on the user's device settings. To change to a different language and a different region, follow the steps depending on where the user is within the software.

Right After Launching Accessible MSN

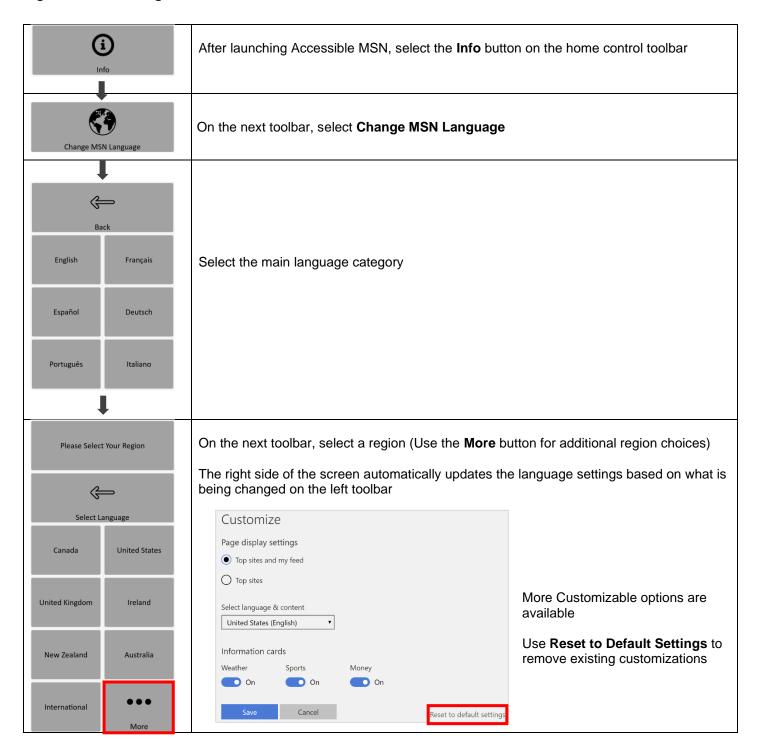


After selecting language and region, Accessible MSN automatically switches to the main control toolbar on the left and the MSN Home Page (in the chosen language) on the right.

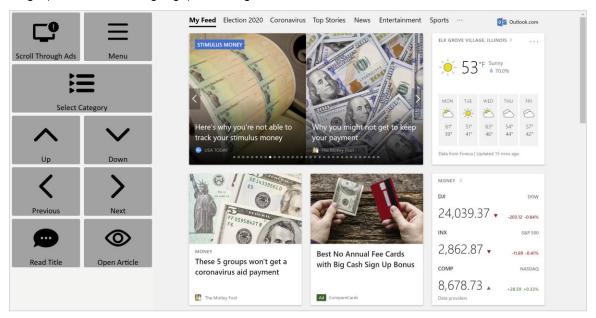

When Navigating and Browsing News

Depending on which page the user is on, use the **Back** button and/or **Exit Article** button to return to the main control toolbar.


- 1. Select Menu
- 2. Select Change MSN Language
- 3. Use the same steps as above to select the main language category and then the region

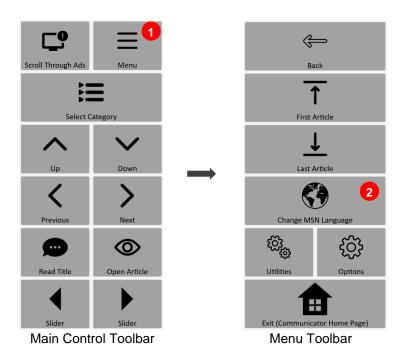


Navigate the MSN Site and View News

After launching Accessible MSN and/or setting up the preferred language and region, the control toolbar allows easy navigation through the MSN site to view news with additional accessible options.


Main Control Toolbar

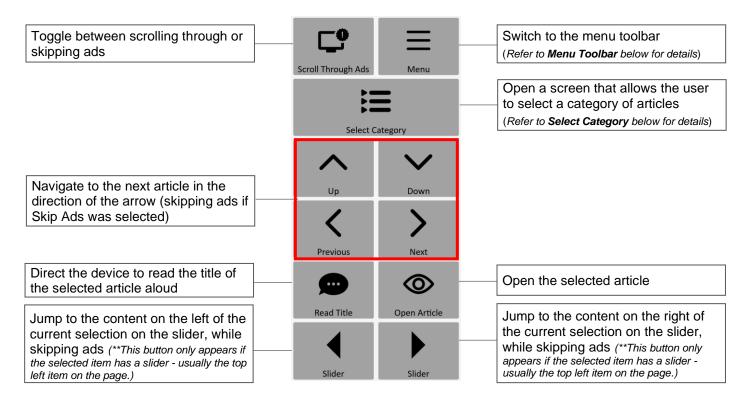

Menu Toolbar

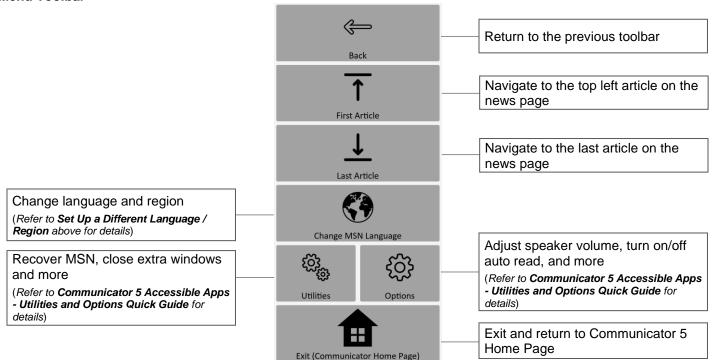


Select Category

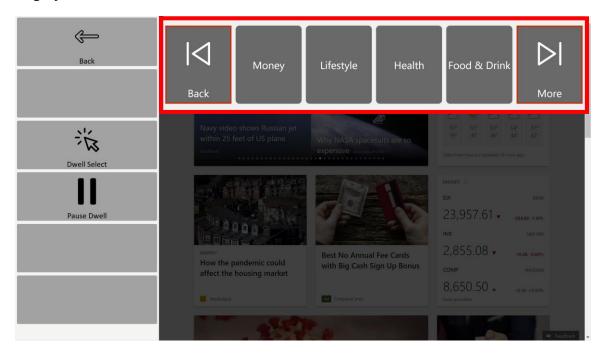


- Use the More or Back button to navigate through additional categories. The More and Back buttons are always highlighted (dwell enabled)
- **Dwell Select** Enable selection using eye gaze. This will highlight the buttons with the category names, and will make them dwell enabled
- Pause Dwell Disable selection using eye gaze

Reading Control


• After selecting a news article, use Open Article to view the page

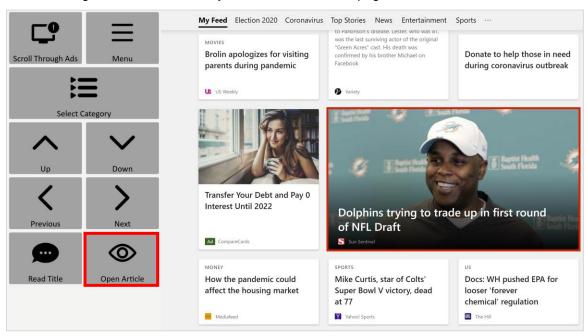

If the article has plain text, select the Reading Enabled button, and toggle between it and the Regular Scrolling
mode

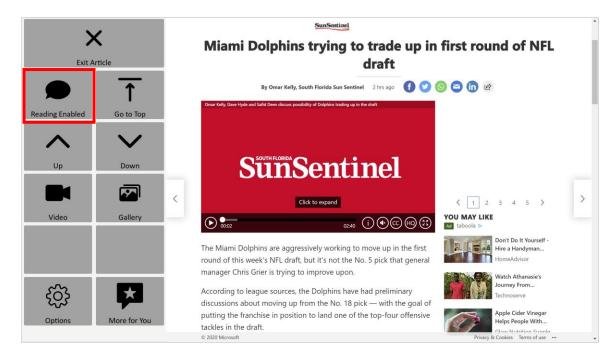

- Reading Enabled Mode
 - In Reading Enabled mode, the paragraph(s) in focus will be highlighted when scrolling up or down.
 The length of the reading focus area can be changed in the Options toolbar.
 - Select Regular Scrolling to exit the Reading Enabled mode

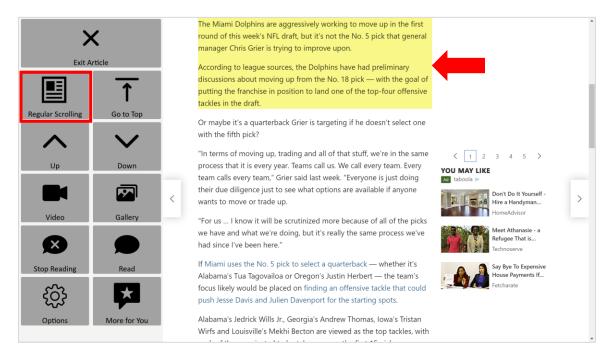

Reading Toolbar

