Quick Guide — Windows Access Icons v.1.0 - en-US

tobiidynavox

Quick Guide — Windows Access Icons

Communicator 5

Mouse emulation is the act of moving the mouse cursor around the screen following a user's gaze as if the user was moving a mouse or other pointing device. Windows Access in Communicator 5 mimics mouse emulation and allows for interaction with sensory games such as Eye FX and Look to Learn. This document provides a reference for all icons associated to Windows Access in Communicator 5.


Locate the Windows Access Page Sets

In Tobii Dynavox Communicator 5, Windows Access page sets are in the **Text Communication > Windows Access** category.



Placement of Windows Access Toolbars

Three options are available for where to place the Windows Access toolbars:

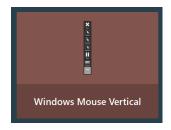

Windows Mouse Vertical – The toolbar shows on the left (default) or the right side of the screen.



Windows Mouse Floating – The toolbar shows on one of the 7 areas specified by the user: center, top left, top center (default), top right, bottom left, bottom center, and bottom right.



Windows Mouse Horizontal – The toolbar shows on the bottom (default) or the top of the screen.







Quick Guide — Windows Access Icons v.1.0 - en-US




Windows Access Icons

×	Closes and exits Windows Access toolbar and returns to Communicator 5 Home Page.		┋ →	Moves the vertical toolbar from the left side of the screen to the right side.
Exit			Move Bar	
*	Left clicks the selection on the screen.		←!	Moves the vertical toolbar from the right side of the screen to the left.
Left Click			Move Bar	
*	Double clicks the selection on the screen.		†	Moves the horizontal toolbar from the bottom of the screen to the top.
Dou- ble Click			Move Bar	
*	Right clicks the selection on the screen.		1	Moves the horizontal toolbar from the top of the screen to the bottom.
Right Click			Move Bar	
	Pauses the current gaze activity; switches to the Resume button.		♦	Moves the floating toolbar to a desired screen location among one of the 7 choices
Pause		Re- sume	Move	For more information, see Floating Toolbar Positions
==	Displays the keyboard on the screen.		•••	Indicates currently on the page 1 of the toolbar and switches to the page 2 of the toolbar.
Key- board			1/2	
Ø.	Opens the Windows Access Settings screen (*Refer to the Communicator 5 Windows Access Settings document for details).		• • •	Indicates currently on the page 2 of the toolbar and switches to the page 1 of the toolbar.
Set- tings	_		2/2	
0	Displays the icons of the programs configured in the Program Shortcut Settings (e.g., the Google Chrome icon) for easy access to frequently used programs.		•	Returns to the previous toolbar screen.
Pro- grams			Back	
	Adjusts active application window to the edge of the Windows Access toolbar for a better view of both windows.		✓	Confirms the successful launch of a selected program in the Programs list and returns to the main toolbar.
Dock Win- dow	non of boar windows.		ОК	
^	Scrolls up the active window if applicable.		~	Scrolls down the active window if applicable.
Scroll Up			Scroll Down	

tobiidynavox

Floating Toolbar Positions

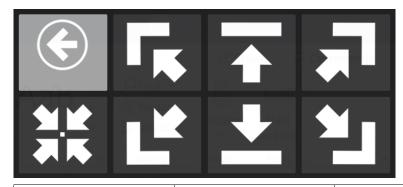

©	r,	→	₹ 7
Back	Top Left	Top Center	Top Right
M M M M M M M M M M M M M M M M M M M	L¢	<u>+</u>	1
Center	Bottom Left	Bottom Center	Bottom Right