

Release Notes TD Control

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 1.80.0

Release date: 2023-10-24

Improvements

- Fixed an issue where the incorrect keyboard language was selected on first startup.
- Updated German translations.
- TD Control now aligns font size option with other TD Suite apps.
- Fixed an issue where keyboard notepad buttons were active while hidden.
- Bug fixes and improvements.

Release Version 1.79.1

Release date: 2023-05-02

Improvements

- Fixed a problem where Hide Trace state was not persisted when opening Repeat Click mode.

Release Version 1.79.0

Release date: 2023-04-25

Improvements

- Fixed a problem where Chinese characters were not visualized correctly.
- Fixed a problem where locked off-screen menu in Continuous Mode could be unlocked after using mouse or touch input.
- Fixed a problem where Trace could be shown in Quick Start Guide in Overlays Mode.
- Fixed a problem where Trace could be shown in screenshot in Adjust Target view.
- Improved stability of the keyboard layouts update system.
- Updated error messages when accessing myTobiiDynavox account.

Release Version 1.78.0

Release date: 2023-03-09

Improvements

- Fixed a problem where settings could become corrupted from missing keyboard layout files.
- Fixed a problem where Control sometimes did not properly hide when other applications were opened.

Release Version 1.77.0

Release date: 2022-02-28

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Updated user interface of application language and keyboard layout settings.
- Split Chinese keyboard layout into separate Simplified Chinese (Stroke) and Chinese Sogou keyboard layouts.
- Updated text in General tab in Settings.
- Updated text in Missing hardware popup window in Gamepad mode.
- Fixed a possible crash of the application when downloading new keyboard layouts in Settings.
- Fixed a problem where it was possible to lose possibility to resume paused eye tracking within Gamepad mode.
- Updated navigation between Change Language menu in keyboard and Keyboard Languages view in Settings.
- Reduced RAM usage by the application.
- Fixed a problem where certain disabled buttons in Settings were still interactable using eye gaze.

Release Version 1.76.0

Release date: 2023-02-14

Improvements

- Added a possibility to execute mouse left click actions within Drawing mode.
- Updated text font size that is shown on partner window on I-Series devices.
- Updated Terms or Service text to have an easier way of accessing the web link to Privacy Policy.

Release Version 1.75.0

Release date: 2023-01-31

Improvements

- Fixed a problem where sometimes it was not possible to close the notification popup in Gamepad mode using eye tracking.
- Updated the text in the notification popup in Gamepad mode.
- Fixed a problem where actions in Continuous Mode remained active when interacting with the off-screen menu of the application.

Release Version 1.74.0

Release date: 2023-01-17

New Features

- Added a possibility to navigate between different pages of an Overlay.

Improvements

- Updated the troubleshooting logs that are collected by the application.
- Updated user interface text in Chinese language.
- Fixed a problem where sometimes the image of the tracker in Display Setup view was not shown correctly.
- Fixed a problem where the application would not properly shut down.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.73.0

Release date: 2023-01-05

Improvements

- Updated user interface of additional keyboard languages panel.
- Added Windows key to keyboard layout with function keys.
- Updated user interface text in Dutch language.

Release Version 1.72.0

Release date: 2022-12-06

New Features

- Added an Overlays mode that allows to interact with custom-made eye trackable regions.
- Added a Gamepad mode that helps to control XBOX device using external accessories.

Improvements

- Updated user interface of scrollable views in Release Notes and Terms of Service pages.
- Updated background of notification popup windows.
- Updated the text of Version and EAN number in the Settings view.
- Updated Orientation guide user interface to align with the one in Configuration guide.
- Fixed a problem where the application in certain situations could crash when using physical keyboard.

Release Version 1.71.0

Release date: 2022-11-22

Improvements

- Added a possibility to pause eye tracking in Drawing mode.
- Added a possibility to get more information about what eye tracker bracket is when doing display setup.
- Fixed a problem where it was possible not to have the Trace visible in Continuous mode.
- Fixed a problem where it was possible to get unintentional left click activations after exiting Repeat Click mode.
- Fixed a problem where the application could crash if the received information about myTobiiDynavox account was incorrect.
- Fixed a problem where the application could crash after closing Track Status window.
- Fixed a problem where Track Status window didn't close after the application was hibernated.

Release Version 1.70.0

Release date: 2022-11-08

Improvements

- Updated user interface of the configuration guide.
- Fixed a bug where it was possible to activate buttons behind shown spinner indicator in the Feedback view.

Release Version 1.69.0

Release date: 2022-10-25

New Features

- Added a drawing mode that helps to draw using eye gaze.

Improvements

- Fixed navigation between emoji keyboard layout and other keyboard layouts.
- Updated animations of main page buttons in Settings.
- Updated orientation guide for the users who use gaze and switch as activation method.
- Fixed a problem where off-screen menu remained shown in the orientation guide after exiting practice mode in the Off-screen menu page.
- Fixed a problem where it was possible to crash the application when using it with certain devices.
- Fixed a problem where it was not possible to click on parts of certain buttons using physical mouse.
- Fixed a visual issue in keyboard language selection menu on small keyboard sizes.

Release Version 1.68.0

Release date: 2022-10-11

Improvements

- Fixed a bug where the word prediction text did not appear in the keyboard in the Feedback page in Settings.
- Updated text for the Font size setting.
- Updated text for the Switch input setting.
- Fixed a bug where it was not possible to activate back button in Adjust target using eye gaze when emojis keyboard was opened.
- Updated user interface of the orientation guide.

Release Version 1.67.0

Release date: 2022-09-27

Improvements

- Added a notification popup to the Feedback page in Settings to notify users when exiting the view without sending the written message.
- Fixed a bug where the word prediction buttons in Japanese keyboard were activated multiple times when using switch as the activation method.
- Fixed a bug where in certain cases users could get additional keyboard languages panel without any languages being shown.
- Fixed a bug where sometimes it was possible to accidentally send a message while the keyboard was opened in the Feedback page in Settings.
- Fixed a bug where it was possible to bring up Windows mouse cursor while using eye gaze in the Repeat Click mode.

Release Version 1.66.0

Release date: 2022-09-13

Improvements

- Aligned Sogou keyboard layout.
- Added underscore keys to certain keyboard layouts.
- Decreased size of the off-screen menu notification popup in the keyboard menu.
- Fixed a bug where double click in Continuous mode with switch activation method didn't work properly.

Release Version 1.65.0

Release date: 2022-08-30

New Features

- Added emojis keyboard layout.
- Added a possibility to send feedback about the application to Tobii Dynavox.
- Added a possibility to switch between keyboard layouts within the keyboard.
- Added a possibility to hide Trace while using the keyboard.
- Added a possibility to change the font size of the application text.

Improvements

- Added a possibility to execute an action directly after fixation in Continuous Mode.
- Updated introduction steps that are shown the first time user start the application to better explain the basic functionality of the software.

Release Version 1.64.0

Release date: 2022-08-16

Improvements

- Added a possibility to access off-screen menu in Repeat Click mode with mouse or touch input even when it is locked.
- Notification to look below the screen to open off-screen menu is shown directly after the application start-up.
- Added information about release dates in the release notes view.
- Updated scrolling in release notes view to look the same as it does in other parts of the application.

Release Version 1.63.0

Release date: 2022-08-01

New Features

- Added a possibility to view release notes for older releases of the application.

Improvements

- Partner window will show the text from the notepad right after the notepad is opened.
- Removed redundant display setup step from the initial onboarding process of the application.

Release Version 1.62.0

Release date: 2022-07-05

New Features

- Updated the icon for adding/removing keyboard languages.
- Fixed a bug where word predictions were not cleared when switching languages.
- Fixed a bug where the Esc key did not work on the PinYin keyboard.

Improvements

- Changed the UI of check boxes in Settings.
- Fixed a bug where text was added to the notepad even if it was hidden.

Release Version 1.61.0

Release date: 2022-06-21

Improvements

- Changed the UI of check boxes in Settings.
- Fixed a bug where text was added to the notepad even if it was hidden.

Release Version 1.60.0

Release date: 2022-06-07

Improvements

- Notepad will no longer be cleared when closed.
- Updated the explanation text for Continuous Mode.
- Fixed a problem where popups could remain open.

Release Version 1.59.0

Release date: 2022-05-24

New Features

- Added the ability to switch between modes.
- Added a way to start the keyboard from the off-screen menu.
- Added the ability to have labels in Interaction Menu.

Improvements

- Changed where Adjust Target is being shown in Continuous Mode.
- Improved the Chinese Keyboard.
- Fixed a problem with Sogou predictions.
- Updated the collect logs confirmation pop up.

- Fixed a potential crash when adjusting target.
- Fixed a potential crash when download new keyboard languages.

Release Version 1.58.0

Release date: 2022-05-10

New Features

- Added Quick Start Guide for the off-screen menu.

Improvements

- Updated the explanation of fixation time.
- Renamed Advanced Continuous Mode to Continuous Mode and Simplified Continuous Mode to Repeat Click.
- Adjusted the size of the icons in Customization.
- Updated Privacy Policy and User License Agreement.
- Fixed a problem with installing TD Control.
- Improved performance of TD Control.
- Fixed a bug that could potentially crash TD Control.

Release Version 1.57.0

Release date: 2022-04-26

Improvements

- Improved Japanese word predictions.
- Added the ability to clear the Notepad.
- Fixed Japanese translation error.
- Added a title to all the Quick Start Guides.
- Improved the usability of the Quick Start Guides.
- Fixed a problem with the Icelandic keyboard.
- Fixed a problem with the Czech keyboard.
- Fixed a bug that could potentially crash TD Control.
- Fixed a cut-off label in Account Login.

Release Version 1.56.0

Release date: 2022-04-12

Improvements

- Added the ability to turn off the Trace.
- Updated the spacing between buttons.

Release Version 1.55.0

Release date: 2022-03-29

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Simplified Continuous Mode will now stay activated after the computer returns from sleep.
- Removed the ability to have multiple buttons with the same action in Continuous Mode.
- Problem fixed that could potentially make TD Control crash.

Release Version 1.54.0

Release date: 2022-03-15

New Features

- Added Double Click to Continuous Mode.

Improvements

- Fixed a problem where TD Control could be visible over other applications while using the adaptive buttons.
- Improved the performance of the keyboard.
- Improved download speed of Chinese Keyboard.
- Fixed a problem where the trace was shown over close button.
- Fixed a problem where Adjust Target did not support mouse.
- Fixed a problem where update could fail.
- Fixed the UI of the close button.
- Fixed Japanese translation errors.

Release Version 1.53.0

Release date: 2022-02-15

New Features

- Added a possibility to restore the default settings in Customization.

Improvements

- Updated the style of Quick Start Guides.
- Fixed the label in Advanced Continuous Mode.
- Fixed layout of Splash Screen when Control is updating.
- Fixed a problem with the Czech keyboard.

Release Version 1.52.0

Release date: 2022-02-01

Improvements

- Added a possibility to restore the default setting in Customization.
- Panning will no longer be activated while off-screen menu is opened.
- Changed the accent color to TD Control.
- Control updater will update or repair TD Control in the background if needed.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- TD Control will not preserve display setup in UI between sessions.
- Added Windows property for eye tracked application detection.
- Changed the structure of the logs collected by TD Control.
- Added consistent behavior while using mouse in the keyboard off-screen menu.
- Fixed a bug that could potentially make Control crash.

Release Version 1.51.0

Release date: 2022-01-18

New Features

- Added the ability to Customize the Advanced Continuous Mode.
- Added release notes in Settings.
- Added a new notification when starting Control after an update.
- Added backup of word predictions that can be used in case of corrupted word prediction file.

Improvements

- Fixed a bug that could potentially make TD Control crash.
- Removed apostrophes from word Chinese word predictions.
- Fixed a problem where you could click outside the view in Adjust Target using a switch.
- Added Hold and Drag with Right Click to Continuous Mode.
- Added right click for Continuous Mode.
- Removed Windows Notifications for release notes.

Release Version 1.50.0

Release date: 2021-12-21

Improvements

- Fixed an issue with displaying the correct text on selected keyboard language.
- Removed text of Copy, Cut and Paste from the Dashboard.
- Left side buttons in Interaction Menu will not be shown as default any more.
- Fixed a problem where Control did not update.
- Fixed a problem where the Launcher does not forward arguments.
- Fixed a problem where Adaptive Buttons kept updating status on PCEye 5.
- Fixed a problem with synchronizing Interaction Menu.

Release Version 1.49.1

Release date: 2021-12-07

New Features

- New Chinese Keyboard with Sogou Predictions.
- Control now shows an indicator when updating.

Improvements

- Fixed a bug where Control sometimes was unable to click when using Manual Adjust Target.
- Improved keyboard experience for users with fast dwell speed.
- Fixed a problem where users could get a big offset in their calibration.
- Fixed a problem where word prediction backups were not uploaded.
- Fixed a potential crash in the calibration process.
- Added title to track status.

Release Version 1.48.2

Release date: 2021-11-23

Improvements

- Fixed a problem where some games did not detect the keyboard.
- Fixed a problem where two languages could be shown at the same time.
- Improved keyboard experience for users with fast dwell speed.

Release Version 1.47.0

Release date: 2021-11-09

New Features

- Added release notes in Settings.

Improvements

- Improved the pause indication.
- Fixed a bug that could affect scrolling behavior.
- Fixed a problem where Control could not detect what version of Switcher that is installed.
- Fixed a bug where regions behind popups were clickable.
- Fixed inconsistency in the Quick Start Guides.
- Removed the internal App Switcher.

Release Version 1.46.1

Release date: 2021-10-26

New Features

- Added XBOX Controller for Game On and Adaptive Controller.

Improvements

- Fixed inconsistency in modifier buttons.
- Fixed a bug that could potentially make TD Control crash.
- Corrected Chinese translation errors.
- Enables the feature flag ConfigurableInteractionMenu

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.45.0

Release date: 2021-10-12

New Features

- Changed Control to TD Control.

Improvements

- Updated the Customization View to mimic the Interaction menu better.
- Fixed a bug where Control Hibernation did not work while using Teams.
- Fixed a bug that could potentially make Control crash.
- Fixed a problem where Guides could not be closed using mouse.
- Fixed incorrect Japanese translations.

Release Version 1.44.0

Release date: 2021-09-28

New Features

- Added a new Chinese keyboard with Soguo predictions.

Improvements

- Disabled Windows IME when using Chinese or Japanese keyboards.
- Fixed a potential crash.

Release Version 1.43.0

Release date: 2021-09-14

New Features

- Added support for hibernation even when recording the screen.

Improvements

- Added underscore to the French Keyboard.
- Fixed a problem where control did not work in some games.

Release Version 1.42.0

Release date: 2021-08-31

New Features

- Added the ability to customize the side panel in the Interaction menu.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Updated the gazable area in the off-screen menu.
- Updated the breadcrumbs in Settings.
- Improved the text "Update available" so it can be seen on different background colors.

Release Version 1.41.0

Release date: 2021-08-17

New Features

- Added a notification in the more menu that is shown when an update for Control is available.

Improvements

- Updated the look and feel of settings.
- Improved activations in corners.
- Updated the tab icon on the keyboard.
- Added an underscore to the German keyboard.
- Fixed an issue where the letter Ø on the Danish keyboard could not be capitalized.
- Fixed a potential crash when trying to open the interaction menu.
- Fixed an issue where Control would be shown over Switcher after exiting secure mode.

Release Version 1.40.0

Release date: 2021-08-03

Improvements

- Fixed a problem with Hibernation while using screen sharing.
- Fix a problem where the Control Trace could be visible over Switcher.
- Updated the off-screen menus.
- Keyboard prediction dwell time will now sync with other applications in the Accessibility Suite.
- Updated the round gaze able buttons to follow the Design Guide.

Release Version 1.39.0

Release date: 2021-07-06

New Features

- Added ability to customize the interaction menu
- Added the possibility for other apps to detect if Control is in hibernation

Improvements

- Settings will now be synced with other apps without the need for change
- Users will not be asked if they are using a bracket on iSeries
- Fixed a problem where opening the keyboard started panning while gaming

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Fixed a bug that could potentially make Control crash

Release Version 1.38.0

Release date: 2021-06-22

Improvements

- Control now adjusts to resolution changes in Windows
- Added missing letter to Bulgarian keyboard
- Improved terms and conditions dialog
- Improved logout dialog
- Fixed bug that would make Control crash (ApplyDisplayConfiguration)
- Fixed bug that would make Control crash (SharedSettingsManager)
- Fixed bug that would make Control crash (AdaptiveButtons)
- Fixed Japanese translations in Calibration settings dialog

Release Version 1.37.0

Release date: 2021-06-08

New Features

- You can now use the adaptive buttons on the I-Series for track status and calibration.

Improvements

- Fixed a bug where it was sometimes difficult to close the keyboard.
- Fixed a bug that could cause Control to crash on the I-Series.
- Fixed a Japanese translation error in settings.
- Some animations in the interaction menu now have a shorter duration.
- The off-screen menu has been updated for a more coherent feel.
- The Account settings UI has been improved.

Release Version 1.36.1

Release date: 2021-05-25

New Features

- Control will now open Switcher if that is installed on the device.

Improvements

- Fixed a bug where Control did not go into hibernation.
- Fixed a bug that could cause Control to crash.
- Fixed a bug that could cause the keyboard to be shown when clicking.
- Fixed a bug that caused the mouse cursor to move unexpectedly.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.35.0

Release date: 2021-05-11

New Features

- Added explanation text to the Quick Start Guide in Simplified Continuous Mode.
- Adjusted mouse behavior to be able to pan the camera while playing games.
- Updated texts for Adjust Target in Settings.

Improvements

- Fixed an issue where calibrations were always based on both eyes, even if the user had chosen to only track left/right eye.

Release Version 1.34.0

Release date: 2021-04-27

New Features

- Added ability to set a precise fixation time.

Improvements

- Fixed a problem where Control could stop working.
- Improved the updating process.

Release Version 1.33.0

Release date: 2021-04-13

New Features

- Added the ability to collect logs from Settings / General.

Release Version 1.32.1

Release date: 2021-03-30

New Features

- Added Auto Adjust Target to settings.
- Increased logging for Updater and Launcher.

Improvements

- Fixed a bug where the keyboard was accidentally closed while using Always Adjust Target.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.30.1

Release date: 2021-03-16

New Features

- Added ability for Control to be inactivated but keep running in the background.
- Added more data to Segment.

Improvements

- Fixed a problem that could cause Control to crash.

Release Version 1.29.0

Release date: 2021-03-02

New Features

- Added Setting for Always Adjust Target before Interaction menu.

Improvements

- Fixed Prolonged sound key in the Japanese Keyboard.
- Fixed a bug where regions behind the loading spinner were still active.

Release Version 1.28.0

Release date:

New Features

- Added device_id to the metrics.
- Updated texts in settings.

Improvements

- Fixed a bug that could cause the application to crash.
- Fixed Japanese translation error.

Release Version 1.26.0

Release date: 2021-01-19

New Features

- Simplified Continuous Mode will now preserve states.
- User credentials will now be stored in a common place so other applications can use them.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed a bug that caused Computer Control not to startup automatically.
- Logout from Account revokes the user tokens.

Release Version 1.25.1

Release date: 2020-12-08

New Features

- Continuous Mode can be set as the default startup mode.
- In Simplified Continuous Mode track status can be toggled and off-screen menu can be locked with keyboard shortcuts.

Improvements

- The outer circle in the trace has been removed in Simplified Continuous Mode.
- Fixed a bug that could cause the updater to stop working.
- Improvements to the startup time.
- Fixed scrolling issues when CNN is disabled.
- Tweaked scroll values

Release Version 1.24.2

Release date: 2020-11-25

Improvements

- Fixed a bug that made scrolling unusable with a non-100% windows scaling setting.

Release Version 1.24.1

Release date: 2020-11-25

Improvements

- Fixed scrolling issues when CNN is disabled.

Release Version 1.24.0

Release date: 2020-11-24

New Features

- Added support for the adaptive buttons on the I-Series
- Added ability to change the settings on the adaptive buttons

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Improved the scrolling experience

Release Version 1.23.0

Release date: 2020-11-10

New Features

- Added ability to search for installed applications and pin them in the App-Switcher.

Improvements

- Added Prolonge sound key and brackets to the Japanese keyboard.
- Changed the animation for Hold and Drag.
- Changed the region and animation for the off-screen menu.

Release Version 1.22.1

Release date: 2020-10-27

New Features

- Added Chinese Stroke Input method to the Chinese keyboard
- Added bracket support
- Added restart to the Quick Start Guide
- Added Turkish keyboard and predictions
- Added Slovak keyboard and predictions
- Added activator data to Segment.io
- Added the ability to pass arguments through Computer Controls launcher

Improvements

- The first login to account on a new device will now download saved predictions
- Fixed a bug where Computer Control could run with PCEye mini
- Restored Click With Modifiers buttons that were accidentally hidden in 1.22.0

Release Version 1.21.0

Release date: 2020-10-12

New Features

- Number of apps in the App Switcher increased from 12 to 16.
- App Switcher can now pin and launch applications.
- Added a Quick Start Guide for Simplified Continuous Mode.
- Added Serbian keyboard layout and (Latin) word prediction.
- Added Xhosa keyboard layout and word prediction.
- Added Zulu keyboard layout and word prediction.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Several bug fixes and improvements to Simplified Continuous Mode.
- Settings UI has been adjusted to fit translations.
- Simplified Continuous Mode now properly hides all cursors.
- Fixed a bug where the Display Setup would be shown behind the Computer Control introduction view.
- Fixed a problem where Computer Control could not upload Word Prediction data to the Web Service.

Release Version 1.20.0

Release date: 2020-09-22

New Features

- Added possibility to download your predictions to a new device.
- Removed the center circle from the trace when CNN is turned off.
- Change so you will go back to settings after failing a calibration.

Improvements

- Fixed the trace color on Simplified Continuous Mode.
- Fixed a bug where keyboard did not work in Profiles for Switch users.
- Fixed text regions in Quick Start Guides.

Release Version 1.19.1

Release date: 2020-08-31

New Features

- The App-Switcher is now located in the off-screen menu.
- UWP apps are now shown inside the App-Switcher.
- Simplified Continuous Mode for left click is now available.

Improvements

- StreamEngine updated to version 4.8.
- InteractionLib updated to version 1.2.
- Fixed a bug in the new Simplified Continuous Mode.
- Continuous modes will now pause trace when mouse is moved.

Release Version 1.18.0

Release date: 2020-08-10

New Features

- Redesigned the Interaction Menu.
- The Chinese and Japanese keyboards will no longer insert a space after characters.
- Replaced "Sign up" with "Create Account" on the login page.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Changed "''" to "?" on the Armenian keyboard.
- Added underscore to the Swedish keyboard.

Improvements

- Fixed a bug that could crash Computer Control. (#1 / #2 / #3)

Release Version 1.17.0

Release date: 2020-07-21

New Features

- Added app-switcher to the menu.
- Added backup for predictions.
- Added automatic sign in.

Improvements

- Fixed a bug where exceptions were not reported to App Insights.
- Fixed crashes reported in Insights: (#1, #2).
- Added missing data in Metrics.
- Fixed German translations errors.

Release Version 1.16.1

Release date: 2020-06-30

New Features

- Added modifier keys to click-actions.
- Added PinYin keyboard.
- Added Terms and Conditions to creating account view.
- Added loading view in account settings.

Improvements

- Fixed a problem where the keyboard could get stuck in a floating state.
- Fixed German translations bugs.
- Fixed background color of increase and decrease buttons.
- Fixed an issue where buttons appeared to jump on selections.
- Fixed crash that could occur while editing account offline.
- Fixed crash that could occur while navigating the new keyboard settings multi page view.

Release Version 1.15.1

Release date: 2020-06-16

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed crash that could occur while navigating the new keyboard settings multi page view.

Release Version 1.15.0

Release date: 2020-06-08

New Features

- Added option to keep device setting our update from saved profile when logging in.
- Added setting for switch users to select keyboard activation method.
- Added toggle to show / hide password visibility.
- Users will no longer be logged out during restart.
- Email will be saved when logged out.
- Added text to error-message when failing to login.
- Attached keyboard to the bottom/top of the screen.

Improvements

- Fixed bug where fixations could cause unintentional clicks.
- Fixed a bug where error messages in create account did not clear.
- Fixed bug where edit button did not remove focus.
- Fixed a bug that caused list of countries not to update when changing the app language.
- Added indicator for vowels in the Arabic keyboard.
- Updated "Cancel" buttons to be consistent with Windows.
- Fixed a bug where icons and labels would flash in settings view.
- Updated Click & Drag colors to be the same as the Trace
- Fixed bug with sound feedback on keyboard.

Release Version 1.14.0

Release date: 2020-05-19

New Features

- Added auto fill to search-box for countries when you try to create a new myTobiiDynavox-account.
- Added messages to myTobiiDynavox login.

Improvements

- Fixed a bug on the Russian keyboard.
- Fixed keyboard quick start guide in German.
- Fixed keyboard settings text in Spanish.
- Fixed Chinese translation errors.
- Fixed an error that could cause the app to crash.

Release Version 1.13.0

Release date: 2020-04-28

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

New Features

- Added restart-button to the exit screen
- Added dwell speed settings on activation

Improvements

- Fixed a bug where off-screen menu didn't close while not gazed on
- Fixed UI bug in settings
- Fixed bug with switch button interaction
- Fixed translation error in settings / activation. German version
- Fixed translation error in settings / general. German version
- Fixed translation error in quick start guide. German version

Release Version 1.12.0

Release date: 2020-04-14

New Features

- Changed animation on calibration to create a better fixation point
- Different dwell times on keyboard keys and predictions
- Added sound feedback to the keyboard
- The keyboard won't disappear when a click action is cancelled

Improvements

- Fixed a bug where the user could get stuck while using a switch
- Fixed a bug that could bring you back to Communicator while using the keyboard

Release Version 1.11.0

Release date: 2020-03-31

New Features

- Added Segment.io analytics
- Changed optimization on adaptive dwell speed for keyboards
- Fixating near the bottom of the screen no longer activates off-screen menu
- Possible to perform display setup for all installed software
- Returning from sleep now activates tracker
- Improved scrolling
- Enabled switching between languages from keyboard
- Added switch for predictions on/off within keyboard settings
- Adjust target on click and drag can now be turned on/off from within settings
- Keyboard no longer inserts space after punctuation
- Added start guide to continuous mode
- Added UDI/EAN13 code to settings

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed Norwegian translation
- Fixed translation in German
- Fixed Spanish translation
- Fixed missing "ج" on Arabic keyboard
- Fixed descriptive text in settings on I-13
- Computer Control is now explicitly referenced in all texts

Release Version 1.10.0

Release date: 2020-03-17

New Features

- Added functionality to allow a user to send text from the note pad partner window on an iSeries
- Added automatic syncing of keyboard language setting with web-service storage.
- Added the ability for a user to select their interaction menu them selves.
- Added the ability for a user to click on hover menus.
- Added improvements to the trace.
- Added functionality that would keep the off-screen menu open once the off-screen keyboard info screen is closed.

Improvements

- Added the "," on the first page of the Danish keyboard.

Release Version 1.9.0

Release date: 2020-03-03

New Features

- A user is unable to turn on switch activation using Eye Gaze and requires manual input
- A user now has a UI to be able to manage their account in Computer Control
- Computer Control will now be able to update on locked devices such as iSeries.
- Keyboard off screen menu has an introduction notification.
- The off-screen menu can now be activated through touch or mouse without needing to block eye gaze data.

Release Version 1.7.0

Release date:

New Features

- Now you can use the same pause functionality in the keyboard.
- New languages have been added: Czech, Bulgarian, Hungarian, Afrikaans and Spanish.
- Delay when selecting Control or Shift is reduced.
- Computer Control now reports version numbers to analytics

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.5.0

Release date: 2019-12-20

New Features

- Continuous functions now have an off screen menu.
- The trace now has a smoother filter when tracking a single eye.

Improvements

- Fixed a bug where snapshots in the interaction view were incorrectly scaled.
- Fixed a bug that was causing eye tracking to stop working on the new I-Series.

Release Version 1.4.0

Release date: 2019-12-10

New Features

- Adjust target can now be used to end an interaction like click and drag
- You can now choose to track just one eye
- The pause button is faster to access
- Specific keys in the keyboard now have more explicit dwell times for added control
- Now you can perform a 'click and hold' function
- The off-screen menu now hides immediately when you fixate

Improvements

- Fixed a bug where the keyboard had issues handling uppercase text.
- Fixed bug where scrolling continued to be active even after the function had been cancelled.
- Fixed a bug where the Track Status disappears on the Test Calibration view.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact