

Release Notes TD Control

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 1.57.0

Release date: 2022-04-26

Improvements

- Improved Japanese word predictions.
- Added the ability to clear the Notepad.
- Fixed Japanese translation error.
- Added a title to all the Quick Start Guides.
- Improved the usability of the Quick Start Guides.
- Fixed a problem with the Icelandic keyboard.
- Fixed a problem with the Czech keyboard.
- Fixed a bug that could potentially crash TD Control.
- Fixed a cut-off label in Account Login.

Release Version 1.56.0

Release date: 2022-04-12

Improvements

- Added the ability to turn off the Trace.
- Updated the spacing between buttons.

Release Version 1.55.0

Release date: 2022-03-29

Improvements

- Simplified Continuous Mode will now stay activated after the computer returns from sleep.
- Removed the ability to have multiple buttons with the same action in Continuous Mode.
- Problem fixed that could potentially make TD Control crash.

Release Version 1.54.0

Release date: 2022-03-15

New Features

- Added Double Click to Continuous Mode.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed a problem where TD Control could be visible over other applications while using the adaptive buttons.
- Improved the performance of the keyboard.
- Improved download speed of Chinese Keyboard.
- Fixed a problem where the trace was shown over close button.
- Fixed a problem where Adjust Target did not support mouse.
- Fixed a problem where update could fail.
- Fixed the UI of the close button.
- Fixed Japanese translation errors.

Release Version 1.53.0

Release date: 2022-02-15

New Features

- Added a possibility to restore the default settings in Customization.

Improvements

- Updated the style of Quick Start Guides.
- Fixed the label in Advanced Continuous Mode.
- Fixed layout of Splash Screen when Control is updating.
- Fixed a problem with the Czech keyboard.

Release Version 1.52.0

Release date: 2022-02-01

Improvements

- Added a possibility to restore the default setting in Customization.
- Panning will no longer be activated while off-screen menu is opened.
- Changed the accent color to TD Control.
- Control updater will update or repair TD Control in the background if needed.
- TD Control will not preserve display setup in UI between sessions.
- Added Windows property for eye tracked application detection.
- Changed the structure of the logs collected by TD Control.
- Added consistent behavior while using mouse in the keyboard off-screen menu.
- Fixed a bug that could potentially make Control crash.

Release Version 1.51.0

Release date: 2022-01-18

New Features

- Added the ability to Customize the Advanced Continuous Mode.
- Added release notes in Settings.
- Added a new notification when starting Control after an update.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Added backup of word predictions that can be used in case of corrupted word prediction file.

Improvements

- Fixed a bug that could potentially make TD Control crash.
- Removed apostrophes from word Chinese word predictions.
- Fixed a problem where you could click outside the view in Adjust Target using a switch.
- Added Hold and Drag with Right Click to Continuous Mode.
- Added right click for Continuous Mode.
- Removed Windows Notifications for release notes.

Release Version 1.50.0

Release date: 2021-12-21

Improvements

- Fixed an issue with displaying the correct text on selected keyboard language.
- Removed text of Copy, Cut and Paste from the Dashboard.
- Left side buttons in Interaction Menu will not be shown as default any more.
- Fixed a problem where Control did not update.
- Fixed a problem where the Launcher does not forward arguments.
- Fixed a problem where Adaptive Buttons kept updating status on PCEye 5.
- Fixed a problem with synchronizing Interaction Menu.

Release Version 1.49.1

Release date: 2021-12-07

New Features

- New Chinese Keyboard with Sogou Predictions.
- Control now shows an indicator when updating.

Improvements

- Fixed a bug where Control sometimes was unable to click when using Manual Adjust Target.
- Improved keyboard experience for users with fast dwell speed.
- Fixed a problem where users could get a big offset in their calibration.
- Fixed a problem where word prediction backups were not uploaded.
- Fixed a potential crash in the calibration process.
- Added title to track status.

Release Version 1.48.2

Release date: 2021-11-23

Improvements

- Fixed a problem where some games did not detect the keyboard.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Fixed a problem where two languages could be shown at the same time.
- Improved keyboard experience for users with fast dwell speed.

Release Version 1.47.0

Release date: 2021-11-09

New Features

- Added release notes in Settings.

Improvements

- Improved the pause indication.
- Fixed a bug that could affect scrolling behavior.
- Fixed a problem where Control could not detect what version of Switcher that is installed.
- Fixed a bug where regions behind popups were clickable.
- Fixed inconsistency in the Quick Start Guides.
- Removed the internal App Switcher.

Release Version 1.46.1

Release date: 2021-10-26

New Features

- Added XBOX Controller for Game On and Adaptive Controller.

Improvements

- Fixed inconsistency in modifier buttons.
- Fixed a bug that could potentially make TD Control crash.
- Corrected Chinese translation errors.
- Enables the feature flag ConfigurableInteractionMenu

Release Version 1.45.0

Release date: 2021-10-12

New Features

- Changed Control to TD Control.

Improvements

- Updated the Customization View to mimic the Interaction menu better.
- Fixed a bug where Control Hibernation did not work while using Teams.
- Fixed a bug that could potentially make Control crash.
- Fixed a problem where Guides could not be closed using mouse.
- Fixed incorrect Japanese translations.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.44.0

Release date: 2021-09-28

New Features

- Added a new Chinese keyboard with Soguo predictions.

Improvements

- Disabled Windows IME when using Chinese or Japanese keyboards.
- Fixed a potential crash.

Release Version 1.43.0

Release date: 2021-09-14

New Features

- Added support for hibernation even when recording the screen.

Improvements

- Added underscore to the French Keyboard.
- Fixed a problem where control did not work in some games.

Release Version 1.42.0

Release date: 2021-08-31

New Features

- Added the ability to customize the side panel in the Interaction menu.

Improvements

- Updated the gazable area in the off-screen menu.
- Updated the breadcrumbs in Settings.
- Improved the text "Update available" so it can be seen on different background colors.

Release Version 1.41.0

Release date: 2021-08-17

New Features

- Added a notification in the more menu that is shown when an update for Control is available.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Updated the look and feel of settings.
- Improved activations in corners.
- Updated the tab icon on the keyboard.
- Added an underscore to the German keyboard.
- Fixed an issue where the letter Ø on the Danish keyboard could not be capitalized.
- Fixed a potential crash when trying to open the interaction menu.
- Fixed an issue where Control would be shown over Switcher after exiting secure mode.

Release Version 1.40.0

Release date: 2021-08-03

Improvements

- Fixed a problem with Hibernation while using screen sharing.
- Fix a problem where the Control Trace could be visible over Switcher.
- Updated the off-screen menus.
- Keyboard prediction dwell time will now sync with other applications in the Accessibility Suite.
- Updated the round gaze able buttons to follow the Design Guide.

Release Version 1.39.0

Release date: 2021-07-06

New Features

- Added ability to customize the interaction menu
- Added the possibility for other apps to detect if Control is in hibernation

Improvements

- Settings will now be synced with other apps without the need for change
- Users will not be asked if they are using a bracket on iSeries
- Fixed a problem where opening the keyboard started panning while gaming
- Fixed a bug that could potentially make Control crash

Release Version 1.38.0

Release date: 2021-06-22

Improvements

- Control now adjusts to resolution changes in Windows
- Added missing letter to Bulgarian keyboard
- Improved terms and conditions dialog
- Improved logout dialog
- Fixed bug that would make Control crash (ApplyDisplayConfiguration)
- Fixed bug that would make Control crash (SharedSettingsManager)
- Fixed bug that would make Control crash (AdaptiveButtons)

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Fixed Japanese translations in Calibration settings dialog

Release Version 1.37.0

Release date: 2021-06-08

New Features

- You can now use the adaptive buttons on the I-Series for track status and calibration.

Improvements

- Fixed a bug where it was sometimes difficult to close the keyboard.
- Fixed a bug that could cause Control to crash on the I-Series.
- Fixed a Japanese translation error in settings.
- Some animations in the interaction menu now have a shorter duration.
- The off-screen menu has been updated for a more coherent feel.
- The Account settings UI has been improved.

Release Version 1.36.1

Release date: 2021-05-25

New Features

- Control will now open Switcher if that is installed on the device.

Improvements

- Fixed a bug where Control did not go into hibernation.
- Fixed a bug that could cause Control to crash.
- Fixed a bug that could cause the keyboard to be shown when clicking.
- Fixed a bug that caused the mouse cursor to move unexpectedly.

Release Version 1.35.0

Release date: 2021-05-11

New Features

- Added explanation text to the Quick Start Guide in Simplified Continuous Mode.
- Adjusted mouse behavior to be able to pan the camera while playing games.
- Updated texts for Adjust Target in Settings.

Improvements

- Fixed an issue where calibrations were always based on both eyes, even if the user had chosen to only track left/right eye.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.34.0

Release date: 2021-04-27

New Features

- Added ability to set a precise fixation time.

Improvements

- Fixed a problem where Control could stop working.
- Improved the updating process.

Release Version 1.33.0

Release date: 2021-04-13

New Features

- Added the ability to collect logs from Settings / General.

Release Version 1.32.1

Release date: 2021-03-30

New Features

- Added Auto Adjust Target to settings.
- Increased logging for Updater and Launcher.

Improvements

- Fixed a bug where the keyboard was accidentally closed while using Always Adjust Target.

Release Version 1.30.1

Release date: 2021-03-16

New Features

- Added ability for Control to be inactivated but keep running in the background.
- Added more data to Segment.

Improvements

- Fixed a problem that could cause Control to crash.

Release Version 1.29.0

Release date: 2021-03-02

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

New Features

- Added Setting for Always Adjust Target before Interaction menu.

Improvements

- Fixed Prolonged sound key in the Japanese Keyboard.
- Fixed a bug where regions behind the loading spinner were still active.

Release Version 1.28.0

Release date:

New Features

- Added device_id to the metrics.
- Updated texts in settings.

Improvements

- Fixed a bug that could cause the application to crash.
- Fixed Japanese translation error.

Release Version 1.26.0

Release date: 2021-01-19

New Features

- Simplified Continuous Mode will now preserve states.
- User credentials will now be stored in a common place so other applications can use them.

Improvements

- Fixed a bug that caused Computer Control not to startup automatically.
- Logout from Account revokes the user tokens.

Release Version 1.25.1

Release date: 2020-12-08

New Features

- Continuous Mode can be set as the default startup mode.
- In Simplified Continuous Mode track status can be toggled and off-screen menu can be locked with keyboard shortcuts.

Improvements

- The outer circle in the trace has been removed in Simplified Continuous Mode.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Fixed a bug that could cause the updater to stop working.
- Improvements to the startup time.
- Fixed scrolling issues when CNN is disabled.
- Tweaked scroll values

Release Version 1.24.2

Release date: 2020-11-25

Improvements

- Fixed a bug that made scrolling unusable with a non-100% windows scaling setting.

Release Version 1.24.1

Release date: 2020-11-25

Improvements

- Fixed scrolling issues when CNN is disabled.

Release Version 1.24.0

Release date: 2020-11-24

New Features

- Added support for the adaptive buttons on the I-Series
- Added ability to change the settings on the adaptive buttons

Improvements

- Improved the scrolling experience

Release Version 1.23.0

Release date: 2020-11-10

New Features

- Added ability to search for installed applications and pin them in the App-Switcher.

Improvements

- Added Prolong sound key and brackets to the Japanese keyboard.
- Changed the animation for Hold and Drag.
- Changed the region and animation for the off-screen menu.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Release Version 1.22.1

Release date: 2020-10-27

New Features

- Added Chinese Stroke Input method to the Chinese keyboard
- Added bracket support
- Added restart to the Quick Start Guide
- Added Turkish keyboard and predictions
- Added Slovak keyboard and predictions
- Added activator data to Segment.io
- Added the ability to pass arguments through Computer Controls launcher

Improvements

- The first login to account on a new device will now download saved predictions
- Fixed a bug where Computer Control could run with PCEye mini
- Restored Click With Modifiers buttons that were accidentally hidden in 1.22.0

Release Version 1.21.0

Release date: 2020-10-12

New Features

- Number of apps in the App Switcher increased from 12 to 16.
- App Switcher can now pin and launch applications.
- Added a Quick Start Guide for Simplified Continuous Mode.
- Added Serbian keyboard layout and (Latin) word prediction.
- Added Xhosa keyboard layout and word prediction.
- Added Zulu keyboard layout and word prediction.

Improvements

- Several bug fixes and improvements to Simplified Continuous Mode.
- Settings UI has been adjusted to fit translations.
- Simplified Continuous Mode now properly hides all cursors.
- Fixed a bug where the Display Setup would be shown behind the Computer Control introduction view.
- Fixed a problem where Computer Control could not upload Word Prediction data to the Web Service.

Release Version 1.20.0

Release date: 2020-09-22

New Features

- Added possibility to download your predictions to a new device.
- Removed the center circle from the trace when CNN is turned off.
- Change so you will go back to settings after failing a calibration.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed the trace color on Simplified Continuous Mode.
- Fixed a bug where keyboard did not work in Profiles for Switch users.
- Fixed text regions in Quick Start Guides.

Release Version 1.19.1

Release date: 2020-08-31

New Features

- The App-Switcher is now located in the off-screen menu.
- UWP apps are now shown inside the App-Switcher.
- Simplified Continuous Mode for left click is now available.

Improvements

- StreamEngine updated to version 4.8.
- InteractionLib updated to version 1.2.
- Fixed a bug in the new Simplified Continuous Mode.
- Continuous modes will now pause trace when mouse is moved.

Release Version 1.18.0

Release date: 2020-08-10

New Features

- Redesigned the Interaction Menu.
- The Chinese and Japanese keyboards will no longer insert a space after characters.
- Replaced "Sign up" with "Create Account" on the login page.
- Changed "™" to "?" on the Armenian keyboard.
- Added underscore to the Swedish keyboard.

Improvements

- Fixed a bug that could crash Computer Control. (#1 / #2 / #3)

Release Version 1.17.0

Release date: 2020-07-21

New Features

- Added app-switcher to the menu.
- Added backup for predictions.
- Added automatic sign in.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed a bug where exceptions were not reported to App Insights.
- Fixed crashes reported in Insights: (#1, #2).
- Added missing data in Metrics.
- Fixed German translations errors.

Release Version 1.16.1

Release date: 2020-06-30

New Features

- Added modifier keys to click-actions.
- Added PinYin keyboard.
- Added Terms and Conditions to creating account view.
- Added loading view in account settings.

Improvements

- Fixed a problem where the keyboard could get stuck in a floating state.
- Fixed German translations bugs.
- Fixed background color of increase and decrease buttons.
- Fixed an issue where buttons appeared to jump on selections.
- Fixed crash that could occur while editing account offline.
- Fixed crash that could occur while navigating the new keyboard settings multi page view.

Release Version 1.15.1

Release date: 2020-06-16

Improvements

- Fixed crash that could occur while navigating the new keyboard settings multi page view.

Release Version 1.15.0

Release date: 2020-06-08

New Features

- Added option to keep device setting our update from saved profile when logging in.
- Added setting for switch users to select keyboard activation method.
- Added toggle to show / hide password visibility.
- Users will no longer be logged out during restart.
- Email will be saved when logged out.
- Added text to error-message when failing to login.
- Attached keyboard to the bottom/top of the screen.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Improvements

- Fixed bug where fixations could cause unintentional clicks.
- Fixed a bug where error messages in create account did not clear.
- Fixed bug where edit button did not remove focus.
- Fixed a bug that caused list of countries not to update when changing the app language.
- Added indicator for vowels in the Arabic keyboard.
- Updated "Cancel" buttons to be consistent with Windows.
- Fixed a bug where icons and labels would flash in settings view.
- Updated Click & Drag colors to be the same as the Trace
- Fixed bug with sound feedback on keyboard.

Release Version 1.14.0

Release date: 2020-05-19

New Features

- Added auto fill to search-box for countries when you try to create a new myTobiiDynavox-account.
- Added messages to myTobiiDynavox login.

Improvements

- Fixed a bug on the Russian keyboard.
- Fixed keyboard quick start guide in German.
- Fixed keyboard settings text in Spanish.
- Fixed Chinese translation errors.
- Fixed an error that could cause the app to crash.

Release Version 1.13.0

Release date: 2020-04-28

New Features

- Added restart-button to the exit screen
- Added dwell speed settings on activation

Improvements

- Fixed a bug where off-screen menu didn't close while not gazed on
- Fixed UI bug in settings
- Fixed bug with switch button interaction
- Fixed translation error in settings / activation. German version
- Fixed translation error in settings / general. German version
- Fixed translation error in quick start guide. German version

Release Version 1.12.0

Release date: 2020-04-14

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

New Features

- Changed animation on calibration to create a better fixation point
- Different dwell times on keyboard keys and predictions
- Added sound feedback to the keyboard
- The keyboard won't disappear when a click action is cancelled

Improvements

- Fixed a bug where the user could get stuck while using a switch
- Fixed a bug that could bring you back to Communicator while using the keyboard

Release Version 1.11.0

Release date: 2020-03-31

New Features

- Added Segment.io analytics
- Changed optimization on adaptive dwell speed for keyboards
- Fixating near the bottom of the screen no longer activates off-screen menu
- Possible to perform display setup for all installed software
- Returning from sleep now activates tracker
- Improved scrolling
- Enabled switching between languages from keyboard
- Added switch for predictions on/off within keyboard settings
- Adjust target on click and drag can now be turned on/off from within settings
- Keyboard no longer inserts space after punctuation
- Added start guide to continuous mode
- Added UDI/EAN13 code to settings

Improvements

- Fixed Norwegian translation
- Fixed translation in German
- Fixed Spanish translation
- Fixed missing "ج" on Arabic keyboard
- Fixed descriptive text in settings on I-13
- Computer Control is now explicitly referenced in all texts

Release Version 1.10.0

Release date: 2020-03-17

New Features

- Added functionality to allow a user to send text from the note pad partner window on an iSeries
- Added automatic syncing of keyboard language setting with web-service storage.
- Added the ability for a user to select their interaction menu themselves.
- Added the ability for a user to click on hover menus.
- Added improvements to the trace.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

- Added functionality that would keep the off-screen menu open once the off-screen keyboard info screen is closed.

Improvements

- Added the "," on the first page of the Danish keyboard.

Release Version 1.9.0

Release date: 2020-03-03

New Features

- A user is unable to turn on switch activation using Eye Gaze and requires manual input
- A user now has a UI to be able to manage their account in Computer Control
- Computer Control will now be able to update on locked devices such as iSeries.
- Keyboard off screen menu has an introduction notification.
- The off-screen menu can now be activated through touch or mouse without needing to block eye gaze data.

Release Version 1.7.0

Release date:

New Features

- Now you can use the same pause functionality in the keyboard.
- New languages have been added: Czech, Bulgarian, Hungarian, Afrikaans and Spanish.
- Delay when selecting Control or Shift is reduced.
- Computer Control now reports version numbers to analytics

Release Version 1.5.0

Release date: 2019-12-20

New Features

- Continuous functions now have an off screen menu.
- The trace now has a smoother filter when tracking a single eye.

Improvements

- Fixed a bug where snapshots in the interaction view were incorrectly scaled.
- Fixed a bug that was causing eye tracking to stop working on the new I-Series.

Release Version 1.4.0

Release date: 2019-12-10

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

New Features

- Adjust target can now be used to end an interaction like click and drag
- You can now choose to track just one eye
- The pause button is faster to access
- Specific keys in the keyboard now have more explicit dwell times for added control
- Now you can perform a 'click and hold' function
- The off-screen menu now hides immediately when you fixate

Improvements

- Fixed a bug where the keyboard had issues handling uppercase text.
- Fixed bug where scrolling continued to be active even after the function had been cancelled.
- Fixed a bug where the Track Status disappears on the Test Calibration view.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact