

Tobii Dynavox Device Decontamination

Equipment Required

Hospital grade cleaning wipes or equivalent disinfectant wipe for infection control.

Applicable Products

Tobii Dynavox I-Series+	Tobii Dynavox PCEye 5	Tobii Dynavox Mobile Mini
Tobii Dynavox I-13/I-16	Tobii Dynavox EyeMobile Bracket	Tobii Dynavox AccessIT 3
TD I-13/I-16	Tobii Dynavox EyeMobile Plus	Tobii Dynavox EyeR
Tobii Dynavox I-110	Tobii Dynavox PCEye Bracket	Tobii M-Series
TD I-110	Tobii Dynavox Indi	Dynavox T-Series
Tobii Dynavox PCEye Explore	Tobii Dynavox Indi 7	TD Pilot
Tobii Dynavox PCEye Mini	Tobii Dynavox Speech Case	TD Speech Case Mini
Tobii Dynavox PCEye Plus	Tobii Dynavox SC Tablet	TD SC Tablet Mini

Procedure

Decontaminate and Clean

Cleaning of the Device

1. Before cleaning, fully shut down your device.
2. Unplug your device.
3. Remove all cables.
4. Wipe over all external surfaces using 70 percent isopropyl alcohol wipe, 75 percent ethyl alcohol wipe, or Clorox Disinfecting Wipe for infection control.
5. Allow the device to dry naturally.
6. Any accessories included must also be disinfected in the same way.
7. If streaky residue remains from cleaning, buff screen with a soft dry cloth.
8. Dispose of used cleaning materials properly.



Do not use spray products directly on to the device as these may saturate the device and allow unwanted moisture into the unit.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact