

Release Notes Tobii Dynavox Hardware Listener

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 1.9.3

Release date: 2024-10-21

Improvements

• Fixed an issue where Hardware Listener was not running at Windows startup.

Release Version 1.9.1

Release date: 2024-10-15

Improvements

Bug fixes

Release Version 1.8.3

Release date: 2024-05-21

Improvements

• Update for I-Series device's Adaptive Button action "Show/Hide Track Status" to work with TD Snap track status, TD Snap version 1.32 or above.

Release Version 1.8.2

Release date: 2024-02-27

Improvements

• Fixed an issue for I-Series devices where the Adaptive Button action "Enable/Disable Eye Tracker Pause" would not work in TD Snap. Requires TD Snap version 1.31 or above.

Release Version 1.8.0

Release date: 2023-03-06

Improvements

· Fixed an issue where logging was not working.

Release Version 1.7.0

Release date: 2022-06-13

Copyright @ Dynavox Group AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owner.

Support for Your Tobii Dynavox Device

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: https://www.tobiidynavox.com/pages/product-support



New Features

· Verifies Partner Window driver at startup and installs correct one if not present.

Release Version 1.6.0

Release date: 2022-02-01

Improvements

• Resolved issue with TD Snap not restoring after closing Track Status.

Release Version 1.5.4

Release date: 2021-12-07

Improvements

• Bug fixes

Release Version 1.5.3

Release date: 2021-07-06

Improvements

Bug fixes

Release Version 1.5.2

Release date: 2021-05-10

New Features

Automatic headphone jack detection (I-13/I-16 only)
 If enabled, the audio output device is automatically changed when headphones are plugged in or unplugged.

Release Version 1.4.0

Release date: 2021-02-08

New Features

- Ability to assign mouse click actions to switches (I-13/I-16 only)
 - Left Click
 - Middle Click
 - Right Click
- Ability to assign start program to switches (I-13/I-16 only)
 - Select a program to start when a switch is selected
- Ability to assign volume actions to switches (I-13/I-16 only)

Copyright © Dynavox Group AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owner.

Support for Your Tobii Dynavox Device

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: https://www.tobiidynavox.com/pages/product-support

#12008793 Release Notes Tobii Dynavox Hardware Listener v.1.9.3 - en-US

tobii dynavox

- Enabled/Disable Mute
- Volume Down
- Volume Up
- Ability to assign eye tracker actions to switches (I-13/I-16 only)
 - Show/Hide Track Status
 - Enable/Disable Eye Tracker Pause
 - Calibrate Eye Tracker
 - Enable/Disable Computer Control

Release Version 1.3.4

Release date: 2020-12-07

Improvements

• Bug fixes

Release Version 1.3.3

Release date: 2020-11-10

New Features

- Ability to assign power off device action to switches (I-13/I-16 only)
 - Press and hold 2-7 seconds for normal Windows shutdown
 - Press and hold 7 seconds or more for forced Windows shutdown

Improvements

Bug fixes

Release Version 1.3.1

Release date: 2020-08-10

Improvements

• Bug fixes

Release Version 1.3.0

Release date: 2020-06-08

New Features

• Ability to assign keyboard key actions to switches (I-13/I-16 only)

Release Version 1.2.0

Release date: 2020-04-08

Copyright @ Dynavox Group AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owner.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: https://www.tobiidynavox.com/pages/product-support



Improvements

• Hardware Listener separated from Hardware Settings into its own installer

Support for Your Tobii Dynavox Device

Get Help Online
See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: https://www.tobiidynavox.com/pages/product-support

Contact Your Solution Consultant or Reseller