

## Release Notes Tobii Dynavox Hardware Listener

**Customer support:** For more information or support, please visit [www.TobiiDynavox.com](http://www.TobiiDynavox.com).

### Release Version 1.4.0.249

**Release date:** 2021-02-08

#### New Features

- Ability to assign mouse click actions to switches (I-13/I-16 only)
  - Left Click
  - Middle Click
  - Right Click
- Ability to assign start program to switches (I-13/I-16 only)
  - Select a program to start when a switch is selected
- Ability to assign volume actions to switches (I-13/I-16 only)
  - Enabled/Disable Mute
  - Volume Down
  - Volume Up
- Ability to assign eye tracker actions to switches (I-13/I-16 only)
  - Show/Hide Track Status
  - Enable/Disable Eye Tracker Pause
  - Calibrate Eye Tracker
  - Enable/Disable Computer Control

### Release Version 1.3.4.243

**Release date:** 2020-12-07

#### Improvements

- Bug fixes

### Release Version 1.3.3.240

**Release date:** 2020-11-10

#### New Features

- Ability to assign power off device action to switches (I-13/I-16 only)
  - Press and hold 2-7 seconds for normal Windows shutdown
  - Press and hold 7 seconds or more for forced Windows shutdown

#### Support for Your Tobii Dynavox Device

##### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com/support-training](http://www.TobiiDynavox.com/support-training)

##### Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## Improvements

- Bug fixes

## Release Version 1.3.1.234

**Release date:** 2020-08-10

## Improvements

- Bug fixes

## Release Version 1.3.0.230

**Release date:** 2020-06-08

## New Features

- Ability to assign keyboard key actions to switches (I-13/I-16 only)

## Release Version 1.2.0.223

**Release date:** 2020-04-08

## Improvements

- Hardware Listener separated from Hardware Settings into its own installer

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com/support-training](http://www.TobiiDynavox.com/support-training)

### Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)