English

Safety and Compliance I-110

1 Safety

The Tobii Dynavox I-Series device has been tested and approved as compliant to all the Specifications and Standards listed in 2 *Compliance Information*, page 3 of this manual and in the 7 *Technical Specifications*, page 5. Nevertheless, in order to ensure safe operation of your Tobii Dynavox I-Series device, there are a few safety warnings to bear in mind:

No modification of this equipment is allowed

The Tobii Dynavox I-Series is not to be used as a life supporting Device, and it shall not be relied on in case of loss of function due to power loss or other causes.

The Tobii Dynavox I-Series device is designed to meet a high ingress protection rating, meaning it will withstand light rain or snow. However, it shall not be exposed to or used in heavy rain or weather conditions outside the Technical Specification of the Tobii Dynavox I-Series device.

The Tobii Dynavox I-Series device is designed to be rugged and durable. It shall still be handled and used with care when moving around.

There could be a choking hazard risk if small parts detach from the Tobii Dynavox I-Series device.

The Tobii Dynavox I-Series device shall only be used with Tobii Dynavox I-Series specific Accessories that have a mounting instruction included with the accessory.

Young children or people with cognitive disabilities should not have access to, or the use of, the Tobii Dynavox I-Series device, with or without carry strap or other accessories, without parental or guardian supervision.

1.1 Avoiding Hearing Damage

Permanent hearing loss may occur if earphones, headphones or speakers are used at high volume. To prevent this, the volume should be set to a safe level. You can become desensitized over time to high sound levels which may then sound acceptable, yet still could be damaging your hearing. If you experience symptoms such as ringing in your ears, please lower the volume or stop using the earphones/headphones. The louder the volume, the less time is required before your hearing could be affected.

Hearing experts suggest the following measures to protect your hearing:

- Limit the amount of time you use earphones or headphones at high volume.
- Avoid turning up the volume to block out noisy surroundings.
- Turn the volume down if you cannot hear people speaking near you.

To establish a safe volume level:

- Set your volume control at a low setting.
- Slowly increase the sound until you can hear it comfortably and clearly, without distortion.

1.2 Temperature

Due to use in direct sunshine or in any other hot environment the Tobii Dynavox I-Series device may have hot surfaces. There is a built in temperature sensor that monitors the temperature. If the sensor detects temperatures above the temperature specified in *Table 6.1 Maximum Allowed Hotspot Temperature*, page 5 the device will automatically trigger a Windows Shut Down to not cause the User any risk of harm. It may take a while before Tobii Dynavox I-Series device can be restarted as the unit may first need time to cool down.

1.3 Power Supply and Batteries

The Tobii Dynavox I-Series device contains a rechargeable battery. All rechargeable batteries degrade over time. Thus the possible usage times for the Tobii Dynavox I-Series after a full charge can become shorter over time than when the device was new.

The Tobii Dynavox I-Series device uses a Li-ion battery.

If you are in a hot environment, be aware that it can affect the ability to charge the battery. The internal temperature has to be between 0 °C/32 °F and 45 °C/113 °F for the battery to charge. If the internal battery temperature raises above 45 °C/113 °F the battery will not charge at all.

Move the Tobii Dynavox I-Series device to a cooler environment to let the battery charge properly.

Avoid exposing the Tobii Dynavox I-Series device to fire or to temperatures above 60 °C/140 °F. These conditions may cause the battery to malfunction, generate heat, ignite or explode. Be aware that it is possible, in a worst case scenario, for temperatures to reach greater than those stated above in, for example, the trunk of a car on a hot day. So, storing the Tobii Dynavox I-Series device, in a hot car trunk could conceivably lead to a malfunction.

Only charge the Tobii Dynavox I-Series battery in an ambient temperature of 0 °C/32 °F to 45 °C/113 °F.

Use only the supplied power adapter to charge the Tobii Dynavox I-Series device. Using unauthorized power adapters may severely damage the Tobii Dynavox I-Series device.

Do not open, or modify, the casing of the Tobii Dynavox I-Series device or of the power supply, since you may be exposed to potentially hazardous electrical voltage. The device contains no serviceable parts. If the Tobii Dynavox I-Series device or it's accessories are mechanically damaged, **do not to use them**.

If the battery are not charged or the Tobii Dynavox I-Series is not connected to the power supply, the Tobii Dynavox I-Series device will shut down.

If the Power Supply Cord is damaged it needs to be replaced by Service Personnel only. Do not use the Power Supply Cord until replaced.

Only connect the Tobii Dynavox I-Series to the Power adapters shipped with this device.

The appliance coupler of the power supply is used as the Mains Disconnection Device, please do not position the Tobii Dynavox I-Series device so that it is difficult to operate the disconnection device.

Special regulations apply to shipping devices containing Li-ion batteries. If dropped, crushed, or short-circuited, these batteries can release dangerous amounts of heat and may ignite, and are dangerous in fires.

Please reference IATA regulations when shipping lithium metal or lithium ion batteries or cells: http://www.iata.org/whatwedo/cargo/dangerous_goods/Pages/lithium_batteries.aspx

The Power adapters shall not be used without an adult or Caregiver supervision.

1.4 Mounting

The Tobii Dynavox I-Series should be mounted according to MANUFACTURER instructions of approved mounts. Tobii Dynavox or its agents are not liable for damage or injuries to a person or its property due to a Tobii Dynavox I-Series falling from a mounted configuration. The mounting of a Tobii Dynavox I-Series is done entirely at the user's own risk.

1.5 Emergency

Do not rely on the Tobii Dynavox I-Series device for emergency calls or banking transactions. We recommend having multiple ways to communicate in emergency situations. Banking transactions should only be carried out with a system recommended by, and approved according to the standards of your bank.

1.6 Electricity

Do not open the casing of the Tobii Dynavox I-Series device, since you may be exposed to potentially hazardous electrical voltage. The device contains no user serviceable parts.

1.7 Child Safety

The Tobii Dynavox I-Series are advanced computer systems and electronic devices. As such they are composed of numerous separate, assembled parts. In the hands of a child certain of these parts, including accessories, have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.

Young children should not have access to, or the use of, the device without parental or guardian supervision.

1.8 Software

Windows 10 comes with Windows Defender included, providing protection from viruses, malware, and other potential threats. Regardless of whether you use Windows Defender or a third party anti-virus software, we recommend that you always keep your antivirus software up-to-date.

Software other than that which is pre-installed on the Tobii Dynavox I-Series is installed at the user's own risk. External software could cause the Tobii Dynavox I-Series to malfunction and might not be covered by the warranty.

1.9 Magnetic Field

The Tobii Dynavox I-Series device contains magnets. Magnetic fields may interfere with pacemakers, defibrillators, or other medical devices. As a general rule, maintain a minimum distance of 6 inches (15 centimeters) between any item with magnets and your heart device.

If you suspect that the Tobii Dynavox I-Series device is interfering with your pacemaker or any other medical device, stop using the Tobii Dynavox I-Series device and consult your physician for specific information about that affected medical device.

1.10 Third Party

Any use of the Tobii Dynavox I-Series outside the intended use and together with any third party software or hardware that changes the intended use is a risk and nothing that Tobii Dynavox can take any responsibility for.

Being a medical device, parts connected for a SIGNAL INPUT/OUTPUT must be compliant to the Medical IEC standard 60601-1

1.11 Environmental Control

The Programmable IR Remote Control — Environmental Control Unit (ECU) is not to be relied on as the only method of interaction with IR controlled devices.

2 Compliance Information

2.1 FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation.

Modifications not expressly approved by Tobii Dynavox could void the user's authority to operate the equipment under FCC rules.

2.1.1 For P15B Equipment

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

2.1.2 For Portable Devices

FCC RF Radiation Exposure Statement:

1. This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
2. This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. This device was tested for typical hand held operations with the device contacted directly to the human body to the sides of the device. To maintain compliance with FCC RF exposure compliance requirements, avoid direct contact to the transmitting antenna during transmitting.

2.2 Radio Wave Exposure and Specific Absorption Rate (SAR) Information

The Tobii Dynavox I-Series device has been tested as a Class 1 Medical Device. Medical electrical equipment such as this requires special caution regarding Electro-magnetic Compatibility (EMC) and thus needs to be installed and placed into service according to the information provided in this manual.

Using other cables and accessories other than those provided or that are integrated in the device may affect EMC performance.

The SAR (Specific Absorption Rate) limit as dictated by the FCC (in the USA) and by the IC (in Canada) is 1.6W/kg averaged over 1 gram of tissue. In Europe/EU (CE regulations) it is 2.0 W/kg averaged over 10 grams of tissue. The Devices, I-110 (FCC ID: W5M-TD1110), (IC ID 5534A-TD1110) have been tested against these SAR limits to maintain compliance with FCC/IC/CE RF exposure requirements.

This equipment complies with FCC/IC/EU RF radiation exposure limits set forth for an uncontrolled environment.

The highest SAR value for the I-110 device is 0.3 W/kg.

In order to maintain compliance as a Class 1 Medical device, the switches that are used in combination with the Tobii Dynavox I-Series device must fulfill the isolation requirements of IEC/EN 60601-1 based on 15 V DC. They must be electrically floating switches (switches that are not grounded).

2.3 Industry Canada Statement

Caution

- (i) The device for operation in the band 5150-5250 MHz is only for indoor use to reduce the potential for harmful interference to co-channel mobile satellite systems;
- (ii) The maximum antenna gain permitted for devices in the bands 5250-5350 MHz and 5470-5725 MHz shall be such that the equipment still complies with the e.i.r.p. limit;
- (iii) The maximum antenna gain permitted for devices in the band 5725-5850 MHz shall be such that the equipment still complies with the e.i.r.p. limits specified for point-to-point and non-point-to-point operation as appropriate; and
- (iv) Users should also be advised that high-power radars are allocated as primary users (i.e. priority users) of the bands 5250-5350 MHz and 5650-5850 MHz and that these radars could cause interference and/or damage to LE-LAN devices.

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This device (IC ID: 5534A-TDI110) has been tested to the safety requirements for exposure to radio waves (SAR) in RSS-102 to the General Public (Uncontrolled Environment) limits (1.6W/Kg). The highest SAR value for the I-110 device is 0.3 W/kg.. This device can be operated safely with a minimum distance 0 cm between the equipment and the user's body.

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This Class B digital apparatus complies with Canadian ICES-003.

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This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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2.4 CE Statement

This equipment complies with the requirements relating to electromagnetic compatibility, the essential protection requirement of Electromagnetic Compatibility (EMC) Directive 2014/30/EU on the approximation of the laws of the Member States relating to electromagnetic compatibility and Radio Equipment Directive (RED) 2014/53/EU to meet the regulation of the radio equipment and telecommunications terminal equipment.

2.5 Directives and Standards

The Tobii Dynavox I-Series complies with the following directives:

- Medical Device Regulation (MDR)
- Low voltage Directive 2014/35/EU
- Electromagnetic Compatibility (EMC) Directive 2014/30/EU
- Radio Equipment Directive (RED) 2014/53/EU
- RoHS2 Directive 2011/65/EU
- WEEE Directive 2012/19/EU
- Reach Directive 2006/121/EC, 1907/2006/EC Annex 17
- Batteries Directive 2013/56/EU

The device has been tested to comply with IEC/EN 60601-1 Ed 3.1, IEC/EN 60950-1, ISO 14971:2007 and EN ISO 14971:2012 and other relevant standards for the intended markets.

3 Customer Support

For support, please contact your local representative or Support at Tobii Dynavox. In order to receive assistance as quickly as possible, make sure you have access to your Tobii Dynavox I-Series device and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you will find on the Service tag label.

For further product information and other support resources, please visit www.tobiidynavox.com or www.myTobiiDynavox.com.

4 Disposing of the Tobii Dynavox I-Series Device

Do not dispose of the Tobii Dynavox I-Series Device in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.

5 Disposing of the Batteries

Do not dispose of the batteries in general household or office waste. Follow your local regulations for the disposal of batteries.

6 Maximum Allowed Temperature

Table 6.1 Maximum Allowed Hotspot Temperature

Allowable maximum temperature for skin contact of the Tobii Dynavox I-Series metal surfaces.	Maximum Temperature °C/°F
	Hotspot ⁱ surface under stand. Not likely to be touched
Surface may feel warm if continued skin contact.	47/117 (55/131 is limit for ITE ⁱⁱ)
Surface temperature may start to feel uncomfortable if continued skin contact.	55/131 (70/158 is limit for ITE ⁱⁱ)
Temperature DEVICE SHUT OFF — not to cause risk of harm.	60/140 (device max limit)
i. For Hotspot location, see <i>Figure 6.1 Hotspot Temperature Area</i>, page 5. ii. ITE refers to Information Technology Equipment, i.g. computers, tablets devices etc	

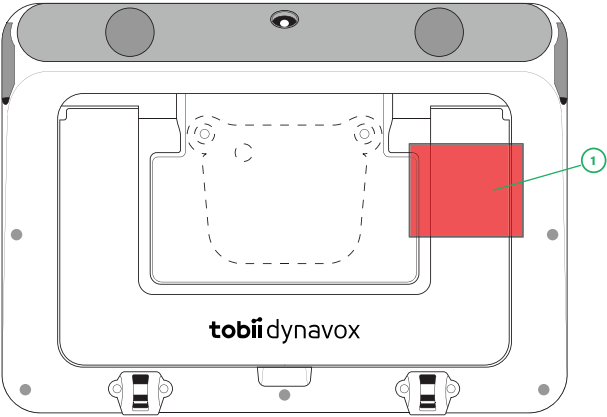
Figure 6.1 Hotspot Temperature Area

The Tobii Dynavox I-Series device may get warm and to avoid discomfort there are three (3) temperature levels the device will be monitoring by itself. If the temperature reaches the first level in *Table 6.1 Maximum Allowed Hotspot Temperature* a notification will be shown on the screen to inform the user. This is to avoid any risk of discomfort for the user due to the device being warm.

The second level notification will inform of the risk that the I-Series will shut down if it will become warmer. This temp level may also cause the user discomfort if holding the device for a longer time.

If reaching the third level, the I-Series device will shut down not to risk the temperature to cause any risk of harm to the user.

If the user tries to start the I-Series when the device temperature is above the third level, the device will not start up. The green LED will flicker when the power button is pressed, but the device will stay in the off condition. Let the I-Series cool down and try starting the device when the device temperature is below the third level.

7 Technical Specifications

7.1 Tobii Dynavox I-Series Device

Standard	Tobii Dynavox I-Series
Type/Model	I-110
Screen	10.1", Aspect ratio: 16:10, LED Backlight Unit
Screen Resolution	1920 × 1200
Touch Panel	Multi point Capacitive Touch Corning Gorilla Glass
Dimensions (WxHxD)	258 × 186 × 38 mm 10.2 × 7.3 × 1.5 inches
Weight	1.15 kg 2.54 lbs
Speakers	2 × 31 mm 4 ohm, 4 W
Microphone	1 × Digital MEMS Microphone
Processor	Intel Atom Z8350 – 4 cores, 2 MB cache, 1.92 Ghz

Standard	Tobii Dynavox I-Series
Type/Model	I-110
RAM	4 GB
Operating system	Microsoft® Windows 10 Pro 64-Bit
Flash drive	128 GB eMMC
Sensors	3-axis gyro, accelerometer, compass, ambient light sensor
Connectors	1 × 3.5 mm Headset/Audio-out port (stereo) with jack detection (Headphone and microphone port) 2 × Switch port, 3,5 mm 1 × USB 3.0
Buttons	1 × Power On 1 × Volume Up 1 × Volume Down
WLAN/Bluetooth®	IEEE 802.11 a/b/g/n Bluetooth® 4.0 / BLE
IR Remote Control (Environmental Control Unit)	Learning remote with integrated library of pre-programmed devices
Camera	Rear-facing: 8 Mega pixels Front-facing: 2 Mega pixels
IP class	IP 54 (with I/O covers in place)

7.2 Battery

Item	Specification
Manufacturer	Tobii Dynavox
Model Name	9789OTG101
Battery Technology	Li-ion rechargeable battery
Battery Capacity	46.62 Wh
Battery Run Time	> 8 hours, normal usage
Battery Charge Time	< 4 hours

7.3 Power Adapter

Item	Specification
Trademark	ADAPTER TECH.
Manufacturer	Adapter Technology Co., Ltd.
Model Name	ATM036T-A150
Rated Input	100–240V~50-60Hz, 1A–0.45A
Rated Output	15VDC, 2.4A
Output Plug	Barrel connector

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