tobiidynavox

Cradle for the I-13/I-16 Powerbank Installation Instructions

The Powerbank Cradle should be fixed to Ø22.25 mm (7/8") mount tubes. It is designed for compatibility with the mounting solutions from Rehadapt.

The power IN and OUT connectors to the Powerbank will pass through the holes in the Powerbank Cradle. Ensure easy access to this side of the cradle when mounting the Powerbank Cradle.



Insert the mounting tube into the fixing on the backside of the cradle.



Adjust the angle of the cradle to the required position.



Tighten both screws on the fixing to secure the cradle in position.



Support for Your Tobii Dynavox Device

Get Help Online
See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Insert the Powerbank into the cradle. Ensure the two holes align with the Power IN and OUT connectors.



Ensure the Powerbank is properly in place, and the cradle edges are securely surrounding the Powerbank.



Support for Your Tobii Dynavox Device

Get Help Online
See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com/support-training

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact