## 1 Safety

The Tobii Dynavox I-Series device has been tested and approved as compliant to all the Specifications and Standards listed in 2 *Compliance Information, page 2* of this manual and in the 7 *Technical Specifications, page 4* -- including, but not limited to, the Medical Device Standard (Class 1/Type B). Nevertheless, in order to ensure safe operation of your Tobii Dynavox I-Series device, there are a few safety warnings to bear in mind:

**No modification of this equipment is allowed**

Repairs to a Tobii Dynavox device must only be performed by Tobii Dynavox or a Tobii Dynavox authorized and approved repair center.

**Contraindication:** The Tobii Dynavox I-Series device should never be, for the user, the only means of communicating important information.

In case of failure of the Tobii Dynavox I-Series device, the user cannot communicate using it.

In case of serious incident that has occurred, please contact the manufacturer and local authorities immediately.

Notice: It is recommended to install the appropriate software, if have any question, please contact the manufacturer for further assistance.

Notice: To prevent unauthorized access, it is recommended to install suitable anti-virus software or do not connect to unsafe external networks.

The Tobii Dynavox I-Series is not to be used as a life supporting device, and it shall not be relied on in case of loss of function due to power loss or other causes.

There could be a choking hazard risk if small parts detach from the Tobii Dynavox I-Series device.

The Tobii Dynavox I-Series device shall not be exposed to or used in weather conditions outside the Technical Specification of the Tobii Dynavox I-Series device.

The Tobii Dynavox I-Series device shall only be used with Tobii Dynavox I-Series specific accessories that have a mounting instruction included with the accessory.

The charging cable could present strangulation hazards to young children. Never leave small children unattended with the charging cable.

Young children or people with cognitive disabilities should not have access to, or the use of, the Tobii Dynavox I-Series device without parental or guardian supervision.

Do not attach any decorations, post it notes or similar on the screen side of the Tobii Dynavox I-Series device. These may interfere with the Eye Gaze or touch screen performance.

Never force a connector into a port. If the connector and port do not join with reasonable ease, they probably do not match. Make sure that the connector matches the port and that you have positioned the connector correctly in relation to the port.

Be extra careful with the USB connectors.

### 1.1 Avoiding Hearing Damage

Permanent hearing loss may occur if earphones, headphones or speakers are used at high volume. To prevent this, the volume should be set to a safe level. You can become desensitized over time to high sound levels which may then sound acceptable yet still could be damaging your hearing. If you experience symptoms such as ringing in your ears, please lower the volume or stop using the earphones/headphones. The louder the volume, the less time is required before your hearing could be affected.

Hearing experts suggest the following measures to protect your hearing:

- Limit the amount of time you use earphones or headphones at high volume.
- Avoid turning up the volume to block out noisy surroundings.
- Turn the volume down if you cannot hear people speaking near you.

To establish a safe volume level:

- Set your volume control at a low setting.
- Slowly increase the sound until you can hear it comfortably and clearly, without distortion.

The Tobii Dynavox I-Series device can produce sounds in decibel ranges that may cause hearing loss for a normal hearing person, even when exposed for less than a minute. The maximum sound level of the unit is in parity with the sound levels that a healthy young person can produce while screaming. Since the Tobii Dynavox I-Series device is intended as a voice prosthetic, it shares the same possibilities and potential risks of causing harm to hearing. The higher decibel ranges are offered to enable communication in a noisy environment and should be used with care and only in said noisy environments.

### 1.2 Temperature

When used in direct sunshine or in any other hot environment, the Tobii Dynavox I-Series device may have hot surfaces. There are built-in temperature sensors that monitors the temperature. If the sensors detects temperatures above the temperature specified in 6 *Maximum Allowed Temperature, page 4*, the device will automatically trigger a Windows Shut Down or Hibernate (depending on Windows Power Button configuration) to prevent risk of harm to the user. It may take a while before the Tobii Dynavox I-Series device can be restarted, as the unit may need time to cool down.

Do not touch the area specified in *Illustration 1: Temperature Threshold Area, page 4* when the Tobii Dynavox I-Series device is running. The specified area can reach temperatures which may cause permanent bodily harm in the event of prolonged direct skin contact. Use appropriate precautions to avoid injury.

### 1.3 Power Supply and Batteries

The Tobii Dynavox I-Series device uses lithium-ion batteries. These batteries have a storage temperature range of between -20 °C/-4 °F and 40 °C/104 °F within 3 months.

If you are in a hot environment, be aware that it can affect the ability to charge the batteries. The internal temperature has to be between 0 °C/32 °F and 45 °C/113 °F for the batteries to charge. If the internal battery temperature raises above 45 °C/113 °F the batteries will not charge at all.

Move the Tobii Dynavox I-Series device and the batteries to a cooler environment to let the batteries charge properly.

Avoid exposing the batteries to fire or to temperatures above 50 °C/122 °F. These conditions may cause a battery to malfunction, generate heat, ignite or explode. Be aware that it is possible, in a worst case scenario, for temperatures to reach greater than those stated above in, for example, the trunk of a car on a hot day. So, storing the device, with batteries installed, in a hot car trunk could conceivably lead to a malfunction.

Do not disassemble or harm the battery. Follow the environmental laws and regulations which apply in your area when disposing of batteries.

The battery may only be replaced by the user with a TDG G2 battery pack sold by Tobii Dynavox. There is a risk of explosion if the battery is replaced with an incorrect type.

-  For safe operation of the Tobii Dynavox I-Series device, use only chargers, batteries, and accessories approved by Tobii Dynavox.
-  Do not open (except for the battery cover), or modify, the metal casing of the Tobii Dynavox I-Series device or of the power supply, since you may be exposed to potentially hazardous electrical voltage. The device contains no serviceable parts. If the Tobii Dynavox I-Series device or its accessories are mechanically damaged, **do not use them**.
-  If the battery is not charged or the Tobii Dynavox I-Series is not powered up by the power supply, the Tobii Dynavox I-Series device will shut down.
-  If the Power Supply Cord becomes damaged, contact Tobii Dynavox for replacement.
-  Do not connect any devices with a non-medical grade power supply to any connector on the Tobii Dynavox I-Series device. Furthermore, all configurations shall comply with the system standard IEC 60601-1-1. Anyone who connects additional equipment to the signal input part or signal output part is configuring a medical system and is therefore responsible for ensuring that the system complies with the requirements of the system standard IEC 60601-1-1. If in doubt, consult the technical services department or your local representative.
-  The appliance coupler of the power supply or separable plug is used as the Mains Disconnection Device, please do not position the Tobii Dynavox I-Series device so that it is difficult to operate the disconnection device.
-  Special regulations apply to shipping Lithium-ion batteries. If dropped, crushed, punctured, thrown, abused or short-circuited, these batteries can release dangerous amounts of heat and may ignite, and are dangerous in fires.  
Please reference IATA regulations when shipping lithium metal or lithium-ion batteries or cells: <http://www.iata.org/whatwedo/cargo/dgr/Pages/lithium-batteries.aspx>

#### 1.4 Mounting

-  The Tobii Dynavox I-Series should be mounted according to manufacturer instructions of approved mounts. Tobii Dynavox or its agents are not liable for damage or injuries to a person or its property due to a Tobii Dynavox I-Series falling from a mounted configuration. The mounting of a Tobii Dynavox I-Series is done entirely at the user's own risk.

#### 1.5 Emergency

-  Do not rely on the device for emergency calls or banking transactions. We recommend having multiple ways to communicate in emergency situations. Banking transactions should only be carried out with a system recommended by, and approved according to the standards of your bank.

#### 1.6 Infrared

-  The Tobii Dynavox I-Series emits pulsed infrared (IR) light both from the eye tracker and from the Environmental Control Unit. Other devices controlled by IR or susceptible to disturbance by IR light might be affected by the IR light emitted from the Tobii Dynavox I-Series. Do not use the Tobii Dynavox I-Series in the vicinity of such devices if their functionality is of critical importance.

#### 1.7 Epilepsy

-  Some people with **Photosensitive Epilepsy** are susceptible to epileptic seizures or loss of consciousness when exposed to certain flashing lights or light patterns in everyday life. This may happen even if the person has no medical history of epilepsy or has never had any epileptic seizures.  
A person with Photosensitive Epilepsy would also be likely to have problems with TV screens, some arcade games, and flickering fluorescent bulbs. Such people may have a seizure while watching certain images or patterns on a monitor, or even when exposed to the light sources of an eye tracker. It is estimated that about 3-5% of people with epilepsy have this type of Photosensitive Epilepsy. Many people with Photosensitive Epilepsy experience an "aura" or feel odd sensations before the seizure occurs. If you feel odd during use, move your eyes away from the eye tracker.

#### 1.8 Electricity

-  Do not open the casing, except for the battery cover, of the Tobii Dynavox I-Series device, since you may be exposed to potentially hazardous electrical voltage. The device contains no user serviceable parts.

#### 1.9 Software

-  Windows comes with Windows Defender included, providing protection from viruses, malware, and other potential threats. Regardless of whether you use Windows Defender or a third party anti-virus software, we recommend that you always keep your antivirus software up-to-date.
-  Software other than that which is pre-installed on the Tobii Dynavox I-Series is installed at the user's own risk. External software could cause the Tobii Dynavox I-Series to malfunction and might not be covered by the warranty.

#### 1.10 Child Safety

-  The Tobii Dynavox I-Series are advanced computer systems and electronic devices. As such they are composed of numerous separate, assembled parts. In the hands of a child these parts have the possibility of being separated from the device, possibly constituting a choking hazard or another danger to the child.  
Young children should not have access to, or the use of, the device without parental or guardian supervision.

#### 1.11 Environmental Control

-  The Programmable IR Remote Control — Environmental Control Unit (ECU) is not to be relied on as the only method of interaction with IR controlled devices.

#### 1.12 Eye Gaze

-  Some people may experience a certain amount of fatigue (due to intentional eye focusing and hard concentration) or even a dryness of the eyes (due to less frequent blinking) when first getting used to Eye Gaze. If you are experiencing fatigue or dry eyes start off slowly and limit the length of your Eye Gaze sessions to your comfort level. Re moisturizing eye drops can be helpful to combat dryness.

#### 1.13 Third Party

-  Tobii Dynavox assumes no responsibility for any consequence resulting from use of the Tobii Dynavox I-Series in a manner inconsistent with its intended use, including any use of the Tobii Dynavox I-Series with third-party software and/or hardware that changes the intended use.

## 2 Compliance Information

The Tobii Dynavox I-Series is CE-marked, indicating compliance with the essential health and safety requirements set out in European Directives.

### 2.1 FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation.

-  Modifications not expressly approved by Tobii Dynavox could void the user's authority to operate the equipment under FCC rules.

#### 2.1.1 For P15B Equipment

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### 2.1.2 For Portable Devices

FCC RF Radiation Exposure Statement:

1. This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.
2. This equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment. This device was tested for typical hand held operations with the device contacted directly to the human body to the sides of the device. To maintain compliance with FCC RF exposure compliance requirements, avoid direct contact to the transmitting antenna during transmitting.

### 2.2 Radio Wave Exposure and Specific Absorption Rate (SAR) Information

The Tobii Dynavox I-Series device has been tested as a Class 1 Medical Device. Medical electrical equipment such as this requires special caution regarding Electro-magnetic Compatibility (EMC) and thus needs to be installed and placed into service according to the information provided in this manual.

Using other cables and accessories other than those provided or that are integrated in the device may affect EMC performance.

The SAR (Specific Absorption Rate) limit as dictated by the FCC (in the USA) and by the IC (in Canada) is 1.6W/kg averaged over 1 gram of tissue. In Europe/EU (CE regulations) it is 2.0 W/kg averaged over 10 grams of tissue. The Devices, TD I-13 (FCC ID: 2AA0V-TD113), (IC ID 5534A-TD113) and TD I-16 (FCC ID: 2AA0V-TD116), (IC ID 5534A-TD116) have been tested against these SAR limits to maintain compliance with FCC/IC/CE RF exposure requirements.

This equipment complies with FCC/IC/EU RF radiation exposure limits set forth for an uncontrolled environment.

The highest SAR value for the TD I-13 device is 1.168 W/kg and for the TD I-16 device is 0.656 W/kg.

In order to maintain compliance as a Class 1 Medical device, the switches that are used in combination with the Tobii Dynavox I-Series device must fulfill the isolation requirements of IEC/EN 60601-1 based on 15 V DC. They must be electrically floating switches (switches that are not grounded).

### 2.3 Industry Canada Statement

#### Caution

- (i) The device for operation in the band 5150-5250 MHz is only for indoor use to reduce the potential for harmful interference to co-channel mobile satellite systems;
- (ii) The maximum antenna gain permitted for devices in the bands 5250-5350 MHz and 5470-5725 MHz shall be such that the equipment still complies with the e.i.r.p. limit;
- (iii) The maximum antenna gain permitted for devices in the band 5725-5850 MHz shall be such that the equipment still complies with the e.i.r.p. limits specified for point-to-point and non-point-to-point operation as appropriate; and
- (iv) Users should also be advised that high-power radars are allocated as primary users (i.e. priority users) of the bands 5250-5350 MHz and 5650-5850 MHz and that these radars could cause interference and/or damage to LE-LAN devices.

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This device (IC ID: 5534A-TD13L) has been tested to the safety requirements for exposure to radio waves (SAR) in RSS-102 to the General Public (Uncontrolled Environment) limits (1.6W/Kg). The highest SAR value for the TD I-13 device is 1.168 W/kg. This device can be operated safely with a minimum distance 0 cm between the equipment and the user's body.

This device (IC ID: 5534A-TD16L) has been tested to the safety requirements for exposure to radio waves (SAR) in RSS-102 to the General Public (Uncontrolled Environment) limits (1.6W/Kg). The highest SAR value for the TD I-16 device is 0.656 W/kg. This device can be operated safely with a minimum distance 0 cm between the equipment and the user's body.

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This Class B digital apparatus complies with Canadian ICES-003.

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This device complies with Industry Canada license-exempt RSS standard(s). Operation is subject to the following two conditions: (1) this device may not cause interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

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### 2.4 CE Statement

This equipment complies with the requirements relating to electromagnetic compatibility, the essential protection requirement of Electromagnetic Compatibility (EMC) Directive 2014/30/EU on the approximation of the laws of the Member States relating to electromagnetic compatibility and Radio Equipment Directive (RED) 2014/53/EU to meet the regulation of the radio equipment and telecommunications terminal equipment.

### 2.5 Directives and Standards

The Tobii Dynavox I-Series complies with the following directives or standards:

- Medical Device Regulation (EU) 2017/745
- Medical Device Directive 93/42/EEC
- Low voltage Directive 2014/35/EU
- Electromagnetic Compatibility (EMC) Directive 2014/30/EU
- Radio Equipment Directive (RED) 2014/53/EU
- RoHS2 Directive 2011/65/EU
- WEEE Directive 2012/19/EU
- Reach Directive 2006/121/EC, 1907/2006/EC Annex 17
- ISO 9001:2015
- ISO 14971:2019
- ISO 13485:2016

The device has been tested to comply with IEC/EN 60601-1 Ed 3.1, IEC 62368 CB Scheme, ISO 14971:2019 and other relevant standards for the intended markets.

## 3 Customer Support

For support, please contact your local representative or Support at Tobii Dynavox. In order to receive assistance as quickly as possible, make sure you have access to your Tobii Dynavox I-Series device and, if possible, an Internet connection. You should also be able to supply the serial number of the device, which you find on the back of the device.

For further product information and other support resources, please visit the Tobii Dynavox website [www.tobiidynavox.com](http://www.tobiidynavox.com).

## 4 Disposing of the Device

Do not dispose of the Tobii Dynavox I-Series device in general household or office waste. Follow your local regulations for the disposal of electrical and electronic equipment.


## 5 Disposing of Batteries

Do not dispose of the batteries in general household or office waste. Follow your local regulations for the disposal of batteries.

## 6 Maximum Allowed Temperature

Table 1: Maximum Allowed Temperature

Allowable maximum temperature for the Tobii Dynavox I-Series metal external surfaces which are likely to be touched	Maximum Temperature °C/°F
Temperature threshold: External surfaces temperature allowed	55/131
External surfaces temperature allowed in an area unlikely to be touched, see position 1 in <i>Illustration 1: Temperature Threshold Area, page 4</i> for the position of these areas.	60/140

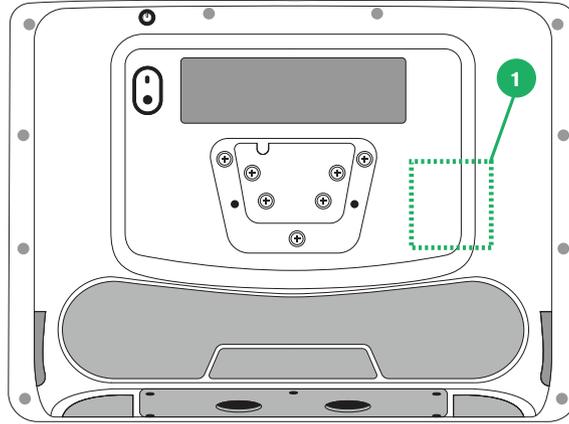**Illustration 1: Temperature Threshold Area**

Do not touch the area specified in *Illustration 1: Temperature Threshold Area, page 4* when the Tobii Dynavox I-Series device is running. The specified area can reach temperatures which may cause permanent bodily harm in the event of prolonged direct skin contact. Use appropriate precautions to avoid injury.

## 7 Technical Specifications

### 7.1 Standard TD I-Series

Type/Model	TD I-13	TD I-16
Screen	13.3", LED Backlight Unit, Brightness: 300 Nits	15.6", LED Backlight Unit Brightness: 350 Nits
Screen Resolution	1920 × 1080 pixels (FHD)	
Touch Panel	Projected Capacitive Touch with Gorilla® Glass, 10 point multi touch	
Rear Display	480 × 128 pixels	
Dimensions (WxHxD)	34,3 × 24,1 × 8,1 cm 13.5 × 9.5 × 3.2 inches	39,1 × 26,9 × 8,1 cm 15.4 × 10.6 × 3.2 inches
Weight	2,4 kg 5.3 lbs	2,7 kg 6.0 lbs
Speakers	2 × 10 W Closed Box Speakers	
Microphone	1 × Analog Microphone	
Processor	Intel Core i5-7300U	
RAM	16 GB LPDDR4	
Operating system	Windows 10	
Hard disk drive	256 GB, SATA based SSD	
Connectors	2 × USB 3.0 (Current limit 1500 mA) 2 × 3.5 mm Switch connector interface, (Pin out for mono plug: Sleeve = Common ground, Tip = Signal) 1 × 3.5 mm* Headphone jack (stereo) with jack detection 1 × 19 VDC in 2.5/5,5 mm (Power connector)	
Buttons	1 × Power On 1 × Volume Up 1 × Volume Down 2 × Programmable	
WLAN (optional)	IEEE 802.11 a/b/g/n/ac	

Type/Model	TD I-13	TD I-16
Bluetooth (optional)	Bluetooth 4.1 Support all standard Windows 10 Bluetooth profiles	
IR Remote Control (Environmental Control Unit)	Vishay TSMP77000 Upright Orientation: 1 × IR filter window with 6 × IR Transmit Diodes	
Camera	User Camera (back) 8 MP AF (Auto Focus), with flash Web Camera (front) 2 MP FF (Fixed Focus)	
Eye Tracker	Tobii IS5TDL Module	
Expected Service Life	5 years	
Battery Run Time <sup>1</sup>	Up to 8 h	
Typical Average Battery Run Time	~6.5 h	
Battery Charge Time	Maximum 5.4 h (0-100%)	
Deskstand	Integrated	
Mounting Systems Supported	Tobii Dynavox QR adapter plate for Daessy and REHAdapt	
Power Supply	65 W AC adapter	
IP class	IP54 Pass for device only, with I/O covers in place. Adaptor mode is excluded.	
	IP22 In adaptor mode.	
1. Indoor with continuous communication and with Eye Gaze if applicable. ~24 hours with a Power optimized scenario.		

## 7.2 Device Power Adapter

Item	Specification
Manufacturer	FSP Group Inc.
Type	Switching Power Adapter
Model	FSP065-DBCM1
Input voltage	100 to 240 VAC
Input current (max)	2.0 – 1.0 A
Input frequency	50 to 60 Hz
Output current	3.43 A
Rated output voltage	19.0 VDC

## 7.3 Battery pack

Item	Specification	Remark
Battery Technology	Li-Ion rechargeable battery pack with gas gauge (SMBus v1.1 interface)	
Cell	8× Panasonic NCR18650GA	
Battery Pack Capacity	95 Wh	Initial capacity, new battery pack
Nominal Voltage	14.4Vdc, 6600mAh, 95.04Wh	
Charge Time	Maximum 5.4 h	Charge from 0 to 100%
Cycle Life	500 cycles	Minimum 75% of initial capacity remaining
Storage Time <sup>1</sup>	Maximum 6 months @ charge ≥ 40%	 Do not store battery packs for a long time with less than 40% charge level.
1. It is recommended that the battery shall not be stored in the device if the device is not going to be used within 6 months. If the battery is removed, the battery will not be drained as quickly as when stored in the device.		

## 7.4 Eye Tracker

If installed

Technical Specifications	Tobii IS5TDL Module
<b>Working Distance</b>	45 — 95 cm 20 — 37 inches
<b>Freedom of Head Movement</b> <sup>1</sup> (Width x Height)	~20 x 20cm (7.9 x 7.9 inches) @ 50 cm from screen ~35 x 35 cm (13.8 x 13.8 inches) between 65 – 80 cm from screen
<b>Positioning</b> Distance (from screen) Track box size (Width x Height) Track box depth	45 — 95 cm (20 — 37 in) 20 x 20 — 35 x 35 cm (7.9 x 7.9 — 13.8 x 13.8 in) 50 cm (19.7 in)
<b>Gaze Data Rate</b>	33 Hz
<b>Gaze Sample Rate</b>	133 Hz
<b>Eye tracking technique</b>	Video-based pupil- and corneal- reflection eye tracking with dark and bright pupil illumination modes.
<b>Outdoor Usage Possible</b>	Yes
<b>User Calibration</b> (former tracking robustness)	>98%
<b>Detect Gaze</b> Interaction >30Hz	98% for 95% of population <sup>2</sup>
<b>Gaze Accuracy</b> Across 95% of population <sup>3</sup>	<1.58 degrees
<b>Gaze Precision</b> Across 95% of population <sup>3</sup>	<0.2°
<b>Max Head Movement Speed</b> Eye Position Gaze Data	40 cm/s (15.7 in/s) 10 cm/s (3.9 in/s)
<b>Max Head Tilt</b>	25°
<b>Max Yaw, pitch</b>	25°
<b>Data Flow and Data Rate</b> Gaze Latency Gaze Recovery	17 ms 0 ms
<b>Mounting</b>	Built-in
<b>Power Supply</b>	Built-in

- Freedom of head movement describes the volume in front of the tracker in which the user must have at least one of the eyes. The numbers are specified parallel/orthogonal to screen surface.
- The test population excluded those who would wear correctional glasses with a +5.00 diopter or higher or suffer from an eye disease.
- The degree-of-accuracy and precision over percentage-of-population numbers result from extensive testing across representatives of the whole population. We have made use of hundreds of thousands of diagnostic images and performed tests on approximately 800 individuals with different conditions, vision, ethnicities, everyday dust, smudges or blemishes around their eyes, with eyes out of focus, etc. This has resulted in a much more robust and high-performance eye tracking experience and a much more realistic representation of true performance across the whole population, not just in a mathematically "ideal" scenario.  
The "ideal" degree numbers are the previous standard of measuring accuracy and precision, both previously from Tobii and presently from all eye tracking competitors. Though "ideal" numbers are useful to get a general feel about comparative quality and performance, they are not applicable to real world usage in the same way as quantitative degree-of-accuracy and precision over percentage-of-population numbers based on extensive testing across representatives of the whole population.

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## 7.4 Augnstýring

If installed

Technical Specifications	Tobii IS5TDL Module
Working Distance	45 — 95 cm 20 — 37 inches
Freedom of Head Movement <sup>1</sup> (Width x Height)	~20 × 20cm (7.9 × 7.9 inches) @ 50 cm from screen ~35 × 35 cm (13.8 × 13.8 inches) between 65 – 80 cm from screen

Technical Specifications	Tobii IS5TDL Module
<b>Positioning</b> Distance (from screen) Track box size (Width × Height) Track box depth	45 — 95 cm (20 — 37 in) 20 × 20 — 35 × 35 cm (7.9 × 7.9 — 13.8 × 13.8 in) 50 cm (19.7 in)
<b>Gaze Data Rate</b>	33 Hz
<b>Gaze Sample Rate</b>	133 Hz
<b>Eye tracking technique</b>	Video-based pupil- and corneal- reflection eye tracking with dark and bright pupil illumination modes.
<b>Outdoor Usage Possible</b>	Yes
<b>User Calibration</b> (former tracking robustness)	>98%
<b>Detect Gaze</b> Interaction >30Hz	98% for 95% of population <sup>2</sup>
<b>Gaze Accuracy</b> Across 95% of population <sup>3</sup>	<1.58 degrees
<b>Gaze Precision</b> Across 95% of population <sup>3</sup>	<0.2°
<b>Max Head Movement Speed</b> Eye Position Gaze Data	40 cm/s (15.7 in/s) 10 cm/s (3.9 in/s)
<b>Max Head Tilt</b>	25°
<b>Max Yaw, pitch</b>	25°
<b>Data Flow and Data Rate</b> Gaze Latency Gaze Recovery	17 ms 0 ms
<b>Mounting</b>	Built-in
<b>Power Supply</b>	Built-in

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