

## Tobii Dynavox Snap Scene

### Quick Start Guide

#### Pathways

Download the Tobii Dynavox Pathways app to your iPad to view videos and read articles that will teach you the research-proven strategies for using Snap Scene with your child. Pathways will help you to use the powerful methods that can result in significant gains in language and communication skills. You can also access all of the instructional videos available in Pathways at [myTobii-Dynavox.com](http://myTobii-Dynavox.com)

#### Play Scenes

You and your child can choose scenes from the scene list and then interact with those scenes. When an interactive area in a scene, called a hotspot, is selected, an audio recording will play and you may see the text label for that hotspot. Audio and text label interaction can be customized.



Your scene interaction response is determined by the settings currently enabled as well as the content (audio recording and text label) associated with each hotspot. Show Hotspot Labels is set to Animate by default.

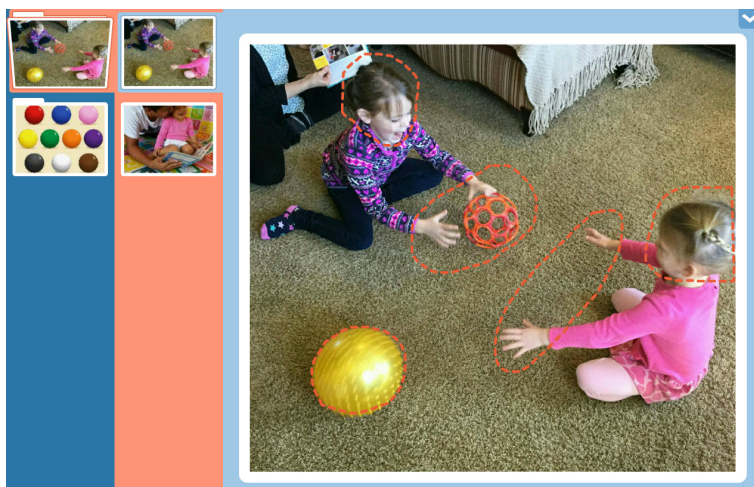


Figure 1 Playing a scene

To interact with a different scene, use the navigation bar on the left side of the screen. Select a category in the blue column, then choose a scene in the pink column. Swipe up and down to scroll in the columns, if necessary.

Use the hardware and operating system audio controls on your device to adjust the volume.

#### Create a New Scene

1. Select **Show Tools**.



#### Support for Your Tobii Dynavox Device

##### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

##### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

2. Select the category folder in which you would like to create your new scene.

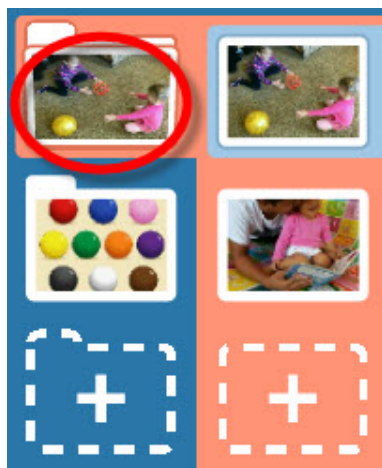


Figure 2 Select Category

3. Select the new scene button (+). Photo mode will open.



Figure 3 New Scene Button

4. Add a photo to your scene by taking a new photo or select an image file stored locally on your device:

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- a. **Take Photo** — Take a photo with the currently selected camera. The viewfinder on the screen shows the image that will be captured when you select this option.

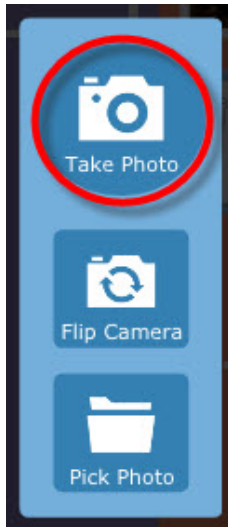


Figure 4 Photo Mode — Take Photo

- b. **Flip Camera** — If your device has both front and back facing cameras, you may use the **Flip Camera** button to select the other camera on your device. When you see the desired view in the viewfinder on your screen, select the **Take Photo** button to capture the image.

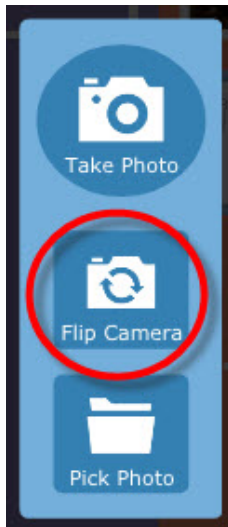


Figure 5 Photo Mode — Flip Camera

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

- c. **Pick Photo** — To choose a photo file stored locally on your device, select the **Pick Photo** button. Browse the folders using the up arrow button or File... button. Select the desired photo file, then select the green **Accept** button.

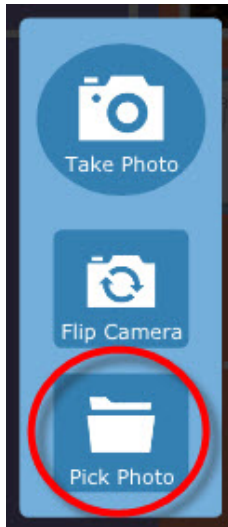


Figure 6 Photo Mode — Pick Photo

- d. If you would like to quit Photo Mode without creating a scene, select **Exit Photo Mode**.



## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

5. Use the slider at the bottom of the photo to scale the image to the desired size. Drag the slider knob toward the **+** to zoom in and toward the **—** to zoom out. On iPad, you can pinch the image to zoom.

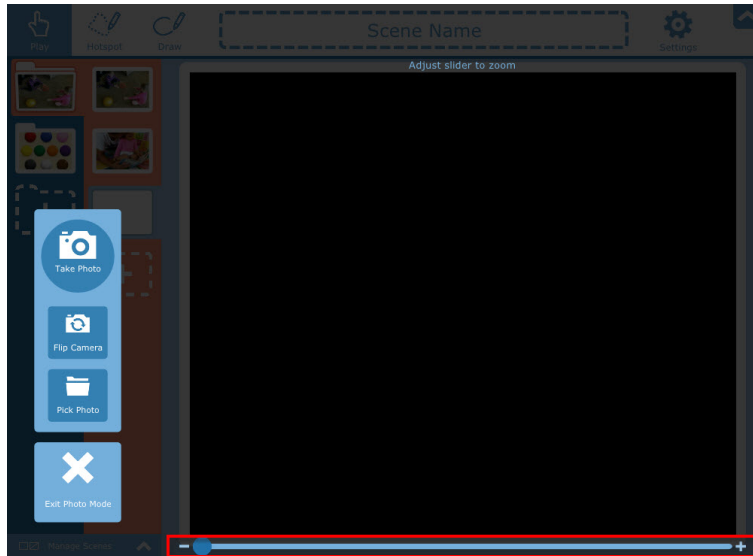


Figure 7 Photo Mode — Scale Tool

6. Drag the photo image to center it.
7. When you have finished scaling and centering your photo, select **Accept** to save. To retake the photo or choose a different image file, select **Retake** or **Back**. To quit the photo screen without saving, select **Exit Photo Mode**.

## Edit a Scene


1. Select **Show Tools**. 
2. Select the scene that you would like to edit.
3. (Optional) Add or Edit the Scene Name by selecting, then typing in the text field at the top of the screen.



Figure 8 Toolbar

4. Use the tools in the toolbar to make changes to your scene.

## Editing Tools

Edit your scene using the tools in the toolbar: Play, Hotspot, and Draw.

### .1 Play Hotspot

Use the **Play tool** to see how a Hotspot will respond when selected.



## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)

## .2 Create and Edit a Hotspot

Create a new hotspot by using the **Hotspot tool** to outline an object or area in the scene. A hotspot is a selectable area of a scene that, when selected, can play an audio recording and display a text label.



When you have created the hotspot (or selected an existing hotspot), you will have the option to give the hotspot a label and make an audio recording.


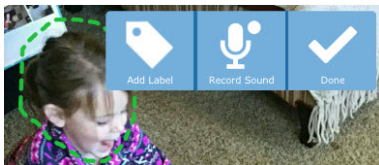


Figure 9 Hotspot Options

### .2.1 Add a Hotspot Label

1. Select **Add Label**.
2. Select in the text field, then type a label.

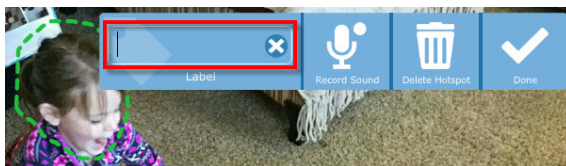

### .2.2 Hotspot Sound Recording

1. Select **Record Sound**. Recording will begin immediately (indicated by a flashing red circle).

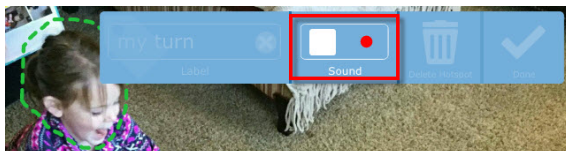


Figure 10 Record Sound

2. When you have finished recording, select the square button to stop recording.
3. To hear your recorded sound, press the triangle (play) button.

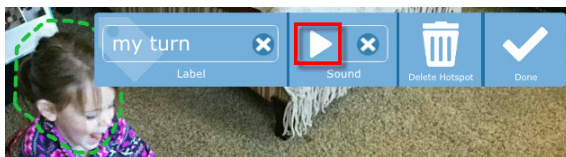


Figure 11 Preview Sound

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)



**i** If you would like to make a new recording for a hotspot that already has a sound, select the **delete** icon beside the Play button to remove the existing recording, then select **Record Sound** to create a new recording.


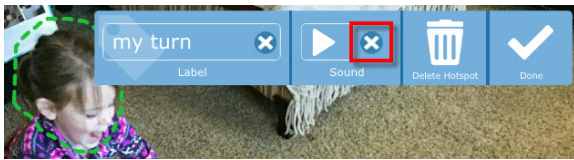


Figure 12 Delete Sound

## .2.3 Delete a Hotspot

1. Use the **Hotspot tool** to select a hotspot.
2. Select **Delete Hotspot**.

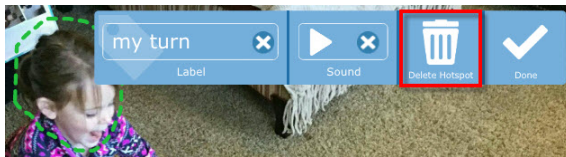

## .3 Draw

Use the **Draw tool** to create freehand lines anywhere in the scene.



Figure 13 Draw Tool

- You may select a **line color** from the six color swatches.
- The **Erase tool** is used to remove parts of your drawings in a controlled fashion.
- The **Clear** button will remove all drawings from your scene.



Figure 14 Scene with Drawings

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: [www.TobiiDynavox.com](http://www.TobiiDynavox.com) or [www.myTobiiDynavox.com](http://www.myTobiiDynavox.com).

### Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit [www.TobiiDynavox.com/contact](http://www.TobiiDynavox.com/contact)