

Tobii Dynavox T15 Quick Start Guide

Welcome to the Tobii Dynavox T15 Quick Start Guide! This document will introduce you to your new communication solution — where to find everything and what to expect when you turn on your T15 for the first time. Have fun exploring the T15 and congratulations on choosing augmentative communication without compromise!

Hardware Overview

1. **Power Button:** To turn the device on, press and hold until the splash screen appears. Press and release to put to sleep. To shut down, press and hold until the shutdown menu appears.
2. **Front-facing Camera:** Takes photographs of objects in front of the screen (e.g., self portraits).
3. **Ambient Light Sensor:** Adjusts the screen brightness based on the surrounding light.
4. **Charging Indicator:** Lit amber when the battery is charging. Turns blue when the battery is fully charged.
5. **Speakers:** Dual speakers provide an extensive range of volume.
6. **Volume Rocker Switch:** Press on the + side of the switch to increase volume. Press on the - side of the switch to decrease volume.
7. **Micro SDHC Card:** Create local backups or provide additional storage for pictures and other media.
8. **Full-Size USB Port:** Easily connect peripherals (mouse, keyboard, head tracker) or access a standard USB drive.
9. **Speaker/Headphone/External Microphone Jack:** Connect an external speaker, headphones for private audio, or an external microphone.
10. **Switch Jacks (2):** Connect external switch(es) for scanning access.
11. **Rear-facing Camera:** Use for taking custom photos.
12. **Rear-Facing Infrared Window:** Sends infrared signals.
13. **Lanyard connections:** Provide attachment points for the T15 strap.
14. **30-Pin Dock Connector:** (for future use)
15. **Charger Port:** Plug in the adapter/charger to charge the battery.
16. **Micro USB Port:** Connect a micro-USB to full-size USB cable adaptor.
17. **Micro HDMI Port:** Connect an external monitor to your device.
18. **Built-in Microphone:** Record speech and custom sounds.



Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact

Battery Charging

Plug the charger supplied with the T15 into a wall outlet. Connect the charger to the Charging Port (16) on the device. The Charging Indicator (4) light will illuminate amber when the device is charging and will turn blue when the device is fully charged.

After the device is fully charged, disconnect the charger and the light will automatically turn off.


Initial Startup

New to the technology and not sure how to get started? We removed the guesswork. When you turn on your device for the first time, just find and select the Compass desktop icon. We'll take it from there.



Overview of Setup Wizard

Use the **Setup Wizard** to make some quick customizations to your device. Just select **Start Setup Wizard** and choose **Go to New Pageset Wizard** (for new users) or **Select a Pageset** (for more experienced users). You can also use your own Compass backup file by restoring it from either myTobiiDynavox or your own computer.

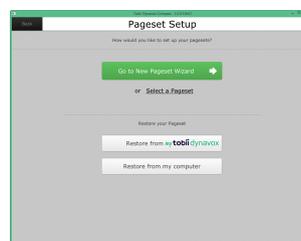

Resources

After completing the Setup Wizard, you'll be ready to go. Still have questions? Our resources will provide the answers.

myTobiiDynavox

Log on to myTobiiDynavox to access implementation support and tools, find technical support resources, and upload, download, and share communication pages! To get started, visit www.myTobiiDynavox.com, click **Sign Up** and register your new Device!

Knowledgebase & Help

Access the Knowledgebase or watch help videos anytime, 24/7. Just select the **Settings** button and choose **Help** from the Settings menu.

Technical Support: Go to www.mytobiidynavox.com > **Community & Support** > **mytobiidynavox Community**

Visit the online library to access Knowledgebase articles, research product tips, and perform custom searches, or visit the **Community Q&A** section. Or - if you have a problem with your Device, start the repair process by creating a new case.

Before you contact Support, be sure to have your Device serial number. It's located on the bottom edge of the Device and in **Settings > Help > About**. My Device serial number is: _____.



Other important information about the Device, including its intended use and copyright information, can be found in the T-Series User's Manual. Get additional information by going to **Settings > Help**.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product. Find our Support pages online at: www.TobiiDynavox.com or www.myTobiiDynavox.com.

Contact Your Sales Representative or Reseller

For questions or problems with your product, contact your Tobii Dynavox sales representative or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit www.TobiiDynavox.com/contact