## Informacije o opozivu TD Navio

### Priprema

Ovaj dokument opisuje kako se pripremiti za popravke nakon opoziva vašeg TD Navio Midi ili TD Navio Maxi uređaja. Slijedite upute za izradu sigurnosne kopije podataka na vašem uređaju i komunikacijskoj aplikaciji, zaštitite svoje osobne podatke i pripremite alternativne opcije komunikacije.

Kada ste dovršili korake u odjeljku Priprema, prijedite na *Pokrenite postupak popravka, stranica 8*

Ako trebate pomoć, obratite se podršci. Znamo da ovo zahtijeva vrijeme i trud, i tu smo da pomognemo.

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Jezik	Veza
English	<a href="https://support.apple.com/en-us/108366">https://support.apple.com/en-us/108366</a>
German	<a href="https://support.apple.com/de-de/108366">https://support.apple.com/de-de/108366</a>
French	<a href="https://support.apple.com/fr-fr/108366">https://support.apple.com/fr-fr/108366</a>
Spanish	<a href="https://support.apple.com/es-mx/108366">https://support.apple.com/es-mx/108366</a>
Dutch	<a href="https://support.apple.com/nl-nl/108366">https://support.apple.com/nl-nl/108366</a>
Swedish	<a href="https://support.apple.com/sv-se/108366">https://support.apple.com/sv-se/108366</a>
Norwegian	<a href="https://support.apple.com/no-no/108366">https://support.apple.com/no-no/108366</a>
Danish	<a href="https://support.apple.com/da-dk/108366">https://support.apple.com/da-dk/108366</a>
Pojednostavljeni kineski	<a href="https://support.apple.com/zh-cn/108366">https://support.apple.com/zh-cn/108366</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Japanese	<a href="https://support.apple.com/ja-jp/108366">https://support.apple.com/ja-jp/108366</a>
Italian	<a href="https://support.apple.com/it-it/108366">https://support.apple.com/it-it/108366</a>
Finnish	<a href="https://support.apple.com/fi-fi/108366">https://support.apple.com/fi-fi/108366</a>
Arabic	<a href="https://support.apple.com/ar-sa/108366">https://support.apple.com/ar-sa/108366</a>
Bugarski	<a href="https://support.apple.com/bg-bg/108366">https://support.apple.com/bg-bg/108366</a>
Poljski	<a href="https://support.apple.com/pl-pl/108366">https://support.apple.com/pl-pl/108366</a>
Slovenski	<a href="https://support.apple.com/sl-si/108366">https://support.apple.com/sl-si/108366</a>
Estonski	<a href="https://support.apple.com/et-ee/108366">https://support.apple.com/et-ee/108366</a>
Portugalski	<a href="https://support.apple.com/pt-pt/108366">https://support.apple.com/pt-pt/108366</a>
Croatian	<a href="https://support.apple.com/hr-hr/108366">https://support.apple.com/hr-hr/108366</a>
Icelandic	<a href="https://support.apple.com/is-is/108366">https://support.apple.com/is-is/108366</a>
Hebrew	<a href="https://support.apple.com/he-il/108366">https://support.apple.com/he-il/108366</a>

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Jezik	Veza
English	<a href="https://support.apple.com/en-us/111107">https://support.apple.com/en-us/111107</a>
German	<a href="https://support.apple.com/de-de/111107">https://support.apple.com/de-de/111107</a>
French	<a href="https://support.apple.com/fr-fr/111107">https://support.apple.com/fr-fr/111107</a>
Spanish	<a href="https://support.apple.com/es-mx/111107">https://support.apple.com/es-mx/111107</a>
Dutch	<a href="https://support.apple.com/nl-nl/111107">https://support.apple.com/nl-nl/111107</a>
Swedish	<a href="https://support.apple.com/sv-se/111107">https://support.apple.com/sv-se/111107</a>
Norwegian	<a href="https://support.apple.com/no-no/111107">https://support.apple.com/no-no/111107</a>
Danish	<a href="https://support.apple.com/da-dk/111107">https://support.apple.com/da-dk/111107</a>
Pojednostavljeni kineski	<a href="https://support.apple.com/zh-cn/111107">https://support.apple.com/zh-cn/111107</a>
Japanese	<a href="https://support.apple.com/ja-jp/111107">https://support.apple.com/ja-jp/111107</a>
Italian	<a href="https://support.apple.com/it-it/111107">https://support.apple.com/it-it/111107</a>
Finnish	<a href="https://support.apple.com/fi-fi/111107">https://support.apple.com/fi-fi/111107</a>
Arabic	<a href="https://support.apple.com/ar-sa/111107">https://support.apple.com/ar-sa/111107</a>
Bugarski	<a href="https://support.apple.com/bg-bg/111107">https://support.apple.com/bg-bg/111107</a>
Poljski	<a href="https://support.apple.com/pl-pl/111107">https://support.apple.com/pl-pl/111107</a>
Slovenski	<a href="https://support.apple.com/sl-si/111107">https://support.apple.com/sl-si/111107</a>
Estonski	<a href="https://support.apple.com/et-ee/111107">https://support.apple.com/et-ee/111107</a>
Portugalski	<a href="https://support.apple.com/pt-pt/111107">https://support.apple.com/pt-pt/111107</a>
Croatian	<a href="https://support.apple.com/hr-hr/111107">https://support.apple.com/hr-hr/111107</a>

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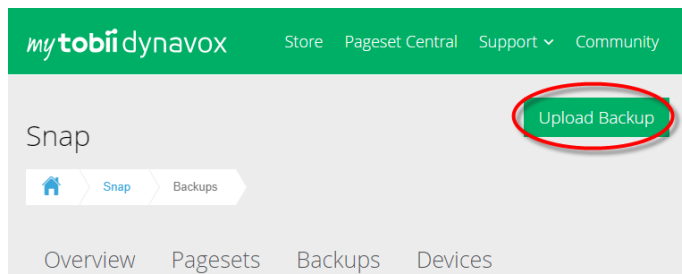
Jezik	Veza
English	<a href="https://support.apple.com/en-us/118291">https://support.apple.com/en-us/118291</a>
German	<a href="https://support.apple.com/de-de/118291">https://support.apple.com/de-de/118291</a>
French	<a href="https://support.apple.com/fr-fr/118291">https://support.apple.com/fr-fr/118291</a>
Spanish	<a href="https://support.apple.com/es-mx/118291">https://support.apple.com/es-mx/118291</a>
Dutch	<a href="https://support.apple.com/nl-nl/118291">https://support.apple.com/nl-nl/118291</a>
Swedish	<a href="https://support.apple.com/sv-se/118291">https://support.apple.com/sv-se/118291</a>
Norwegian	<a href="https://support.apple.com/no-no/118291">https://support.apple.com/no-no/118291</a>
Danish	<a href="https://support.apple.com/da-dk/118291">https://support.apple.com/da-dk/118291</a>
Pojednostavljeni kineski	<a href="https://support.apple.com/zh-cn/118291">https://support.apple.com/zh-cn/118291</a>
Japanese	<a href="https://support.apple.com/ja-jp/118291">https://support.apple.com/ja-jp/118291</a>
Italian	<a href="https://support.apple.com/it-it/118291">https://support.apple.com/it-it/118291</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Finnish	<a href="https://support.apple.com/fi-fi/118291">https://support.apple.com/fi-fi/118291</a>
Arabic	<a href="https://support.apple.com/ar-sa/118291">https://support.apple.com/ar-sa/118291</a>
Bugarski	<a href="https://support.apple.com/bg-bg/118291">https://support.apple.com/bg-bg/118291</a>
Poljski	<a href="https://support.apple.com/pl-pl/118291">https://support.apple.com/pl-pl/118291</a>
Slovenski	<a href="https://support.apple.com/sl-si/118291">https://support.apple.com/sl-si/118291</a>
Estonski	<a href="https://support.apple.com/et-ee/118291">https://support.apple.com/et-ee/118291</a>
Portugalski	<a href="https://support.apple.com/pt-pt/118291">https://support.apple.com/pt-pt/118291</a>
Croatian	<a href="https://support.apple.com/hr-hr/118291">https://support.apple.com/hr-hr/118291</a>
Icelandic	<a href="https://support.apple.com/is-is/118291">https://support.apple.com/is-is/118291</a>
Hebrew	<a href="https://support.apple.com/he-il/118291">https://support.apple.com/he-il/118291</a>

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Da biste vratili uređaj iz sigurnosne kopije, slijedite ove upute:

Jezik	Veza
English	<a href="https://support.apple.com/en-us/118105">https://support.apple.com/en-us/118105</a>
German	<a href="https://support.apple.com/de-de/118105">https://support.apple.com/de-de/118105</a>
French	<a href="https://support.apple.com/fr-fr/118105">https://support.apple.com/fr-fr/118105</a>
Spanish	<a href="https://support.apple.com/es-mx/118105">https://support.apple.com/es-mx/118105</a>
Dutch	<a href="https://support.apple.com/nl-nl/118105">https://support.apple.com/nl-nl/118105</a>
Swedish	<a href="https://support.apple.com/sv-se/118105">https://support.apple.com/sv-se/118105</a>
Norwegian	<a href="https://support.apple.com/no-no/118105">https://support.apple.com/no-no/118105</a>
Danish	<a href="https://support.apple.com/da-dk/118105">https://support.apple.com/da-dk/118105</a>
Pojednostavljeni kineski	<a href="https://support.apple.com/zh-cn/118105">https://support.apple.com/zh-cn/118105</a>
Japanese	<a href="https://support.apple.com/ja-jp/118105">https://support.apple.com/ja-jp/118105</a>
Italian	<a href="https://support.apple.com/it-it/118105">https://support.apple.com/it-it/118105</a>
Finnish	<a href="https://support.apple.com/fi-fi/118105">https://support.apple.com/fi-fi/118105</a>
Arabic	<a href="https://support.apple.com/ar-sa/118105">https://support.apple.com/ar-sa/118105</a>
Bugarski	<a href="https://support.apple.com/bg-bg/118105">https://support.apple.com/bg-bg/118105</a>
Poljski	<a href="https://support.apple.com/pl-pl/118105">https://support.apple.com/pl-pl/118105</a>
Slovenski	<a href="https://support.apple.com/sl-si/118105">https://support.apple.com/sl-si/118105</a>
Estonski	<a href="https://support.apple.com/et-ee/118105">https://support.apple.com/et-ee/118105</a>
Portugalski	<a href="https://support.apple.com/pt-pt/118105">https://support.apple.com/pt-pt/118105</a>
Croatian	<a href="https://support.apple.com/hr-hr/118105">https://support.apple.com/hr-hr/118105</a>
Icelandic	<a href="https://support.apple.com/is-is/118105">https://support.apple.com/is-is/118105</a>
Hebrew	<a href="https://support.apple.com/he-il/118105">https://support.apple.com/he-il/118105</a>

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## Afrikans

Set stranica	URL
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_af_ZA.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_af_ZA.pdf</a>

## Arabic

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_ar.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_ar.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_ar.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_ar.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_ar.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_ar.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_ar.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_ar.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ar.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ar.pdf</a>

## Bugarski

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_bg-BG.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_bg-BG.pdf</a>

## Danish

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_da.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_da.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_da.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_da.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_da.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_da.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_da.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_da.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_da_DK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_da_DK.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_da.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_da.pdf</a>

## Dutch

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_nl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_nl.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_nl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_nl.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_nl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_nl.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_nl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_nl.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nl_NL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nl_NL.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_nl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_nl.pdf</a>

## Engleski - Australija

Set stranica	URL
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_AU.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_AU.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_AU.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_AU.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en_AU.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en_AU.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en_AU.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en_AU.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-UK.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_en-UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_en-UK.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_en-UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_en-UK.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_en-UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_en-UK.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_UK.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_UK.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_UK.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en_GB.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en_GB.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en_GB.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en_GB.pdf</a>

## Engleski (SAD)

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_en.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_en.pdf</a>
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-US.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_en-US.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_en-US.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_en-US.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en-US.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en-US.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en-US.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en-US.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en-US.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_en.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_en.pdf</a>

## Estonski

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_et-EE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_et-EE.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_et_EE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_et_EE.pdf</a>

## Finnish

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fi.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fi.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fi.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fi.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fi.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fi.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fi.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fi.pdf</a>

## Francuski (Francuska)

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fr.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fr.pdf</a>
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_fr_FR.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_fr_FR.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-FR.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-FR.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-FR.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-FR.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-FR.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-FR.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_FR.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_FR.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_fr.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_fr.pdf</a>

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Set stranica	URL
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-CA.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-CA.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-CA.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-CA.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-CA.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-CA.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_CA.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_CA.pdf</a>

## German

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_de.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_de.pdf</a>
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_de_DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_de_DE.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_de-DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_de-DE.pdf</a>
Core First 5x5 (PCS)	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_PCS_de.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_PCS_de.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de-DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de-DE.pdf</a>
Core First 6x6 (PCS)	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de_PCS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de_PCS.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_de-DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_de-DE.pdf</a>
Core First 7x7 (PCS)	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_PCS_de.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_PCS_de.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_de_DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_de_DE.pdf</a>
Express METACOM	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_de-DE-Metacom.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_de-DE-Metacom.pdf</a>
Motor Plan 30 METACOM	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_METACOM_de-DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_METACOM_de-DE.pdf</a>
Motor Plan 30 PCS	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30-PCS_all_de_DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30-PCS_all_de_DE.pdf</a>
Motor Plan 40 METACOM	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_de-DE-Metacom.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_de-DE-Metacom.pdf</a>
Motor Plan 40 PCS	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40-PCS_all_de_DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40-PCS_all_de_DE.pdf</a>
Motor Plan 66 METACOM	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_de-DE-Metacom.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_de-DE-Metacom.pdf</a>
Motorni plan 66 PCS	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66-PCS_all_de_DE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66-PCS_all_de_DE.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_de.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_de.pdf</a>

## Hebrew

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner assisted scanning he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner assisted scanning he-IL.pdf</a>
Core First 5x5 za žene	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_female_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_female_he-IL.pdf</a>
Core First 5x5 za muškarce	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_male_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_male_he-IL.pdf</a>
Core First 6x6 ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_female_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_female_he-IL.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Core First 6x6 muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_male_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_male_he-IL.pdf</a>
Core First 7x7 ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_female_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_female_he-IL.pdf</a>
Core First 7x7 muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_male_he-IL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_male_he-IL.pdf</a>

## Icelandic

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_is-IS.pdf</a>
Core First 5x5 za žene	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_female_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_female_is-IS.pdf</a>
Core First 5x5 za muškarce	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_male_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_male_is-IS.pdf</a>
Core First 6x6 ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_female_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_female_is-IS.pdf</a>
Core First 6x6 muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_male_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_male_is-IS.pdf</a>
Core First 7x7 ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_female_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_female_is-IS.pdf</a>
Core First 7x7 muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_male_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_male_is-IS.pdf</a>
Express ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-female.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-female.pdf</a>
Express muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-male.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-male.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_is-IS.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_is-IS.pdf</a>

## Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

## Irski

Set stranica	URL
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ga_IE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ga_IE.pdf</a>

## Italian

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_it.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_it.pdf</a>
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_it_IT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_it_IT.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_it-IT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_it-IT.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_it-IT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_it-IT.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_it-IT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_it-IT.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_it_IT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_it_IT.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_it.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_it.pdf</a>

## Japanese

Set stranica	URL
Core First 5x5 Hiragana	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Hiragana_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Hiragana_ja.pdf</a>
Core First 5x5 Kanji	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Kanji_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Kanji_ja.pdf</a>
Core First 6x6 Hiragana	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Hiragana_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Hiragana_ja.pdf</a>
Core First 6x6 Kanji	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Kanji_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Kanji_ja.pdf</a>
Core First 7x7 Hiragana	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Hiragana_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Hiragana_ja.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Core First 7x7 Kanji	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Kanji_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Kanji_ja.pdf</a>
Express Hiragana	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Hiragana_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Hiragana_ja.pdf</a>
Express Kanji	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Kanji_ja.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Kanji_ja.pdf</a>

## Norwegian

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_no.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_no.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_no.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_no.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_no.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_no.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_no.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_no.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nb_NO.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nb_NO.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_no.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_no.pdf</a>

## Poljski

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pl.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pl.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pl.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pl.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pl.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pl.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

## Portugalski - Portugal

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pt.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pt.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pt.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pt.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pt.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pt.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pt.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pt.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pt_PT.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pt_PT.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_pt.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_pt.pdf</a>

## Slovenski

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_sl-SL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_sl-SL.pdf</a>
Express ženski	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_female_sl-SL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_female_sl-SL.pdf</a>
Express muški	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_male_sl-SL.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_male_sl-SL.pdf</a>

## Španjolski - Meksiko

Set stranica	URL
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es_MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es_MX.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_MX.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_es_MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_es_MX.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_es_MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_es_MX.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_es_MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_es_MX.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_es-MX.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_es-MX.pdf</a>

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Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_es.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_es.pdf</a>
Afazija	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es-ES.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es-ES.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_es.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_es.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_es.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_es.pdf</a>
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_es.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_es.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_ES.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_ES.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_es_ES.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_es_ES.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_es_ES.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_es_ES.pdf</a>
Motor Plan	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_es_ES.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_es_ES.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_es.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_es.pdf</a>

## Swedish

Set stranica	URL
Skeniranje uz pomoć partnera	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_sv.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_sv.pdf</a>
Core First 5x5	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_sv.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_sv.pdf</a>
Core First 6x6	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_sv.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_sv.pdf</a>

### Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

#### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.

Set stranica	URL
Core First 7x7	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_sv.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_sv.pdf</a>
Express	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_sv_SE.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_sv_SE.pdf</a>
Tekst	<a href="https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_sv.pdf">https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_sv.pdf</a>

## Podrška za uređaj Tobii Dynavox

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Pogledajte stranicu za podršku za svoj uređaj Tobii Dynavox. Ona sadrži ažurne informacije o rješavanju problema, kao i praktične savjete i upute u vezi s proizvodom. Potražite internetske stranice naše korisničke podrške: <https://www.tobiidynavox.com/pages/product-support>

### Obratite se svom savjetniku za opremu ili distributeru

Za pitanja ili probleme u vezi s vašim proizvodom kontaktirajte svog savjetnika za opremu Tobii Dynavox ili ovlaštenog distributera koji će vam pomoći. Oni su najbolje upoznati s vašim osobnim postavkama i mogu vam najbolje pomoći savjetima i obukom u vezi s proizvodom. Za kontakt podatke pogledajte <https://www.tobiidynavox.com/pages/contact-us>.