Upplýsingar um innköllun TD Navio

Undirbúningur

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| Tungumál | Tengill |
|---------------------|---|
| Enska | https://support.apple.com/en-us/108366 |
| Þýska | https://support.apple.com/de-de/108366 |
| Franska | https://support.apple.com/fr-fr/108366 |
| Spænska | https://support.apple.com/es-mx/108366 |
| Hollenska | https://support.apple.com/nl-nl/108366 |
| Sænska | https://support.apple.com/sv-se/108366 |
| Norska | https://support.apple.com/no-no/108366 |
| Danska | https://support.apple.com/da-dk/108366 |
| Einfölduð kínverska | https://support.apple.com/zh-cn/108366 |
| Japanska | https://support.apple.com/ja-jp/108366 |

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| | |
|------------|---|
| Ítalska | https://support.apple.com/it-it/108366 |
| Finnska | https://support.apple.com/fi-fi/108366 |
| Arabíska | https://support.apple.com/ar-sa/108366 |
| Búlgarska | https://support.apple.com/bg-bg/108366 |
| Pólska | https://support.apple.com/pl-pl/108366 |
| Slóvenska | https://support.apple.com/sl-si/108366 |
| Eistneska | https://support.apple.com/et-ee/108366 |
| Portúgölsk | https://support.apple.com/pt-pt/108366 |
| Króatíska | https://support.apple.com/hr-hr/108366 |
| Íslenska | https://support.apple.com/is-is/108366 |
| Hebreska | https://support.apple.com/he-il/108366 |

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| Tungumál | Tengill |
|---------------------|---|
| Enska | https://support.apple.com/en-us/111107 |
| Þýska | https://support.apple.com/de-de/111107 |
| Franska | https://support.apple.com/fr-fr/111107 |
| Spænska | https://support.apple.com/es-mx/111107 |
| Hollenska | https://support.apple.com/nl-nl/111107 |
| Sænska | https://support.apple.com/sv-se/111107 |
| Norska | https://support.apple.com/no-no/111107 |
| Danska | https://support.apple.com/da-dk/111107 |
| Einfölduð kínverska | https://support.apple.com/zh-cn/111107 |
| Japanska | https://support.apple.com/ja-jp/111107 |
| Ítalska | https://support.apple.com/it-it/111107 |
| Finnska | https://support.apple.com/fi-fi/111107 |
| Arabíska | https://support.apple.com/ar-sa/111107 |
| Búlgarska | https://support.apple.com/bg-bg/111107 |
| Pólska | https://support.apple.com/pl-pl/111107 |
| Slóvenska | https://support.apple.com/sl-si/111107 |
| Eistneska | https://support.apple.com/et-ee/111107 |
| Portúgölsk | https://support.apple.com/pt-pt/111107 |
| Króatíska | https://support.apple.com/hr-hr/111107 |
| Íslenska | https://support.apple.com/is-is/111107 |
| Hebreska | https://support.apple.com/he-il/111107 |

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Afríkanska

| Síðusettt | URL |
|-----------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_af_ZA.pdf |

Arabíska

| Síðusetting | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_ar.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_ar.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_ar.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_ar.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ar.pdf |

Búlgarska

| Síðusetting | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_bg-BG.pdf |

Danska

| Síðusetting | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_da.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_da.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_da.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_da.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_da_DK.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_da.pdf |

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| Síðusettt | URL |
|----------------|---|
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_nl.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_nl.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_nl.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nl_NL.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_nl.pdf |

Enska - Ástralía

| Síðusettt | URL |
|---------------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_AU.pdf |
| Motor Plan 30 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_AU.pdf |
| Motor Plan 40 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en_AU.pdf |
| Motor Plan 66 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en_AU.pdf |

Enska - Stóra-Bretland

| Síðusettt | URL |
|----------------|---|
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-UK.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_en-UK.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_en-UK.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_en-UK.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en_UK.pdf |
| Motor Plan 30 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en_UK.pdf |

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Enska (Bandaríkin)

| Síðusettt | URL |
|------------------------|---|
| Skimun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_en.pdf |
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_en-US.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_en-US.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_en-US.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_en-US.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_en-US.pdf |
| Motor Plan 30 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_en-US.pdf |
| Motor Plan 40 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_en-US.pdf |
| Motor Plan 66 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_en-US.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_en.pdf |

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Finnska

| Síðusettt | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fi.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fi.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fi.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fi.pdf |

Franska (Frakkland)

| Síðusettt | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_fr.pdf |
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_fr_FR.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-FR.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-FR.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-FR.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_FR.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_fr.pdf |

Franska - Kanada

| Síðusettt | URL |
|----------------|---|
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_fr-CA.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_fr-CA.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_fr-CA.pdf |

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| Síðusettt | URL |
|-----------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_fr_CA.pdf |

Þýska

| Síðusettt | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_de.pdf |
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_de_DE.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_de-DE.pdf |
| Core First 5x5 (PCS) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_PCS_de.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de-DE.pdf |
| Core First 6x6 (PCS) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_de_PCS.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_de-DE.pdf |
| Core First 7x7 (PCS) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_PCS_de.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_de_DE.pdf |
| Express METACOM | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_de-DE-Metacom.pdf |
| Motor Plan 30 METACOM | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_METACOM_de-DE.pdf |
| Mótorplanið 30 PCS | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_PCS_all_de_DE.pdf |
| Motor Plan 40 METACOM | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_de-DE-Metacom.pdf |
| Motor Plan 40 PCS | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_PCS_all_de_DE.pdf |

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Hebreska

| Síðuset | URL |
|-------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_he-IL.pdf |
| Core First 5x5 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_female_he-IL.pdf |
| Core First 5x5 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_male_he-IL.pdf |
| Core First 6x6 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_female_he-IL.pdf |
| Core First 6x6 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_male_he-IL.pdf |
| Core First 7x7 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_female_he-IL.pdf |
| Core First 7x7 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_male_he-IL.pdf |

Íslenska

| Síðuset | URL |
|-------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_is-IS.pdf |
| Core First 5x5 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_female_is-IS.pdf |

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| Síðusettt | URL |
|-------------------------|---|
| Core First 5x5 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_male_is-IS.pdf |
| Core First 6x6 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_female_is-IS.pdf |
| Core First 6x6 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_male_is-IS.pdf |
| Core First 7x7 Kvenkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_female_is-IS.pdf |
| Core First 7x7 Karlkyns | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_male_is-IS.pdf |
| Express (kvenkyns) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-female.pdf |
| Express (karlkyns) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_is-IS-male.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_is-IS.pdf |

Írska

| Síðusettt | URL |
|-----------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_ga_IE.pdf |

Ítalska

| Síðusettt | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_it.pdf |
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_it_IT.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_it-IT.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_it-IT.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_it-IT.pdf |

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| Síðuset | URL |
|---------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_it_IT.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_it.pdf |

Japanska

| Síðuset | URL |
|-------------------------|---|
| Core First 5x5 Hiragana | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Hiragana_ja.pdf |
| Core First 5x5 Kanji | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_all_Kanji_ja.pdf |
| Core First 6x6 Hiragana | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Hiragana_ja.pdf |
| Core First 6x6 Kanji | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_all_Kanji_ja.pdf |
| Core First 7x7 Hiragana | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Hiragana_ja.pdf |
| Core First 7x7 Kanji | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_all_Kanji_ja.pdf |
| Express Hiragana | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Hiragana_ja.pdf |
| Express Kanji | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_Kanji_ja.pdf |

Norska

| Síðuset | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_no.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_no.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_no.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_no.pdf |

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| Síðusetting | URL |
|-------------|---|
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_nb_NO.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_no.pdf |

Pólska

| Síðusetting | URL |
|------------------------|---|
| Skimun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pl.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pl.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pl.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pl.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pl.pdf |

Portúgalska (Portúgal)

| Síðusetting | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_pt.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_pt.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_pt.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_pt.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_pt_PT.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_pt.pdf |

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| Síðuset | URL |
|--------------------|---|
| Express (kvenkyns) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_female_sl-SL.pdf |
| Express (karlkyns) | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_male_sl-SL.pdf |

Spænsku - Mexíkó

| Síðuset | URL |
|---------------|---|
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es_MX.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_MX.pdf |
| Motor Plan 30 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan30_all_es_MX.pdf |
| Motor Plan 40 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan40_all_es_MX.pdf |
| Motor Plan 66 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_MotorPlan66_all_es_MX.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_es-MX.pdf |

Spænska (Spánn)

| Síðuset | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_es.pdf |
| Málstol | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Aphasia_all_es-ES.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_es.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_es.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_es.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_es_ES.pdf |

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Sænska

| Síðusettt | URL |
|------------------------|---|
| Skoðun með aðstoð maka | https://download.mytobiidynavox.com/Resources/printable_comm_boards/Partner_assisted_scanning_sv.pdf |
| Core First 5x5 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_5x5_sv.pdf |
| Core First 6x6 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_6x6_sv.pdf |
| Core First 7x7 | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_CoreFirst_7x7_sv.pdf |
| Express | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Express_all_sv_SE.pdf |
| Texti | https://download.mytobiidynavox.com/Resources/printable_comm_boards/TD_Snap_Text_sv.pdf |

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