

Customer support: For more information or support, please visit www.TobiiDynavox.com.

Release Version 1.19.3 (iPadOS only)

Release date: 2024-02-08

New Features

- Added ability to use Apple Personal Voice on iPadOS.

Release Version 1.19.2

Release date: 2023-12-11

Improvements Windows

- Fixed an issue where speaking with a SAPI 5 voice could cause the application to crash.

Improvement iPadOS

- Fixed an issue where metrics were being reported to the wrong destination.
- Improved stability of app by batching web executions from the native code.
- Fixed issue where buttons could continually react to gaze after enabling Assistive Touch.

Release Version 1.19.1

Release date: 2023-10-31

New Features

- Added application language option Japanese (ja-JP).
- Added ability for Flemish users to use word prediction.

Improvements

- Fixed an issue where word predictions for the Japanese Keyboard were not behaving correctly when given Latin input.
- Fixed an issue where word predictions would not update using the Japanese keyboard.

Release Version 1.18.3 (iPadOS only)

Release date: 2023-07-06

Improvements

- Fixed issue where buttons could continually react to gaze after enabling Assistive Touch.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: <https://www.tobiidynavox.com/pages/product-support>

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit <https://www.tobiidynavox.com/pages/contact-us>.

Release Version 1.18.2 (iPadOS only)

Release date: 2023-06-27

Improvements

- Improved stability of app by batching web executions from the native code.

Release Version 1.18.1 (iPadOS only)

Release date: 2023-06-13

Improvements

- Fixed issue where app could freeze when either changing keyboard language or bringing the app back to the foreground.
- Fixed issue where metrics were being reported against the wrong destination.

Release Version 1.18.0

Release date: 2023-06-07

New Features

- Added ability to configure the rate of a voice via Voice Settings.
- Updated keyboards to change the hyphen (-) character to an underscore (_) character when the Shift key is active.
- Added ability to configure the pitch of a voice via Voice Settings.
- Added ability to preview a voice with a certain rate and pitch via Voice Settings.
- Added ability to change system volume via +/- Adaptive Buttons on I-Series devices.
- Added analytics surrounding the usage of the Editing Menu.
- Added ability to configure the text size within the application.
- Updated Voices Setting UX to be consistent with the Accessibility Suite.
- Updated all keyboards to present Shift Key in a position consistent with the Accessibility Suite.
- Added ability to configure pronunciations via Voice Settings.
- Added flags to keyboard items in Keyboard Settings.
- Updated sound played when adjusting system volume via Adaptive Buttons.

Improvements

- Fixed issue where no word predictions were suggested after starting up without being connected to the internet.
- Fixed issue where eye tracking could contain large offsets when running TD Talk on certain systems.

Release Version 1.14.2 (Windows only)

Release date: 2023-05-17

Improvements

- Fixed issue where eye tracking could contain large offsets when running TD Talk on certain systems.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: <https://www.tobiidynavox.com/pages/product-support>

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit <https://www.tobiidynavox.com/pages/contact-us>.

Release Version 1.14.0

Release date: 2023-05-04

New Features

- Added ability to navigate to Keyboard Settings from within the Languages View of the Keyboard.

Improvements

- Updated analytics reporting to report which version of the application is being run.
- Fixed issue where speaking certain phrases with an Acapela my-own-voice voice could cause the application to crash.
- Fixed issue where Acapela my-own-voice accounts with certain characters in their password could not be logged into.

Windows Only

- Updated word prediction to be shared with TD Phone and TD Browse.
- Updated font to be consistent with the rest of the Accessibility Suite.
- Updated start up procedure to show the splash screen at the earliest possible moment.
- Updated analytics reporting to report with the Accessibility Suite shared user identifier.
- Fixed issue where Quick Messages at the top of TD Talk were not able to be activated with touch.
- Fixed issue where TD Talk bundle installer could not successfully install on an TD I-Series device.

iPadOS Only

- Updated font to be Gilroy.

Release Version 1.12.0

Release date: 2022-06-21

New Features

- Added ability to log into and install voices from an Acapela my-own-voice service account.
- Added ability to log into and install voices from a ModelTalker account.
- Added ability to log into and install voices from a The Voice Keeper account.
- Updated Quick Messages to be spoken with the voice configured against the language (i.e. the keyboard) the Quick Message item was originally created with.

Improvements

- Fixed issue where context menus were always English, context menus are now localized according to the iPadOS system language if it is one of the following - Dansk, Deutsch, English, Español, Français, Italiano, Norsk, Nederlands or Svenska.
- Fixed issue where word predictions for new sentences were being predicted according to the previous sentence content.
- Fixed issue where replacing words immediately before a punctuation with a word prediction selection would add a whitespace before the punctuation.

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: <https://www.tobiidynavox.com/pages/product-support>

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit <https://www.tobiidynavox.com/pages/contact-us>.

- Fixed issue where keyboard installations could get into a corrupt state preventing installation to perform even after a restart.
- Fixed issue where the Spanish keyboards presented the incorrect accented 'i' character.
- Fixed issue where it was possible to start multiple keyboard installations at once, causing the application to stop responding to gaze.
- Fixed issue where surrounding buttons would respond to gaze while looking at the voice installation progress bar.
- Fixed issue where the Discard button on Save Quick Message confirmation dialog showed wrong text in French language.

Release Version 1.4.0

Release date: 2022-01-19

New Features

- Added ability to sample iPadOS system voices.
- Added ability to pipe speech through a FaceTime call (requires use of an iPadOS system voice).

Improvements

- Updated numeric layout of all Keyboards to be consistent with TD Control (English (US) and English (UK) update immediately, others require internet connection to automatically update).
- Fixed issue where speech was not being heard when iPad not connected to TD Pilot and "Silent Mode" was active.
- Fixed issue where TD Talk could sometimes start in a way where it did not respond to gaze until it was suspended and resumed.

Release Version 1.3.0

Release date: 2022-01-05

New Features

- Added an English (UK) keyboard that is immediately available without need of download.
- Added the Tamira Acapela voice available for download (internet connectivity required to detect the voice).

Improvements

- Fixed issue where some voices were not being displayed in their respective language filter in Voice Settings.
- Fixed issue where language names would display as a language code in Application Language Settings if device not connected to the internet on very first run.
- Fixed issue where TD Talk sometimes crashed when connecting to TD Pilot.
- Fixed issue where native iPadOS voices were not available for Norwegian.
- Fixed issue where keyboard was reset from selected keyboard to keyboard of application language after a restart of TD Talk.
- Fixed issue that scroll buttons in Keyboard Languages Settings page were not disabled when page was scrolled to beginning or end of scroll content.

Support for Your Tobii Dynavox Device

Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.
Find our Support pages online at: <https://www.tobiidynavox.com/pages/product-support>

Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit <https://www.tobiidynavox.com/pages/contact-us>.