

# TD Talk

## User's Manual



## User's Manual TD Talk

Version 1.8

2025-07-08

All rights reserved.

Copyright © Dynavox Group AB (publ)

No part of this document may be reproduced, stored in a retrieval system, or transmitted in any form, by any means (electronic, photocopying, recording, or otherwise) without the prior written permission of the publisher.

Copyright protection claimed includes all forms and matters of copyrightable material and information allowed by statutory or judicial law or hereafter granted, including without limitation, material generated from the software programs which are displayed on the screen such as screen displays, menus, etc.

The information contained in this document is proprietary to Dynavox Group AB. Any reproduction in part or whole without prior written authorization by Dynavox Group AB is prohibited.

Products that are referred to in this document may be either trademarks and/or registered trademarks of the respective owners. The publisher and the author make no claim to these trademarks.

While every precaution has been taken in the preparation of this document, the publisher and the author assume no responsibility for errors or omissions, or for damages resulting from the use of information contained in this document or from the use of programs and source code that may accompany it. In no event shall the publisher and the author be liable for any loss of profit or any other commercial damage caused or alleged to have been caused directly or indirectly by this document.

Content subject to change without notice.

Please check Tobii Dynavox website.

[www.TobiiDynavox.com](http://www.TobiiDynavox.com) for updated versions of this document.

### Contact Information:

Dynavox Group AB  
Löjtnantsgatan 25  
115 50 Stockholm  
Sweden

+46 8 522 950 20

Tobii Dynavox LLC  
2100 Wharton Street, Suite 400  
Pittsburgh, PA 15203  
USA

+1-800-344-1778

Tobii Dynavox Ltd.  
Sheffield Technology Parks  
Cooper Buildings  
Arundel Street  
Sheffield S1 2NS  
United Kingdom

+44 (0)114 481 00 11

Tobii Dynavox Pty. Ltd.  
Trading as Link Assistive  
11B MAB Eastern Promenade  
Tonsley SA, 5042  
Australia

+61 8 7120 6002

Tobii Dynavox (Suzhou) Co. Ltd  
Unit 11/12, Floor 3, Building B, No.5 Xinghan Street, SIP,  
Suzhou  
P.R.China 215021

+86 512 69362880




# Table of Contents

<b>1</b>	<b>Welcome.....</b>	<b>6</b>
1.1	System Requirements.....	6
1.2	Supported Languages.....	6
1.3	Getting TD Suite .....	6
<b>2</b>	<b>Resources and Support.....</b>	<b>7</b>
2.1	Web Resources and Support.....	7
2.2	Learning Hub .....	7
<b>3</b>	<b>Getting Started.....</b>	<b>8</b>
3.1	Overview of TD Talk .....	8
3.2	The Off-Screen Menu .....	9
3.2.1	How do I access the Off-Screen Menu in TD Talk? .....	9
3.3	The Dashboard .....	10
3.3.1	How do I access the Dashboard in TD Talk? .....	10
3.3.2	How do I Exit and Close Down TD Talk? .....	10
3.4	Setting Up TD Talk.....	11
<b>4</b>	<b>Using TD Talk.....</b>	<b>12</b>
4.1	Using the Keyboard .....	12
4.2	Using Predictions .....	12
4.2.1	Word Predictions.....	12
4.2.2	Phrase Prediction.....	13
4.3	Editing a Message .....	13
4.3.1	How do I Edit a Message in TD Talk? .....	13
4.4	Copy and Paste .....	14
4.4.1	How do I Use Copy in TD Talk?.....	14
4.4.2	How do I Use Paste in TD Talk? .....	14
4.5	Using Quick Messages .....	14
4.5.1	How do I use Quick Messages in TD Talk? .....	14
4.5.2	How do I Change the Topic for Quick Messages?.....	15
4.6	Changing the Language.....	16
4.6.1	How can I Switch Speaking Languages in TD Talk? .....	16
4.7	Chat .....	16
4.8	See Through Mode .....	17
4.8.1	How do I Start See Through Mode.....	17
4.8.2	How do I End See Through Mode.....	17
4.9	How do I Use Abbreviation Expansions in TD Talk? .....	17
4.10	How do I Undo an Action in TD Phone? .....	18
<b>5</b>	<b>TD Talk Settings .....</b>	<b>19</b>
5.1	My Account .....	20
5.1.1	What is a myTobiiDynavox Account?.....	20
5.1.2	How do I Create a myTobiiDynavox Account?.....	20
5.1.3	How do I Backup my Personalized Data? .....	20
5.1.4	How do I Restore my Personalized Data? .....	21

5.2	Activation .....	21
5.2.1	Activation Method.....	22
5.2.1.1	How do I Select my Activation Method in TD Talk? .....	22
5.2.2	Buttons .....	23
5.2.2.1	How do I Change the Dwell Time for Activating Buttons in TD Talk?.....	23
5.2.2.2	How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?.....	23
5.2.3	Switch Input.....	24
5.2.3.1	How do I Set a Switch Input for my Switch in TD Talk? .....	24
5.3	Keyboard .....	24
5.3.1	Activation Method.....	25
5.3.1.1	How do I Select my Activation Method for the Keyboard in TD Talk?.....	25
5.3.2	Buttons .....	25
5.3.2.1	How do I Change the Dwell Time for Keyboard Buttons in TD Talk?.....	25
5.3.2.2	How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?.....	26
5.3.3	Prediction .....	26
5.3.3.1	How do I Change the Dwell Time for Prediction Buttons in TD Talk?.....	26
5.3.3.2	How do I Fine Tune the Dwell Time for Prediction Buttons in TD Talk?.....	27
5.3.4	General .....	27
5.3.4.1	How do I Add/Download a New Language/Keyboard Language in TD Talk?.....	27
5.3.4.2	How do I select and activate the Keyboard Languages to use/be available in TD Talk?.....	28
5.3.4.3	How do I Turn On/Off Type Sound Feedback in TD Talk?.....	28
5.4	Conversations .....	29
5.4.1	Topics.....	30
5.4.1.1	How can I Create a New Topic in TD Talk? .....	30
5.4.1.2	How can I Change the Name of a Topic in TD Talk? .....	30
5.4.1.3	How can I Make the Topic Private in TD Talk? .....	31
5.4.1.4	How can I Select the Active Topics TD Talk? .....	31
5.4.1.5	How can I Delete a Topic in TD Talk?.....	32
5.4.1.6	How can I Edit the Quick Messages in TD Talk? .....	32
5.4.1.7	How can I Add a Quick Message in TD Talk?.....	33
5.4.1.8	How can I Delete a Quick Message in TD Talk?.....	34
5.4.2	Abbreviation Expansions.....	35
5.4.2.1	How do I Add an Abbreviation Expansion in TD Talk?.....	35
5.4.2.2	How do I Edit an Abbreviation Expansion in TD Talk?.....	36
5.4.2.3	How do I Delete an Abbreviation Expansion in TD Talk?.....	36
5.5	Voice .....	37
5.5.1	Voices .....	38
5.5.1.1	Installed & Downloadable Voices .....	38
5.5.1.1.1	How do I Download a New Voice in TD Talk?.....	38
5.5.1.1.2	How do I Select my Voice in TD Talk? .....	38
5.5.1.1.3	How can I Apply a Filter to the Available Voices in TD Talk? .....	39
5.5.1.2	Voice Configuration .....	40
5.5.1.2.1	How do I Make Changes to my Voice in TD Talk? .....	40
5.5.1.3	Pronunciations.....	41

	5.5.1.3.1	How do I Create Pronunciation Exceptions in TD Talk?.....	42
5.5.2		Voice Banking Accounts .....	42
	5.5.2.1	Voice Banking Accounts.....	43
	5.5.2.1.1	How can I use my Voice from my Voice Banking Service? .....	43
5.6		Device .....	44
	5.6.1	Partner Window.....	44
	5.6.1.1	How do I Enable/Disable the Partner Window in TD Talk? .....	44
	5.6.1.2	How do I Set the Display Options for the Partner Window in TD Talk?.....	45
5.7		General .....	46
	5.7.1	Application Language.....	46
	5.7.1.1	How do I set my Application language in TD Talk?.....	46
	5.7.2	Text Size .....	47
	5.7.2.1	How do I Change the Text Size in TD Talk? .....	47
	5.7.3	Feedback .....	47
	5.7.3.1	How do I Send Feedback to Tobii Dynavox in TD Talk? .....	47
	5.7.4	Terms of Service & Privacy Notice.....	48
	5.7.4.1	How do I Open the Terms of Service .....	48
	5.7.4.2	How do I Open the Privacy Notice .....	48
	5.7.5	Collect Logs .....	49
	5.7.5.1	How do I Collect Logs for TD Talk?.....	49

# 1 Welcome

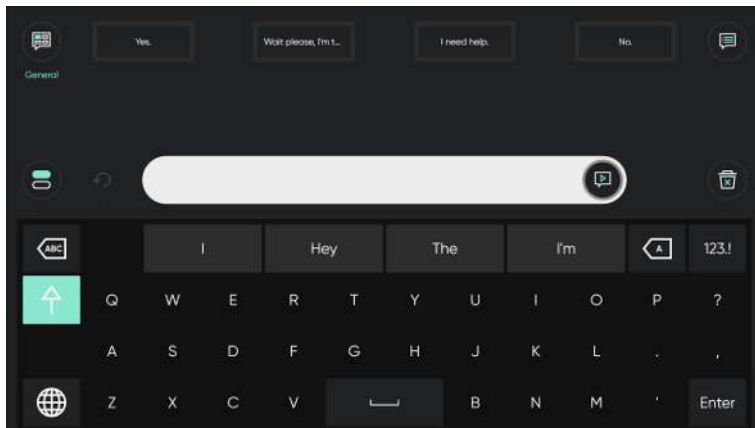

**Illustration 1:** TD Talk

TD Talk empowers people to communicate who they are and what they want to say, in the most fluid way possible. It is designed to make sentence construction as fast as possible. TD Talk enables individuals using it to remain an active part of the conversation. TD Talk looks and feels familiar so that people using it can get started in the shortest time possible. By learning a person's unique sentence patterns and word use, TD Talk continuously adapts to offer personally relevant options for communication.

## 1.1 System Requirements

*Table 1:* iOS

	Operating System	Device	Memory	Touch	Camera
<b>Minimum</b>	iOS 15.1	iPad 7th generation	4 GB	Integrated touch	Integrated camera
<b>Recommended</b>	iOS 15.1	iPad 8th generation	4 GB	Integrated touch	Integrated camera

## 1.2 Supported Languages

- Danish (DK)
- Dutch (NL)
- English (US)
- French (FR)
- German (DE)
- Italian (IT)
- Japanese (JP)
- Norwegian (NO)
- Spanish (Latin America)
- Swedish (SE)
- More languages coming soon!

## 1.3 Getting TD Suite

TD Talk is available through [TobiiDynavox.com](https://TobiiDynavox.com) for Windows devices and the Apple App Store for iPadOS.

## 2 Resources and Support

Tobii Dynavox offers several free resources to support and supplement your experience with TD Talk. Don't miss out on these tools that will help you get the most out of your Tobii Dynavox software!

### 2.1 Web Resources and Support

If you need support, your first stop should be the Training & Support section of the Tobii Dynavox website.

### 2.2 Learning Hub



The Learning Hub is currently English-only.

Visit [learn.tobiidynavox.com](https://learn.tobiidynavox.com) for documentation, training, and other resources, including:

- answers to common questions
- training videos
- recorded webinars
- online courses

## 3 Getting Started

If TD Talk is going to be used together with an Eye tracker, please make sure the to have an active calibration for the Eye tracker.

- iOS — use the TD CoPilot app for setting up the Eye tracker.

For more information, see the TD Pilot User's manual.

### 3.1 Overview of TD Talk

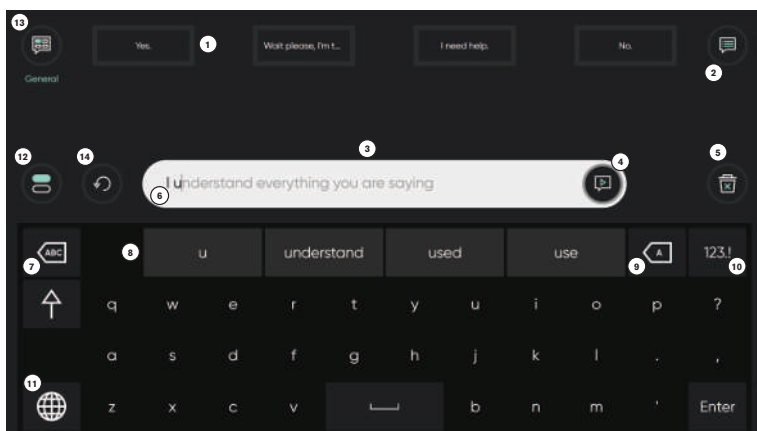



Illustration 2: TD Talk

Position	Description	Position	Description
1	<b>Quick Messages</b> Pre-written messages that speak immediately when selected	8	<b>Word Prediction/Abbreviation Expansions</b> Automatically predicts words based on what you've typed so far. When using Abbreviation Expansions the first word will be replace with the Expansion.
2	<b>More Quick Messages</b> Access all of the Quick Messages for the current Topic.	9	<b>Backspace</b> Delete the character to the left of the cursor.
3	<b>Message Window</b> View and edit the text you would like to speak here.	10	<b>Numbers Page</b> Access to numbers and special characters.
4	<b>Speak</b> Speak the text in the Message Window. You may also pause speech and resume speaking.	11	<b>Change Language</b> Switch between the languages you speak.
5	<b>Clear</b> Clear all text in the Message Window.	12	<b>Chat</b> Switch to a second Message Window to write a new message without losing the text in the first Message Window.
6	<b>Phrase Prediction</b> TD Talk learns from what you type and uses that information to predict phrases. Prediction is based on frequency and can make communication faster and more personal.	13	<b>Topics</b> Organize your Quick Messages by Topic. Switch to a Topic to access those messages.



Position	Description	Position	Description
7	<b>Delete Word</b> Delete the word to the left of the cursor.	14	<b>Undo</b> Undo the actions in the Message Window.

## 3.2 The Off-Screen Menu

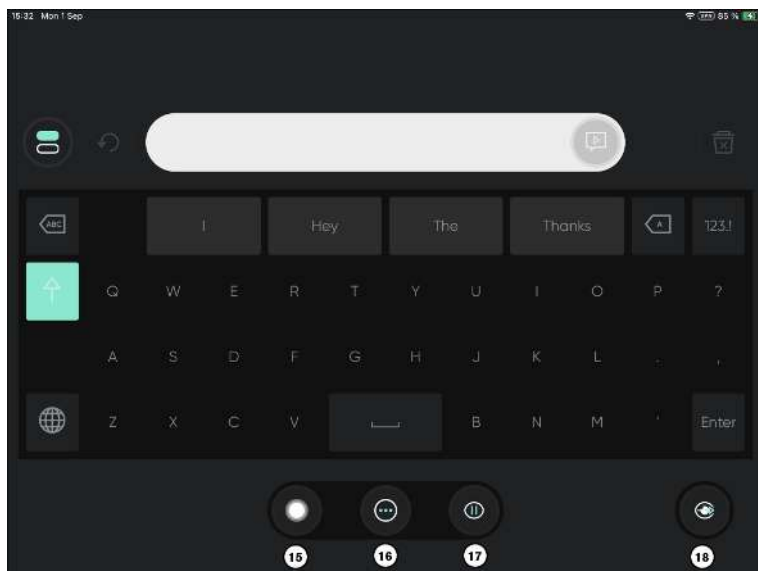

**Illustration 3:** TD Talk Off-Screen Menu


Position	Description	Position	Description
15	<b>AssistiveTouch</b> Turn on AssistiveTouch. Use AssistiveTouch to access the Home Screen and all of your other apps.	17	<b>Pause/Resume Eye Gaze</b> Toggle button to temporarily pause and resume eye gaze.
16	<b>More</b> Access the Dashboard and Settings.	18	<b>See Through Mode</b> Make the TD Talk interface translucent and turn on the rear facing camera. This mode can be used to see what is in front of your device, such as your communication partner, while you use TD Talk.

### 3.2.1 How do I access the Off-Screen Menu in TD Talk?

There are two ways of accessing the Off-Screen Menu in TD Talk:

- **As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
- **As a caregiver:** Swipe up from the letter **T** on the TD Talk keyboard.

## 3.3 The Dashboard

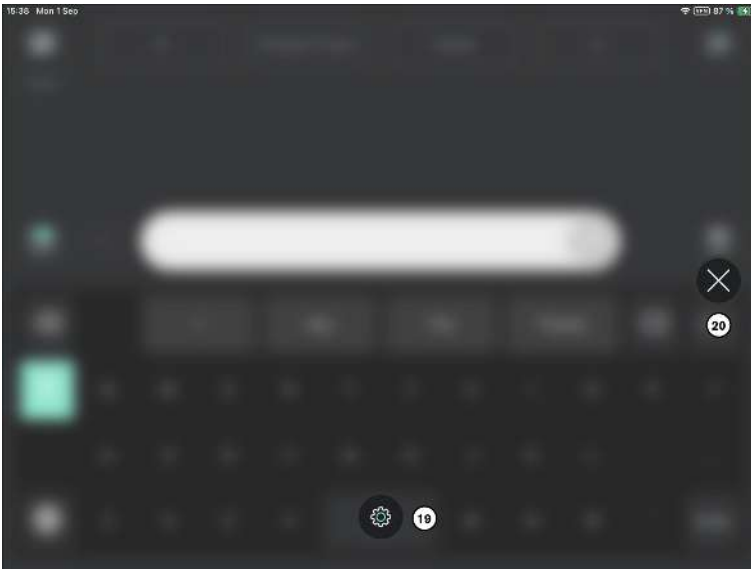



**Illustration 4:** TD Talk Dashboard

Position	Description	Position	Description
19	<b>TD Talk Settings</b> For more information, see <i>5 TD Talk Settings, page 19</i>	20	<b>Exit the Dashboard</b> Select the button to exit the Dashboard.

### 3.3.1 How do I access the Dashboard in TD Talk?


- Access the settings in one of the following ways:
  - As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - As a caregiver:** Swipe up from the letter T on the TD Talk keyboard.



The  (Dashboard) button will become visible in the lower middle part of the screen.

- Select the  (Dashboard) button to open the Dashboard.


### 3.3.2 How do I Exit and Close Down TD Talk?


- Access the settings in one of the following ways:
  - As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - As a caregiver:** Swipe up from the letter T on the TD Talk keyboard.



The  (AssistiveTouch) button will become visible in the lower middle part of the screen.



- Select the  (AssistiveTouch) button to open the AssistiveTouch Menu.
- Select the  (Home) button from the AssistiveTouch Menu.


## 3.4 Setting Up TD Talk


1. Select the  TD Talk app on the Home screen of the iPadOS device.
2. Access the settings in one of the following ways:
  - **As a user:** Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.
  - **As a caregiver:** Swipe up from the letter T on the TD Talk keyboard.


The  (Dashboard) button will become visible in the lower middle part of the screen.

3. Select the  (Dashboard) button to open the Dashboard.
4. Select the  (Settings) action to open the TD Talk Settings.
5. Set the TD Talk application language:

- Select the  (Next Screen) button on the right-hand side.
- Look for the  (General) icon to find the correct settings card.
- Select the **Open** button.
- Look in the **Application Language** section.
- Select the radio button next to the language you want the application to be in.



- Select the  (Back) button to go back to TD Talk Settings.
6. Select the voice(s) to use:


- Look for the  (Voice) icon to find the correct settings card.
- Select the **Open** button.
- Look in the **Your Voice/s** section.
- Select the radio button under the installed language you want to speak.

 If the wanted language is missing, see 5.3.4.1 *How do I Add/Download a New Language/Keyboard Language in TD Talk?*, page 27 and 5.3.4.2 *How do I select and activate the Keyboard Languages to use/be available in TD Talk?*, page 28.


- Select the radio button next to the voice to use in the list of voices available for the language to speak.
- Use the “Scroll up” and Scroll Down” buttons on the right to scroll through the list of voices available for the language.

 To try out a voice, select the  next to the voice. If available.

-  If there is a  (download) instead of a radio button, select the button to download the voice.
- Repeat the last two steps above if you want to be able to speak in more than one language.

7. Select the  (Close) button to exit TD Talk Settings.



In TD Talk you can quickly switch languages between the selected ones by selecting the  button in the lower left corner of the keyboard.

## 4 Using TD Talk

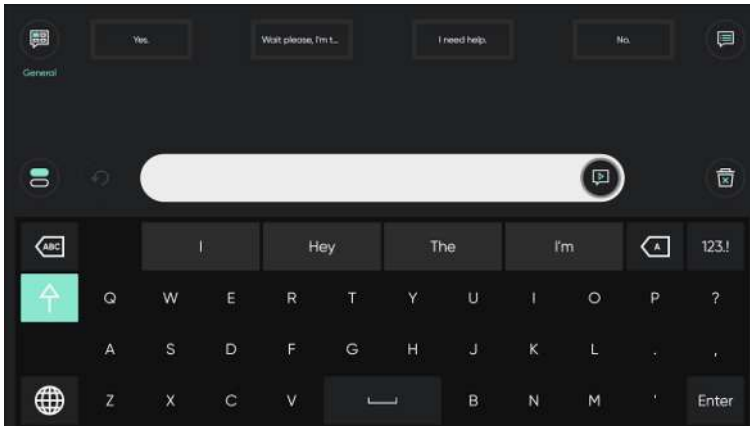


Illustration 5: TD Talk

### 4.1 Using the Keyboard

By selecting letters on the keyboard, words and sentences are formulated (or built) in the message window. Select the




(Speak) button when ready to speak.

### 4.2 Using Predictions

TD Talk offers both Word Prediction and Phrase Prediction which may decrease effort and/or increase speed of producing messages.

#### 4.2.1 Word Predictions

The word prediction engine used in TD Talk is from SwiftKey™ and is commonly used in tablets and smartphones. SwiftKey™ uses a blend of artificial intelligence technologies to predict the next word the user intends to type. SwiftKey™ learns from previous texts the user has written and offers predictions or presents predictions based on the current input text and what it has learned. The Word Prediction language will change depending on the Keyboard Language setting.

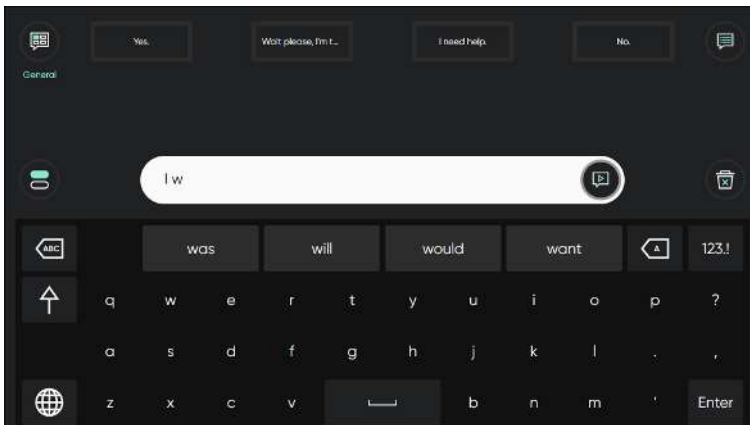


When Word Prediction is in use, four (4) boxes will appear above the Keyboard. The leftmost box will display the most probable predicted word, and the remaining three (3) boxes will contain one predicted word each. To select a predicted word, select the box with the correct word in it. If you can't find the word you are looking for, just keep on typing, and the word you are looking for may appear after the next character(s).

If you are using Word Prediction and want to type a word containing an accent, like café you will save time and effort by letting Word Prediction handle the accent. Simply type "cafe", and you will get a suggestion for "café".

### 4.2.2 Phrase Prediction


TD Talk learns and analyses how you speak over time. Word prediction learns the words you say. Phrase prediction learns how you put those words together forming a personal phrase set. When you start typing, predicted phrases from your phrase set will be offered in the Message Window in grey text.

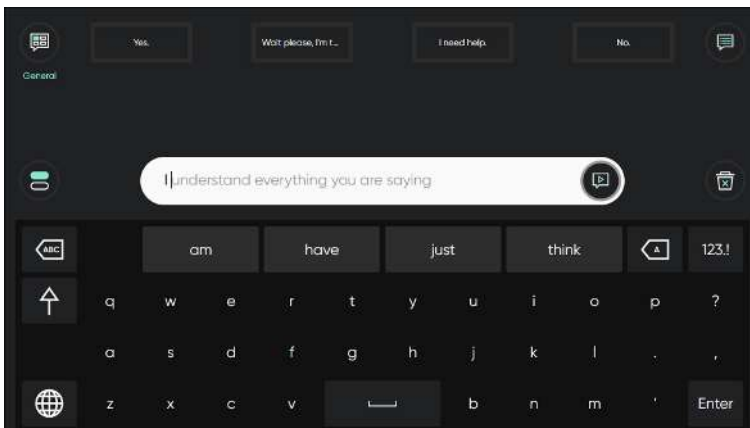

Look and fixate the gaze on the last word in the predicted phrase to select the phrase. The color on the phrase will change from grey to black indicating that it has been selected. it can now be spoken or edited.



## 4.3 Editing a Message

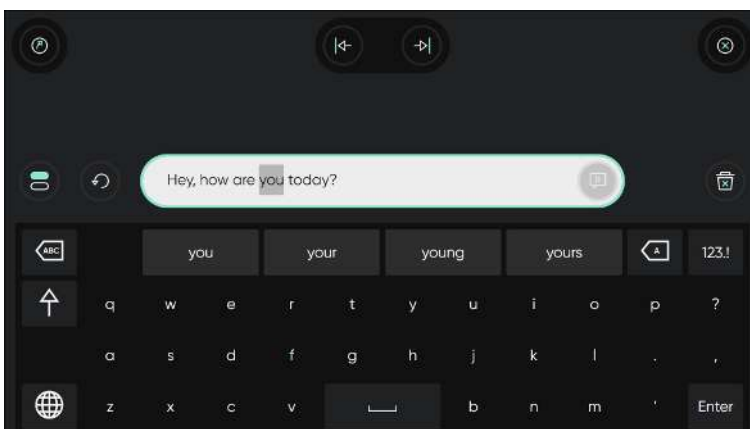
### 4.3.1 How do I Edit a Message in TD Talk?



**Illustration 6:** The Edit Menu

1. Gaze at the text in the Message Window to edit in the Message Window. The Edit Menu will open.
2. When a word is selected, type to replace the word, select a word from word prediction or select delete.

Use the  (Move Left) and  (Move Right) arrows if the cursor is in the wrong place.

3. Select the  (Close) button to exit Edit mode.

## 4.4 Copy and Paste

### 4.4.1 How do I Use Copy in TD Talk?



The copy function within TD Talk copies the entire Message Window.

1. Gaze at the text in the Message Window to copy the text in the Message Window.  
The Edit Menu will open.



Select the (Copy) button.

### 4.4.2 How do I Use Paste in TD Talk?

1. Gaze at the empty Message Window.  
The Copy Menu will open.



Select the (Paste) button.

The message will appear in the Message Window.



You can paste text copied from a text source in any other software on your device.



In iOS, a popup from Apple will show when pasting between applications. You can allow it every time with assistive touch OR change Apple Settings to always allow this. TD Talk will ask if you want to go to that Setting.

## 4.5 Using Quick Messages

### 4.5.1 How do I use Quick Messages in TD Talk?

There are two (2) types of Quick messages, the first ones are always visible on the Home screen. The second ones are



located under the (More Quick Messages) button.


#### Quick Messages on the Home Screen



1. Select the **Quick Message** to speak.  
The selected message will be spoken immediately without being added to the Message Window and interfering with any message you have in process.

#### More Quick Messages

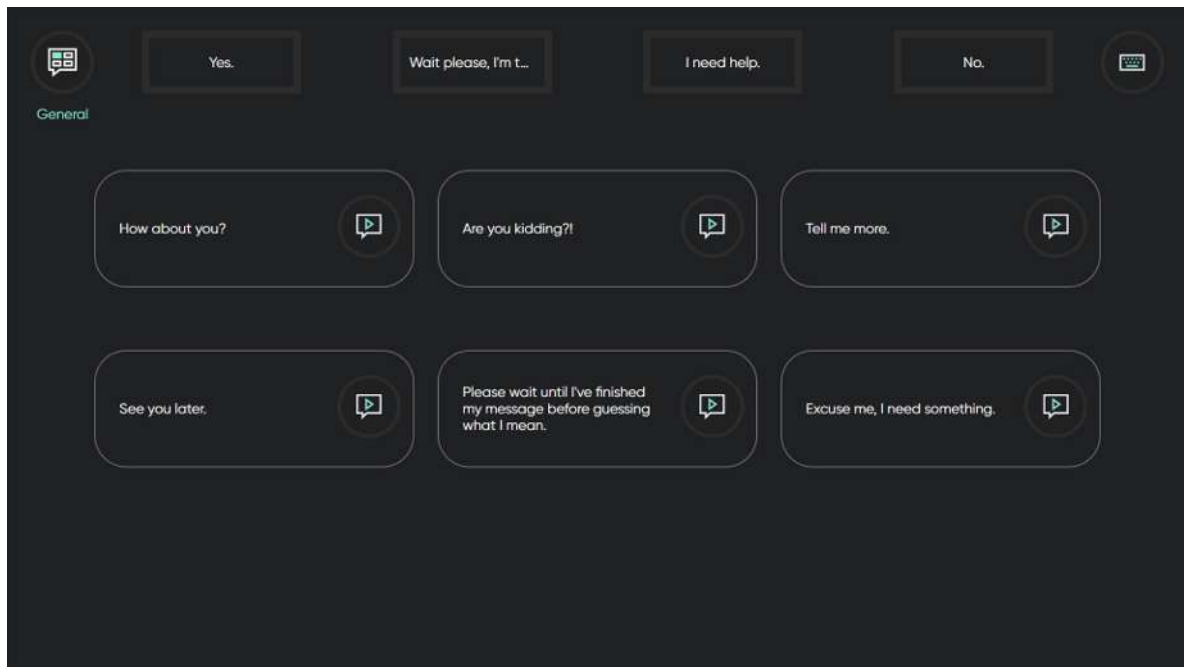


1. Select the (More Quick Messages) button.





2. Select the  (Speak) button next to the message.
3. Select the  (Keyboard) button to return to the Home screen.

#### 4.5.2 How do I Change the Topic for Quick Messages?

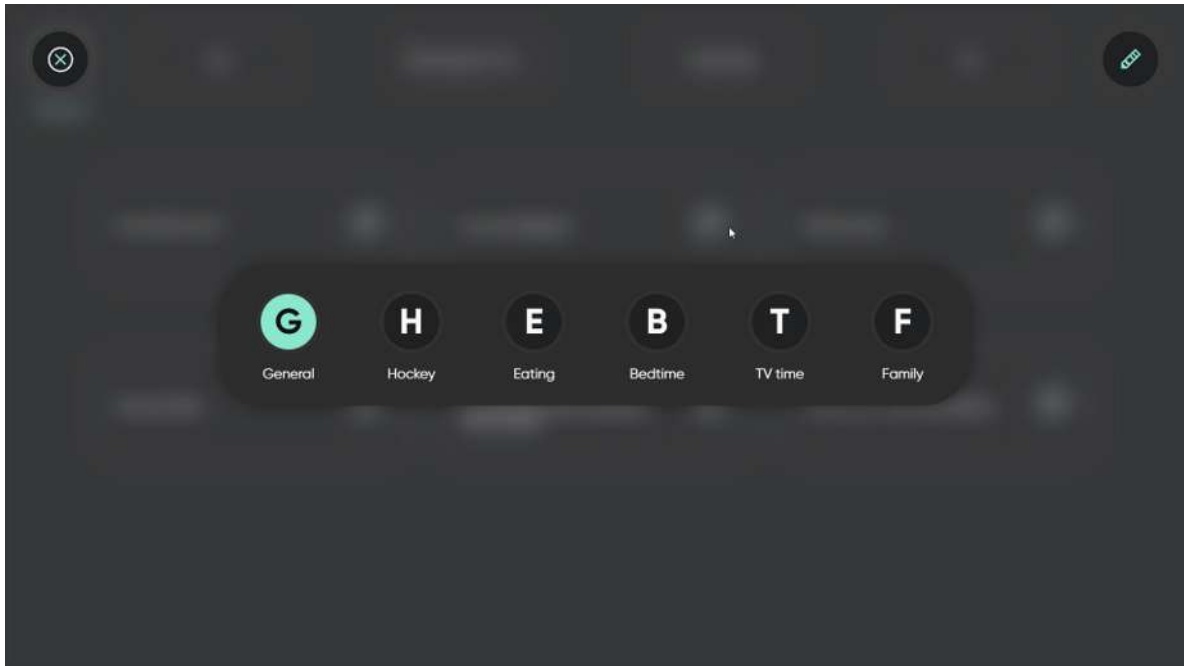

1. Select the  (Topics Menu) button.
2. Select the new topic.





 Select the  (Edit) button to change the Topics shown/made available in the Topics menu. For more information, see 5.4.1 Topics, page 30.



## 4.6 Changing the Language

### 4.6.1 How can I Switch Speaking Languages in TD Talk?

1. Select the  (Language) button on the keyboard.
2. Select the language to speak  
If the wanted language is missing, see 5.3.4.1 How do I Add/Download a New Language/Keyboard Language in TD Talk?, page 27
3. Select the  (Back) button to go back


## 4.7 Chat

  
Select the  (Chat) icon to switch to a second Message Window to write a new message without losing the text in the first Message Window.

  
Select the  (Chat return) icon to switch back to the first Message Window.



## 4.8 See Through Mode

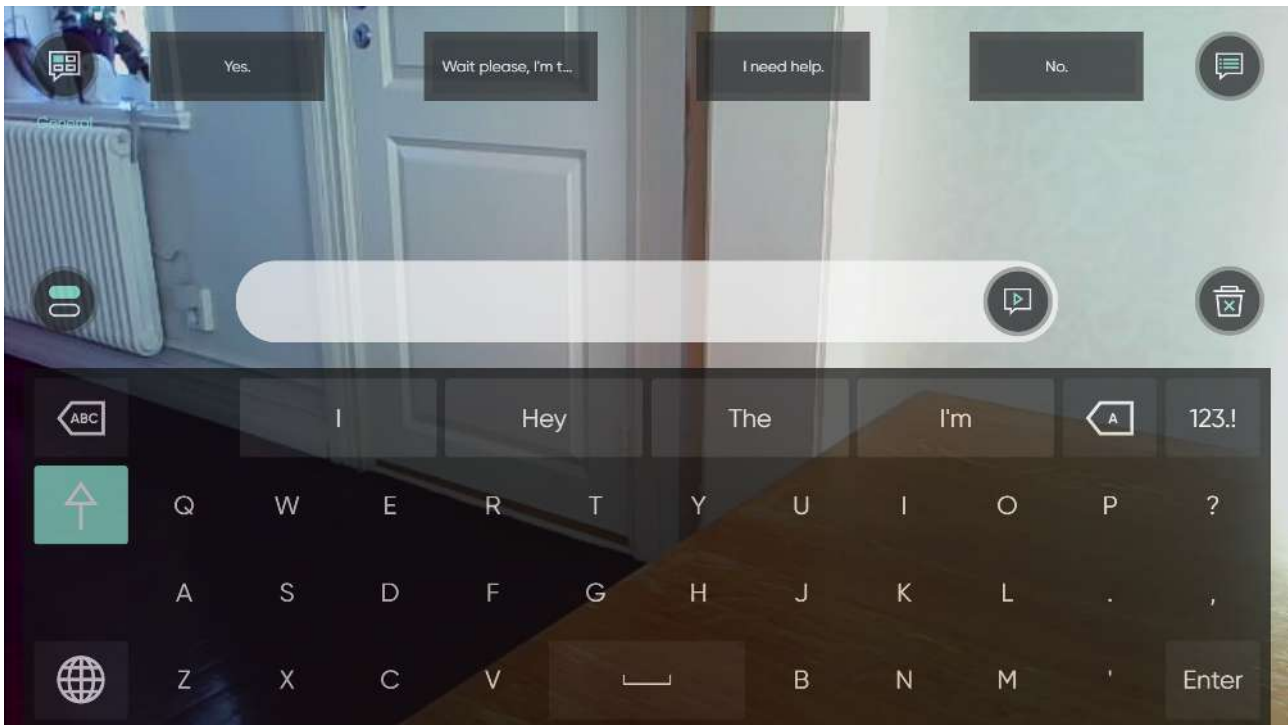

Make the TD Talk interface translucent and turn on the rear facing camera. This mode can be used to see what is in front of your device, such as your communication partner, while you use TD Talk.


### 4.8.1 How do I Start See Through Mode

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.  
The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (See Through Mode) button.


### 4.8.2 How do I End See Through Mode

1. Fixate your gaze below the screen, in the middle of the eye tracker, or the area of the eye tracker.  
The Off-Screen Menu will become visible in the lower middle part of the screen.

2. Select the  (End See Through Mode) button.


## 4.9 How do I Use Abbreviation Expansions in TD Talk?

When typing an existing abbreviation the first position in the word predictions will be changed to the Abbreviation Expansion.

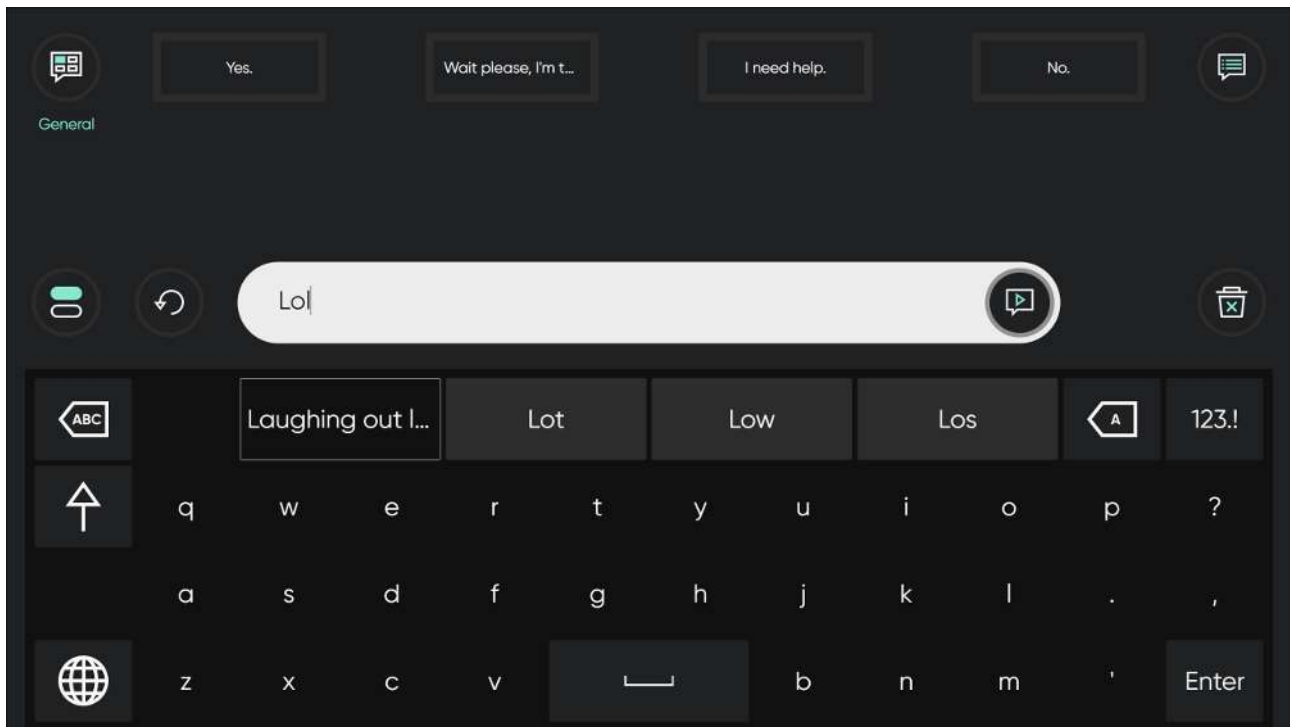




Select the Abbreviation Expansion in the word prediction area to expand the abbreviation into the message window. For more information about adding, editing and deleting abbreviation expansions, see [Abbreviation Expansions](#).

## 4.10 How do I Undo an Action in TD Phone?

Select the  (Undo) button next to the Message Window to undo actions made in the message window since you started typing in the active message window, one by one.
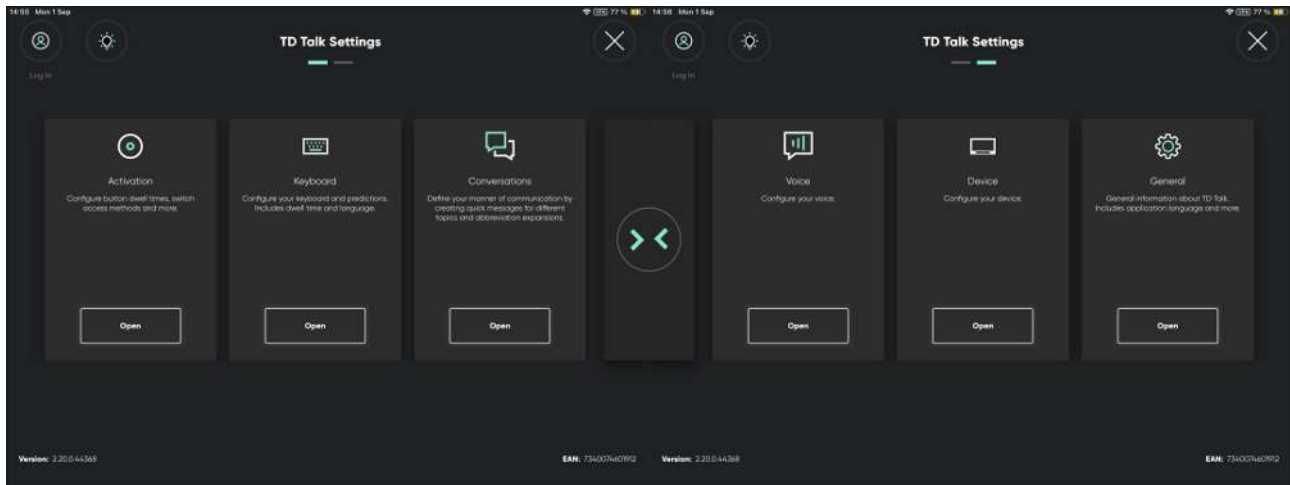
## 5 TD Talk Settings



Icons	Action	Description
	My Account	Opens the My Account page. For more information, see <i>5.1 My Account</i> , page 20.
	Release Notes	Opens the Release Notes for TD Talk. If TD Talk has been updated to a newer version, it will be indicated with a  in the top right corner of the icon.
	Activation	Only available if an Eye Tracker is installed and is activated. Opens the <b>Activation</b> settings. For more information, see <i>5.2 Activation</i> , page 21.
	Keyboard	Opens the <b>Keyboard</b> settings. For more information, see <i>5.3 Keyboard</i> , page 24.
	Conversations	Opens the <b>Conversations</b> settings. For more information, see <i>5.4 Conversations</i> , page 29.
	Voice	Opens the <b>Voice</b> Settings. For more information, see <i>5.5 Voice</i> , page 37.
	Device	 Only available on the TD Pilot. Opens the <b>Device</b> settings. For more information, see <i>5.6 Device</i> , page 44.
	General	Opens the <b>General</b> settings. For more information, see <i>5.7 General</i> , page 46.

## 5.1 My Account



Log in to your myTobiiDynavox Account to back up and restore your personal word predictions, pronunciations, phrase predictions, topics, and quick messages between your devices.

### 5.1.1 What is a myTobiiDynavox Account?

A myTobiiDynavox Account is an online platform provided by Tobii Dynavox, a company specializing in assistive technology for individuals with communication and mobility challenges. The myTobiiDynavox Account allows users to access various resources, including personalized data backup and restore, software updates, support materials, user forums, and online training courses. It also serves as a central hub for managing device settings, creating and sharing communication boards, and accessing additional features tailored to the needs of individuals using Tobii Dynavox devices. Overall, it's a central platform for users to manage their assistive technology solutions and find support within the Tobii Dynavox community.

### 5.1.2 How do I Create a myTobiiDynavox Account?

Go to [Create a myTobiiDynavox account](#) to create a myTobiiDynavox Account.

### 5.1.3 How do I Backup my Personalized Data?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.






4. Look for the  (Log in) icon in the top-left corner of the screen.
5. Use the keyboard to type in your user name in the **Username** field.
6. Use the keyboard to type in your password in the **Password** field.
7. Select the **Log in** button.
8. Select the **Backup** button.

The date the backup is made and from which device will show to the left of the Backup button when a backup is done.



9. Select the  (Close) button to exit .


#### 5.1.4 How do I Restore my Personalized Data?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The  (Dashboard) button will become visible in the lower middle part of the screen.




2. Select the  (Dashboard) button to open the Dashboard.




3. Select the  (Settings) action to open the TD Talk Settings.



4. Look for the  (Log in) icon in the top-left corner of the screen.
5. Use the keyboard to type in your user name in the **Username** field.
6. Use the keyboard to type in your password in the **Password** field.
7. Select the **Log in** button.
8. Select the **Restore** button.

The date the restore is made will show to the left of the Restore button when a restore is done.




9. Select the  (Close) button to exit .

## 5.2

### Activation




Only available if an Eye Tracker is installed and is activated.

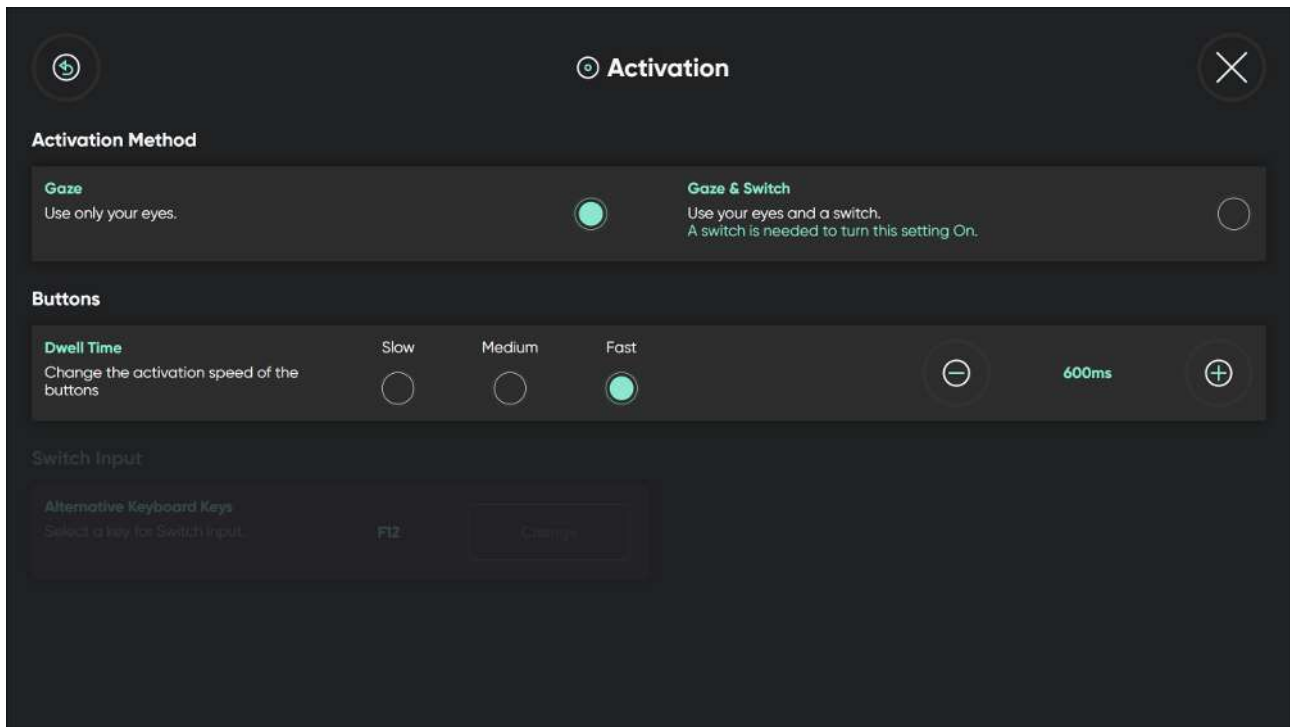



## 5.2.1 Activation Method


### 5.2.1.1 How do I Select my Activation Method in TD Talk?


 The selection of Activation Method for Activation does not refer to activation for the keyboard keys, but for all other activations in TD Talk.

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.


4. Look for the  (Activation) icon to find the correct settings card.


5. Select the **Open** button.

6. Look in the **Activation Method** section.

7. Select the radio button for the **Activation Method** to use from the following values:
  - **Gaze** - Use only your eyes (default)
  - **Gaze & Switch** - Use your eyes and a switch.

 A switch is needed to turn this setting On.

8. Select the  (Back) button to go back to TD Talk Settings.

9. Select the  (Close) button to exit TD Talk Settings.

## 5.2.2 Buttons

### 5.2.2.1 How do I Change the Dwell Time for Activating Buttons in TD Talk?



Buttons does not refer to those on the keyboard but the ones located in all other places in TD Talk.

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Activation) icon to find the correct settings card.

5. Select the **Open** button.

6. Look in the **Buttons** section.

7. Select the radio button for the **Dwell Time** for one of the following values:

- Slow
- **Medium** (default)
- Fast



8. Select the (Back) button to go back to TD Talk Settings.



9. Select the (Close) button to exit TD Talk Settings.

### 5.2.2.2 How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Activation) icon to find the correct settings card.

5. Select the **Open** button.

6. Look in the **Buttons** section.

7. Select one of the following buttons to fine tune the **Dwell Time**:



- — to lower the Dwell Time




- — to increase the Dwell Time

The value can be change between 200 ms — infinitive



8. Select the (Back) button to go back to TD Talk Settings.

9. Select the  (Close) button to exit TD Talk Settings.

### 5.2.3 Switch Input


Some switches requires that a keyboard key is set for switch input.





If a Switch input is set, that keyboard key will also act as a switch if selected on any connected external keyboard.


#### 5.2.3.1 How do I Set a Switch Input for my Switch in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Activation) icon to find the correct settings card.


5. Select the **Open** button.


6. Look in the **Switch Input** section.

7. Select the **Change** button.


8. Select the radio button for the keyboard key to be used as **Switch Input**.

9. Select the **Ok** button to apply, or the **Cancel** button to cancel.

10. Select the  (Back) button to go back to TD Talk Settings.

11. Select the  (Close) button to exit TD Talk Settings.

## 5.3 Keyboard

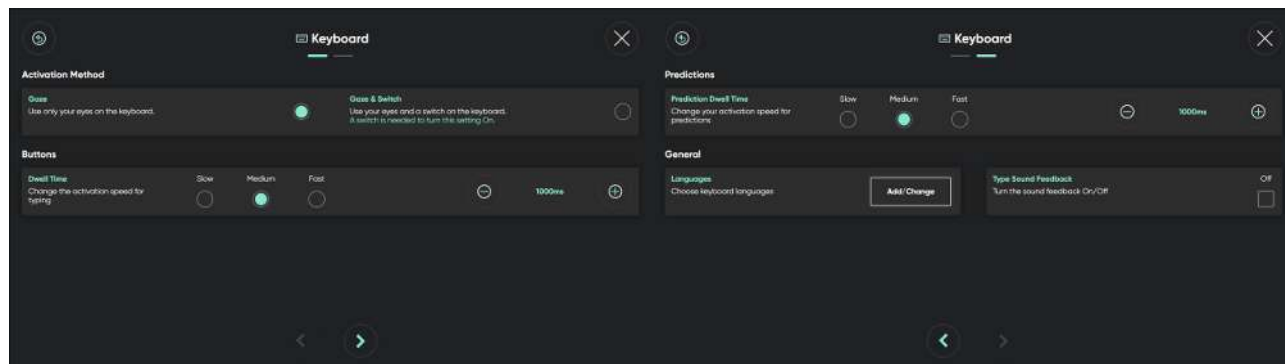



### 5.3.1 Activation Method

#### 5.3.1.1 How do I Select my Activation Method for the Keyboard in TD Talk?



The selection of Activation Method for Keyboard refers only to activating keyboard buttons in TD Talk.

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Keyboard) icon to find the correct settings card.

5. Select the **Open** button.

6. Look in the **Activation Method** section.

7. Select the radio button for the **Activation Method** to use from the following values:
  - **Gaze** - Use only your eyes (default)
  - Gaze & Switch - Use your eyes and a switch.



A switch is needed to turn this setting On.



8. Select the (Back) button to go back to TD Talk Settings.



9. Select the (Close) button to exit TD Talk Settings.

### 5.3.2 Buttons

#### 5.3.2.1 How do I Change the Dwell Time for Keyboard Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Keyboard) icon to find the correct settings card.


5. Select the **Open** button.

6. Look in the **Buttons** section.

7. Select the radio button for the **Dwell Time** for one of the following values:
  - Slow
  - **Medium** (default)
  - Fast





8. Select the (Back) button to go back to TD Talk Settings.


9. Select the  (Close) button to exit TD Talk Settings.


#### 5.3.2.2 How do I Fine Tune the Dwell Time for Activating Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Keyboard) icon to find the correct settings card.

5. Select the **Open** button.


6. Look in the **Buttons** section.


7. Select one of the following radio buttons to fine tune the **Dwell Time**:

–  — to lower the Dwell Time

–  — to increase the Dwell Time

The value can be change between 200 ms — infinitive


8. Select the  (Back) button to go back to TD Talk Settings.


9. Select the  (Close) button to exit TD Talk Settings.


### 5.3.3 Prediction


#### 5.3.3.1 How do I Change the Dwell Time for Prediction Buttons in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Keyboard) icon to find the correct settings card.


5. Select the **Open** button.


6. Select the  (Next page) button in the bottom of the screen.

7. Look in the **Prediction** section.

8. Select the radio button for the **Dwell Time** for one of the following values:
  - Slow
  - **Medium** (default)


– Fast


9. Select the  (Back) button to go back to TD Talk Settings.


10. Select the  (Close) button to exit TD Talk Settings.


#### 5.3.3.2 How do I Fine Tune the Dwell Time for Prediction Buttons in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.


4. Look for the  (Keyboard) icon to find the correct settings card.  
5. Select the **Open** button.

6. Select the  (Next page) button in the bottom of the screen.

7. Look in the **Prediction** section.


8. Select one of the following radio buttons to fine tune the **Dwell Time**:

–  — to lower the Dwell Time

–  — to increase the Dwell Time

The value can be change between 200 ms — infinitive


9. Select the  (Back) button to go back to TD Talk Settings.


10. Select the  (Close) button to exit TD Talk Settings.


#### 5.3.4 General


##### 5.3.4.1 How do I Add/Download a New Language/Keyboard Language in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Keyboard) icon to find the correct settings card.  
5. Select the **Open** button.

6. Select the  (Next page) button in the bottom of the screen.
7. Look in the **General** section.
8. Select the “Choose keyboard language” **Add/Change** button.
9. Select the **Download** tab.


10. Select the  (Download) button next to the language to start the download of the new keyboard.
11. Select the **Download** button to start the download or the **Cancel** button to cancel the download.
12. Select the **Installed** tab.
13. Select the radio button next to the languages to activate the language.


14. Select the  (Back) button to go back to TD Talk Settings.


15. Select the  (Close) button to exit TD Talk Settings.


#### 5.3.4.2 How do I select and activate the Keyboard Languages to use/be available in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.

6. Select the  (Next page) button in the bottom of the screen.
7. Select the “Choose keyboard language” **Add/Change** button.
8. Select the **Installed** tab.
9. Select the radio button next to the languages to activate the keyboard language for that languages.


 Make sure to select the radio button for all the languages that you want a voice available for.


10. Select the  (Back) button to go back to TD Talk Settings.


11. Select the  (Close) button to exit TD Talk Settings.

#### 5.3.4.3 How do I Turn On/Off Type Sound Feedback in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.



4. Look for the  (Keyboard) icon to find the correct settings card.
5. Select the **Open** button.




6. Select the  (Next page) button in the bottom of the screen.
7. Look in the **General** section.
8. Toggle the **On/Off** button for "**Type Sound Feedback**" to:
  - On
  - Off




9. Select the  (Back) button to go back to TD Talk Settings.




10. Select the  (Close) button to exit TD Talk Settings.

## 5.4 Conversations

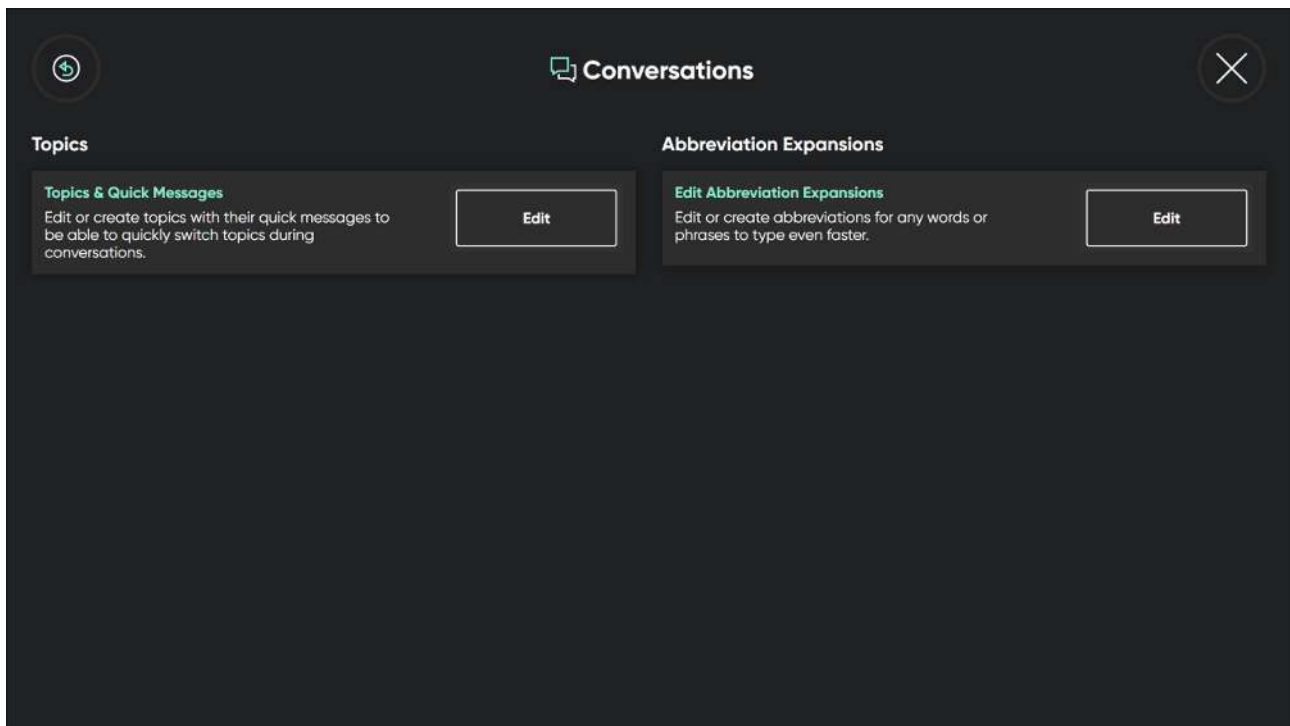

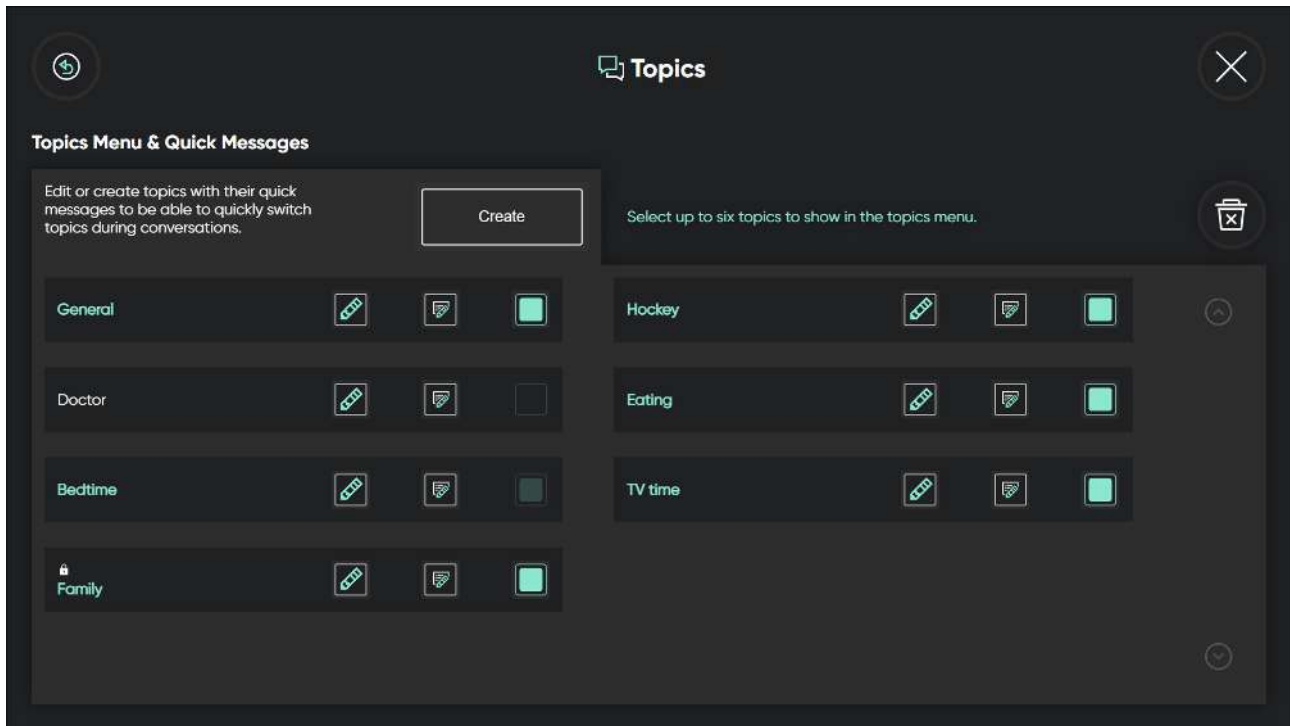

### 5.4.1 Topics





#### 5.4.1.1 How can I Create a New Topic in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.


4. Look for the  (Conversation) icon to find the correct settings card.

5. Select the **Open** button.


6. Select the **Edit** button for the Topics & Quick Messages in the Topics section.

7. Select the **Create** button.

8. Use the keyboard to name the new topic.


 Select the **Privacy Mode** checkbox to turn off Phrase prediction and learning.





9. Select the  (Save) icon to save.

10. Select the  (Close) button to exit .



#### 5.4.1.2 How can I Change the Name of a Topic in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversation) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for the Topics & Quick Messages in the Topics section.
7. Select the  (Edit) button next to the Topic name.
8. Use the keyboard to change name for the topic.





 Select the **Privacy Mode** checkbox to turn off Phrase prediction and learning.



9. Select the  (Save) icon to save.
10. Select the  (Close) button to exit .

#### 5.4.1.3 How can I Make the Topic Private in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.




2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversation) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for the Topics & Quick Messages in the Topics section.
7. Select the  (Edit) button next to the Topic name.
8. Select the **Privacy Mode** checkbox to turn off Phrase prediction and learning.

9. Select the  (Save) icon to save.
10. Select the  (Close) button to exit .

#### 5.4.1.4 How can I Select the Active Topics TD Talk?



1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversations) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for Topics & Quick Messages in the Topics section.
7. Select the **Checkbox** for the Topic that shall be active.












Up to six (6) topics can be selected and available in the topics menu.

8. Select the  (Close) button to leave the editing view.
9. Select the  (Keyboard) button to return to the Home screen.

#### 5.4.1.5 How can I Delete a Topic in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.









2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversations) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for Topics & Quick Messages in the Topics section.
7. Select the  (Delete) button.
8. Select the  (Delete) button next to the name of the topic you want to delete.
9. Select the **Delete** button to delete the topic or the **Cancel** button to cancel.
10. Select the  (Done) button when done deleting.
11. Select the  (Close) button to leave the editing view.
12. Select the  (Keyboard) button to return to the Home screen.

#### 5.4.1.6 How can I Edit the Quick Messages in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.










2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversations) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for Topics & Quick Messages in the Topics section.
7. Select the  (Edit Quick Messages) button.
8. Select the  (Edit) button next to the message to edit.
9. Make the changes to the message with the keyboard.
10. Select the  (Save) button to save the changes to the Quick Message.
11. Select the  (Close) button to leave the editing view.
12. Select the  (Keyboard) button to return to the Home screen.



#### 5.4.1.7 How can I Add a Quick Message in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Look for the  (Conversations) icon to find the correct settings card.
5. Select the **Open** button.
6. Select the **Edit** button for Topics & Quick Messages in the Topics section.
7. Select the  (Edit Quick Messages) button.
8. Select the  (Add) button next to the Add New location.  
You can have up to three (3) pages of Quick Messages for each Topic.

Use the  and  buttons to navigate between the pages.

9. Write the new message with the keyboard.
10. Select the  (Save) button to save the new Quick Message.
11. Select the  (Close) button to leave the editing view.



12. Select the (Keyboard) button to return to the Home screen.

#### 5.4.1.8 How can I Delete a Quick Message in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The (Dashboard) button will become visible in the lower middle part of the screen.



2. Select the (Dashboard) button to open the Dashboard.



3. Select the (Settings) action to open the TD Talk Settings.



4. Look for the (Conversations) icon to find the correct settings card.

5. Select the **Open** button.

6. Select the **Edit** button for Topics & Quick Messages in the Topics section.



7. Select the (Edit Quick Messages) button.



8. Select the (Edit) button next to the message to delete.



9. Select the Delete Message button.



10. Select the (Save) button to save the changes to the Quick Message.




11. Select the (Close) button to leave the editing view.




12. Select the (Keyboard) button to return to the Home screen.


## 5.4.2 Abbreviation Expansions

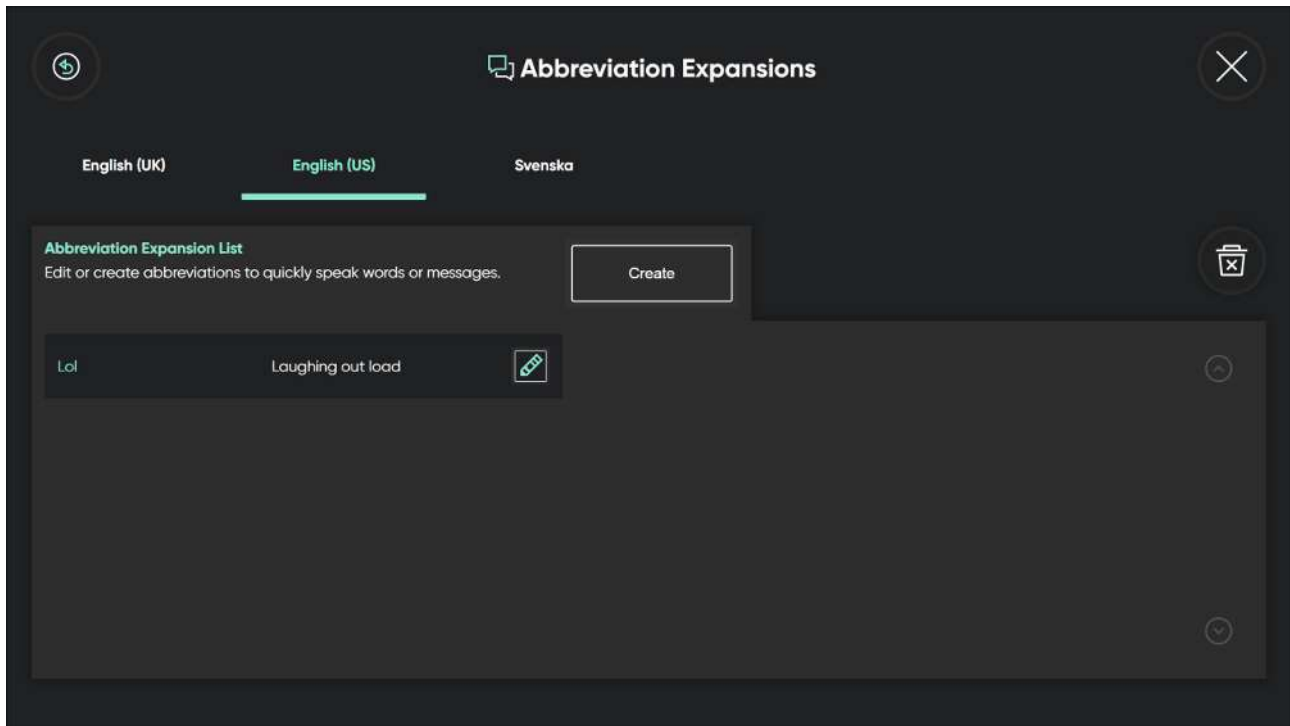



### 5.4.2.1 How do I Add an Abbreviation Expansion in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Conversations) icon to find the correct settings card.


5. Select the **Open** button.

6. Select the **Edit** button in the Edit Abbreviation Expansions area under Abbreviation Expansions.


7. Select the **language** tab for the installed language you want to add an abbreviation expansion for.


8. Select the **Create** button.


9. Write in the **Abbreviation** text box the abbreviation you want to create.


10. Select the  icon next to the **Word Or Message** text box.

11. Write in the **Word Or Message** text box the word or message you have created the abbreviation for.

12. Select the  (Save) button to save the changes to the Abbreviation Expansion.


13. Select the  (Back) button to go back to Conversations Settings.


14. Select the  (Back) button to go back to TD Talk Settings.


15. Select the  (Close) button to exit TD Talk Settings.


#### 5.4.2.2 How do I Edit an Abbreviation Expansion in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Conversations) icon to find the correct settings card.


5. Select the **Open** button.

6. Select the **Edit** button in the Edit Abbreviation Expansions area under Abbreviation Expansions.


7. Select the **language** tab for the installed language you want to edit an abbreviation expansion for.


8. Select the  icon next to the Abbreviation you want to edit.


9. Edit in the **Abbreviation** text box the new abbreviation you want to create.


10. Select the  icon next to the **Word Or Message** text box.

11. Edit in the **Word Or Message** text box the new word or message you have created the abbreviation for.

12. Select the  (Save) button to save the changes to the Abbreviation Expansion.


13. Select the  (Back) button to go back to Conversations Settings.


14. Select the  (Back) button to go back to TD Talk Settings.


15. Select the  (Close) button to exit TD Talk Settings.


#### 5.4.2.3 How do I Delete an Abbreviation Expansion in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.







3. Select the  (Settings) action to open the TD Talk Settings.

4. Look for the  (Conversations) icon to find the correct settings card.

5. Select the **Open** button.


6. Select the **Edit** button in the Edit Abbreviation Expansions area under Abbreviation Expansions.


7. Select the **language** tab for the installed language you want to delete an abbreviation expansion for.

8. Select the  (Delete) button.
9. Select the  (Delete) button next to the abbreviation that you want to delete.
10. Select the **Delete** button to confirm, or the **Cancel** button to cancel, deleting the abbreviation.
11. Select the  (Done) button when done deleting.
12. Select the  (Back) button to go back to Conversations Settings.
13. Select the  (Back) button to go back to TD Talk Settings.
14. Select the  (Close) button to exit TD Talk Settings.

## 5.5 Voice

Set the speaking voice of your keyboard languages here

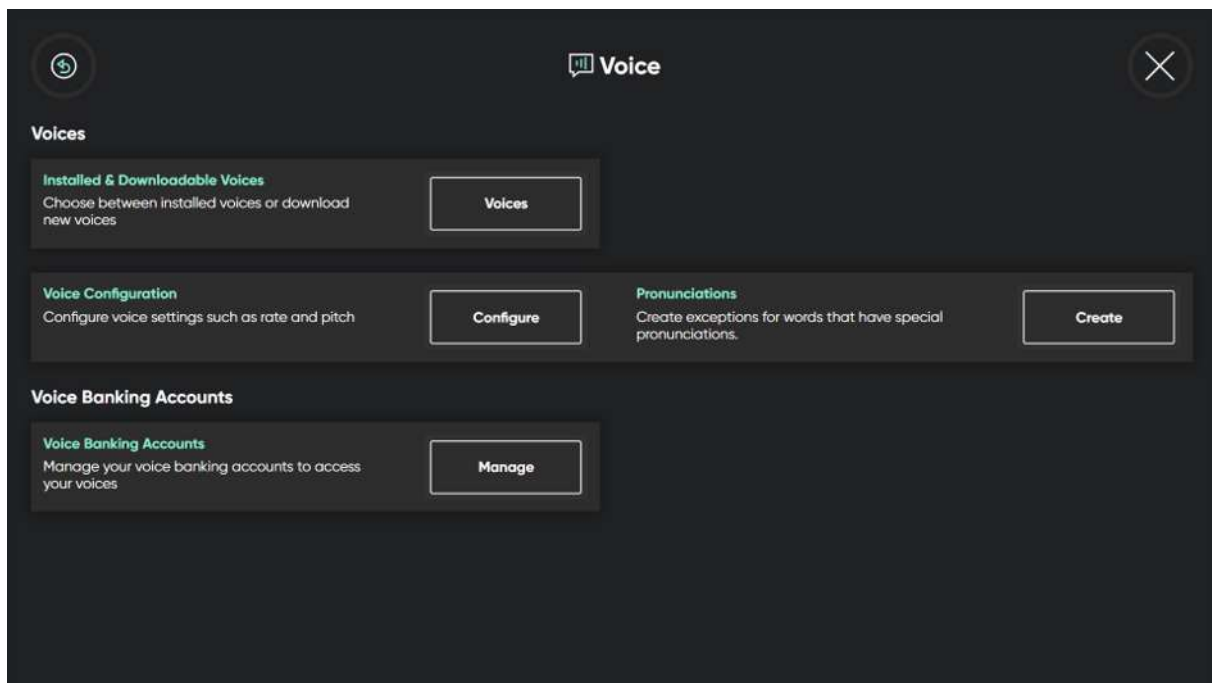

All languages installed are shown in the top part of the screen. When the language has been selected, the available voices for that language will show in the list beneath the languages. To try out a voice, select the  next to the voice to try out the voice. If available.

## 5.5.1 Voices


### 5.5.1.1 Installed & Downloadable Voices


There are a lot of voices available within TD Talk, especially for some languages.


Filters can be applied to make it easier for the user to find their voice. For more information, see [5.5.1.1.3 How can I Apply a Filter to the Available Voices in TD Talk?](#), page 39


#### 5.5.1.1.1 How do I Download a New Voice in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Select the  (Next Screen) button on the right-hand side.

5. Look for the  (Voice) icon to find the correct settings card.


6. Select the **Open** button.


7. Look in the **Voices** section



8. Select the **Voices** button for Installed and Downloadable Voices.

9. Select the **Download** tab.

10. Select the **language** tab for the installed language you want to speak.

 If the wanted language is missing, see [5.3.4.1 How do I Add/Download a New Language/Keyboard Language in TD Talk?](#), page 27 and [5.3.4.2 How do I select and activate the Keyboard Languages to use/be available in TD Talk?](#), page 28.


11. Select the  (Download) button next to the voice to start the download of the new voice.  
Use the "Scroll up" and Scroll Down" buttons on the right to scroll through the list of voices available for the language.

 To try out a voice, select the  next to the voice.

12. Select the **Download** button to start the download or the **Cancel** button to cancel the download.


13. Select the  (Back) button to go back to Voice Settings.





14. Select the  (Back) button to go back to TD Talk Settings.

15. Select the  (Close) button to exit TD Talk Settings.

#### 5.5.1.1.2 How do I Select my Voice in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.


2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Select the  (Next Screen) button on the right-hand side.
5. Look for the  (Voice) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Voices** section
8. Select the **Voices** button for Installed and Downloadable Voices.
9. Select the **language** tab for the installed language you want to speak.






If the wanted language is missing, see [5.3.4.1 How do I Add/Download a New Language/Keyboard Language in TD Talk?](#), page 27 and [5.3.4.2 How do I select and activate the Keyboard Languages to use/be available in TD Talk?](#), page 28.

10. Select the voice to use in the list of voices available for the language to speak.  
Use the “Scroll up” and “Scroll Down” buttons on the right to scroll through the list of voices available for the language.




To try out a voice, select the  next to the voice.


11. Select the  (Back) button to go back to Voice Settings.
12. Select the  (Back) button to go back to TD Talk Settings.
13. Select the  (Close) button to exit TD Talk Settings.

#### 5.5.1.1.3 How can I Apply a Filter to the Available Voices in TD Talk?

On the **Installed** or **Download** tab, do the following:


1. Select the  (Filter) button.
2. Select the check box(es) next to the option(s) you want to use.  
You can choose:
  - Gender - Female, Male
  - Age - Child, Teen and Adult


### 5.5.1.2 Voice Configuration

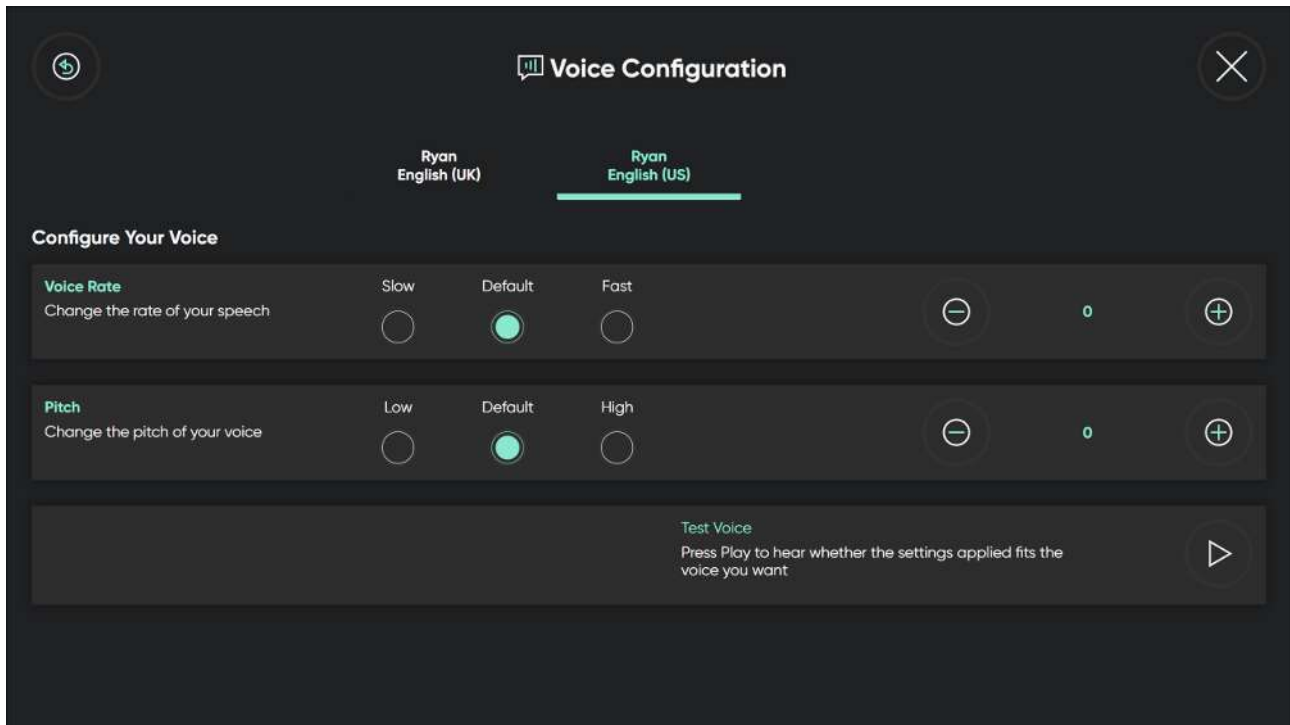



#### 5.5.1.2.1 How do I Make Changes to my Voice in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Select the  (Next Screen) button on the right-hand side.

5. Look for the  (Voice) icon to find the correct settings card.

6. Select the **Open** button.

7. Look in the **Voices** section


8. Select the **Configure** button for Voice Configuration.

9. Select the voice to change from the available voices.

10. Select the radio button for the **Voice Rate** for one of the following values:
  - Slow
  - **Default**
  - Fast

to change the rate of your speech.

11. Select one of the following radio buttons to fine tune the **Voice Rate**:

–  — to lower the Voice Rate





— to increase the Voice Rate

The value can be change between -10 — +10.

12. Select the radio button for the **Pitch** for one of the following values:

- Low
- **Default**
- High

to change the pitch of your speech.



To listen the new voice, select the  in the Test Voice section.

13. Select one of the following radio buttons to fine tune the **Pitch**:



— to lower the Pitch



— to increase the Pitch

The value can be change between -10 — +10.



To listen the new voice, select the  in the Test Voice section.



14. Select the (Back) button to go back to Voice Settings.




15. Select the (Back) button to go back to TD Talk Settings.




16. Select the (Close) button to exit TD Talk Settings.


### 5.5.1.3 Pronunciations

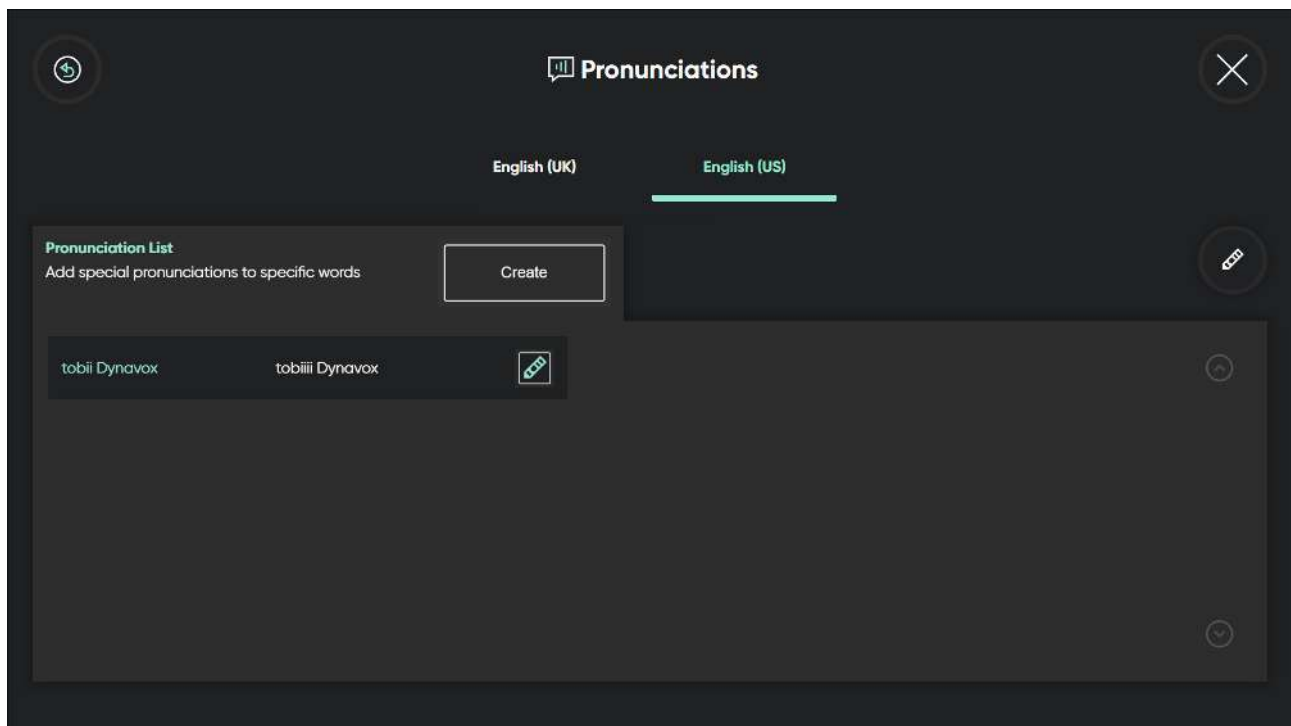



#### 5.5.1.3.1 How do I Create Pronunciation Exceptions in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Select the  (Next Screen) button on the right-hand side.

5. Look for the  (Voice) icon to find the correct settings card.

6. Select the **Open** button.


7. Look in the **Voices** section

8. Select the **Create** button for Pronunciations.



9. Select the **language** tab for the installed language you want to add a pronunciation exception for.

10. Select the **Create** button.

11. Write in the **Word** text box the word you want to create a pronunciation exception for.


12. Select the  icon next to the **Pronunciation** text box.


13. Write in the **Pronunciation** text box how you want the word to be pronounced.

 To listen to the Pronunciation exception, select the  next to the Pronunciation text box..

14. Select the **Create** button to save your Pronunciation exception.




15. Select the  (Back) button to go back to Voice Settings.

16. Select the  (Back) button to go back to TD Talk Settings.

17. Select the  (Close) button to exit TD Talk Settings.

#### 5.5.2 Voice Banking Accounts


TD Talk can connect to three different Voice Banking Services:

- Acapela MyOwnVoice 
- ModelTalker  ModelTalker
- The Voice Keeper 


The user will need an account with and the login credentials for one of these providers of voice banking services to be able to connect and use the personal synthetic voice.


### 5.5.2.1 Voice Banking Accounts


#### 5.5.2.1.1 How can I use my Voice from my Voice Banking Service?


 Internet access is needed for log in to the Voice Banking Service.


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.

4. Select the  (Next Screen) button on the right-hand side.

5. Look for the  (Voice) icon to find the correct settings card.

6. Select the **Open** button.

7. Look in the **Voice Banking Accounts Section**.


Select the **Manage** button for Voice Banking Accounts.

8. Select the **Log in** button in the section for your Voice Banking Service.

9. Enter the **User name** and **Password**.



10. Select the **Log In** button.

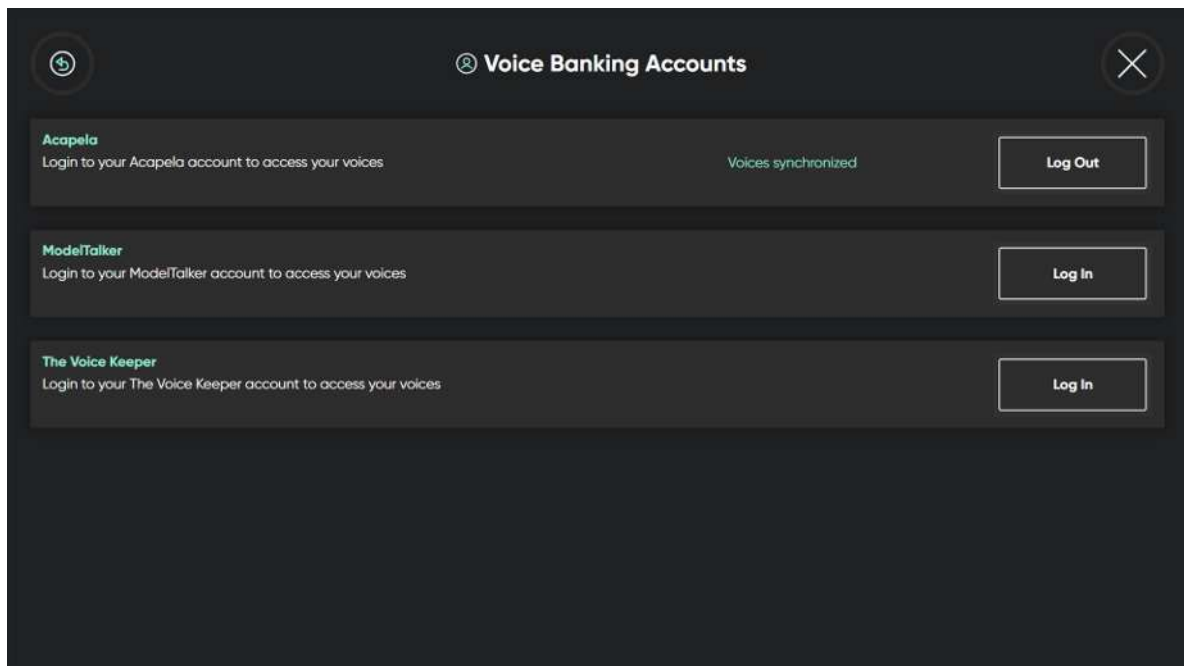The personal voice will be downloaded and automatically selected as the default voice for the user.




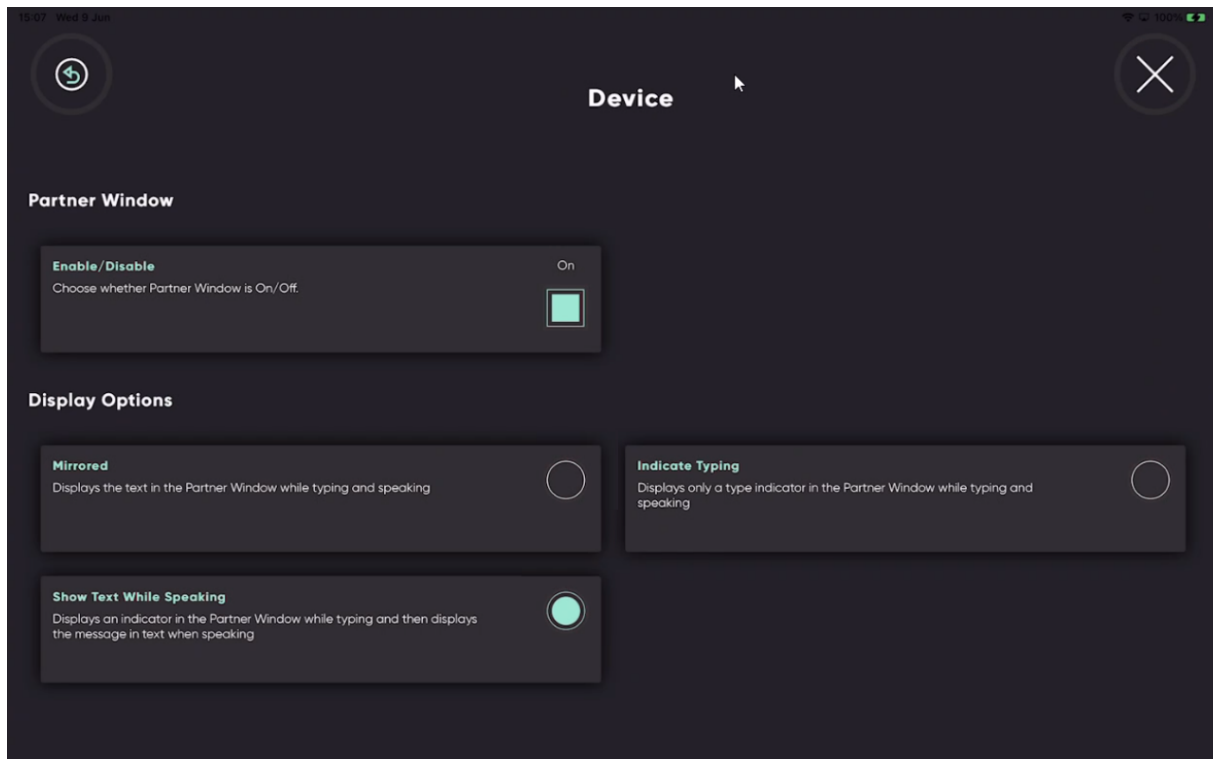
 This will be the case when personal synthetic voices in multiple languages are available

11. Select the  (Back) button to go back to Voice Settings.

12. Select the  (Back) button to go back to TD Talk Settings.
13. Select the  (Close) button to exit TD Talk Settings.

## 5.6 Device


 Only available on the TD Pilot.






### 5.6.1 Partner Window


#### 5.6.1.1 How do I Enable/Disable the Partner Window in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Select the  (Next Screen) button on the right-hand side.




5. Look for the  (Device) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Partner Window** section.
8. Toggle the **Enable/Disable** button for “**Choose whether Partner Window is On/Off**” to:
  - On
  - Off



9. Select the  (Back) button to go back to TD Talk Settings.



10. Select the  (Close) button to exit TD Talk Settings.


#### 5.6.1.2 How do I Set the Display Options for the Partner Window in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The  (Dashboard) button will become visible in the lower middle part of the screen.




2. Select the  (Dashboard) button to open the Dashboard.




3. Select the  (Settings) action to open the TD Talk Settings.



4. Select the  (Next Screen) button on the right-hand side.




5. Look for the  (Device) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Display Options** section.
8. Select one of the following radio buttons choose the Display Option for the Partner Window:
  - Mirrored — Displays the text in the Partner Window while typing and speaking.
  - Show Text While Speaking — Displays an indicator in the Partner Window while typing and the displays the message in text when speaking.
  - Indicate Typing — Displays only a type indicator in the Partner Window while typing and speaking.




9. Select the  (Back) button to go back to TD Talk Settings.



10. Select the  (Close) button to exit TD Talk Settings.


## 5.7 General

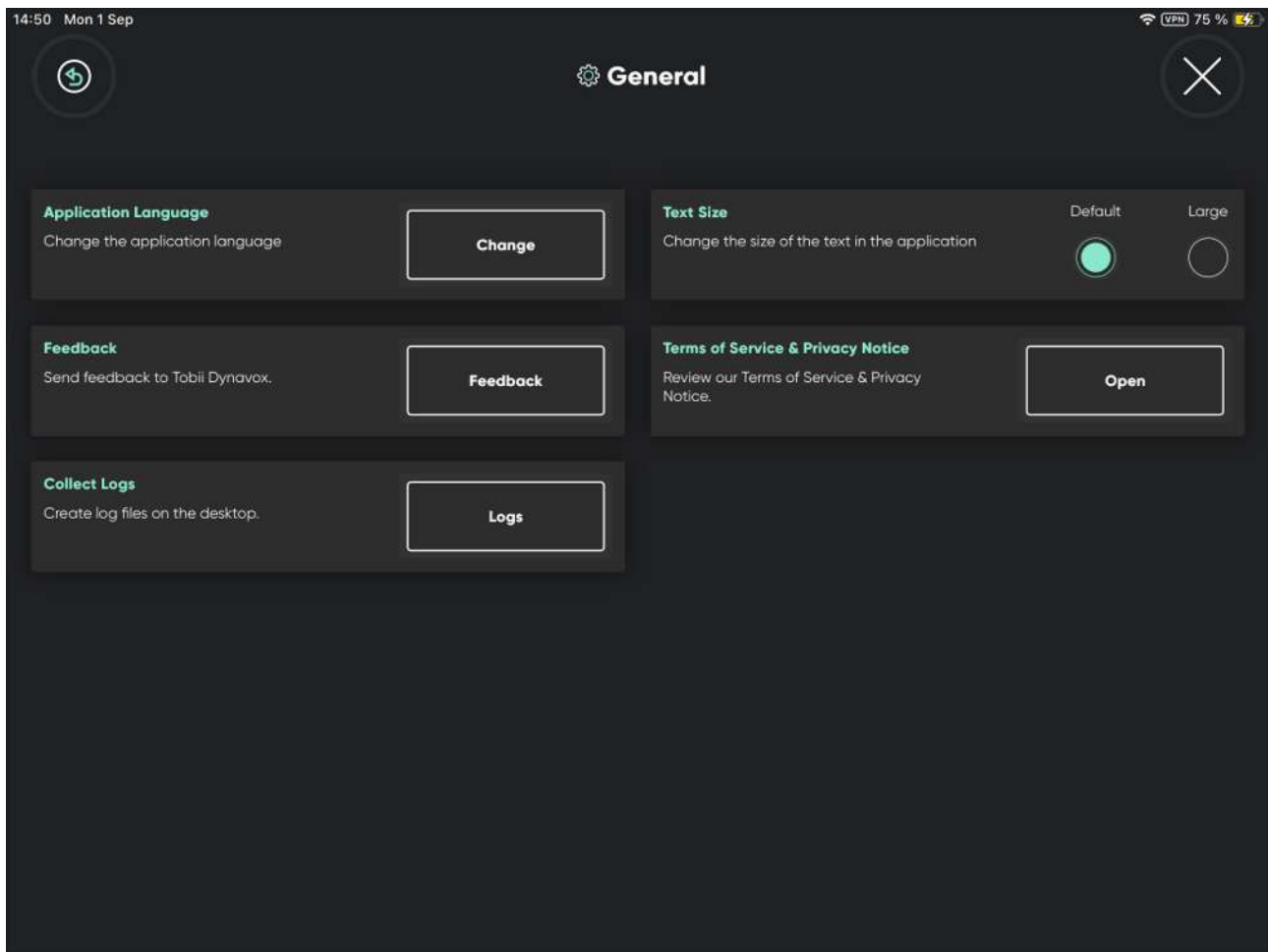



### 5.7.1 Application Language


#### 5.7.1.1 How do I set my Application language in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.


4. Select the  (Next Screen) button on the right-hand side.


5. Look for the  (General) icon to find the correct settings card.

6. Select the **Open** button.

7. Look in the **Application Language** section.

8. Select the **Change** button.
9. Select the radio button next to the language you want the application to be in.


10. Select the  (Back) button to go back to TD Talk Settings.


11. Select the  (Close) button to exit TD Talk Settings.


## 5.7.2 Text Size


### 5.7.2.1 How do I Change the Text Size in TD Talk?


1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.

3. Select the  (Settings) action to open the TD Talk Settings.


4. Select the  (Next Screen) button on the right-hand side.


5. Look for the  (General) icon to find the correct settings card.

6. Select the **Open** button.

7. Look in the **Text Size** section.

8. Select the radio button for the **Text Size** for one of the following values:
  - **Default** (default)
  - Large


9. Select the  (Back) button to go back to TD Talk Settings.


10. Select the  (Close) button to exit TD Talk Settings.


## 5.7.3 Feedback


### 5.7.3.1 How do I Send Feedback to Tobii Dynavox in TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.


The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.


3. Select the  (Settings) action to open the TD Talk Settings.

4. Select the  (Next Screen) button on the right-hand side.




5. Look for the  (General) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Feedback** section.
8. Select the **Feedback** button.
9. Use the **Keyboard** to type your email address.




10. Select the  button for the Subject text field.
11. Use the **Keyboard** to fill in the subject for the email.



12. Select the  button for the Message text field.
13. Use the **Keyboard** to message.




14. Select the  button to close the Message text field.
15. Select the **Send Feedback** button to send the feedback to Tobii Dynavox.



16. Select the  (Back) button to go back to TD Talk Settings.



17. Select the  (Close) button to exit TD Talk Settings.

## 5.7.4 Terms of Service & Privacy Notice


### 5.7.4.1 How do I Open the Terms of Service

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.



The  (Dashboard) button will become visible in the lower middle part of the screen.



2. Select the  (Dashboard) button to open the Dashboard.




3. Select the  (Settings) action to open the TD Talk Settings.



4. Select the  (Next Screen) button on the right-hand side.




5. Look for the  (General) icon to find the correct settings card.
6. Select the **Open** button.
7. Look in the **Terms of Service & Privacy Notice** section.
8. Select the **Open** button.



9. Select the  (Back) button to go back to TD Talk Settings.












10. Select the  (Close) button to exit TD Talk Settings.

### 5.7.4.2 How do I Open the Privacy Notice

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.




The  (Dashboard) button will become visible in the lower middle part of the screen.





2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Select the  (Next Screen) button on the right-hand side.
5. Select the  (Next Screen) button on the right-hand side.
6. Look for the  (General) icon to find the correct settings card.
7. Select the **Open** button.
8. Look in the **Terms of Service & Privacy Notice** section.
9. Select the **Open** button.
10. Select the  (Forward) button.
11. Select the  (Back) button to go back to TD Talk Settings.
12. Select the  (Close) button to exit TD Talk Settings.

## 5.7.5 Collect Logs



### 5.7.5.1 How do I Collect Logs for TD Talk?

1. Fixate your gaze below the screen, in the middle of the eye tracker or the area of the eye tracker.

The  (Dashboard) button will become visible in the lower middle part of the screen.

2. Select the  (Dashboard) button to open the Dashboard.
3. Select the  (Settings) action to open the TD Talk Settings.
4. Select the  (Next Screen) button on the right-hand side.
5. Look for the  (General) icon to find the correct settings card.
6. Select the **Open** button.  
The settings page for General will open.
7. Look in the **Collect Logs** section.
8. Select the **Log** button.
9. Select the **Yes** button

 Logs will be created on the Desktop.

10. Select the  (Back) button to go back to TD Talk Settings.
11. Select the  (Close) button to exit TD Talk Settings.

Copyright © Dynavox Group AB (Publ). Not all products and services offered in each local market. Specifications are subject to change without prior notice. All trademarks are the property of their respective owners.

## Support for Your Tobii Dynavox Device

### Get Help Online

See the product-specific Support page for your Tobii Dynavox device. It contains up-to-date information about issues and tips & tricks related to the product.  
Find our Support pages online at: <https://www.tobiidynavox.com/pages/product-support>

### Contact Your Solution Consultant or Reseller

For questions or problems with your product, contact your Tobii Dynavox solution consultant or authorized reseller for assistance. They are most familiar with your personal setup and can best help you with tips and product training. For contact details, visit <https://www.tobiidynavox.com/pages/contact-us>.